TensioFlash®

FR

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Distribué par / Distributed by
www.visiomed-lab.com

Marché Européen / European Market
Visiomed Group SA
Service clients / Customers service
8, avenue Kléber
75116 Paris - France
Tel : +33 8 92 350 334
Fax : +33 1 44 17 93 10
contact@visiomed-lab.fr

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EN

USER MANUAL

**PLEASE CAREFULLY READ THIS MANUAL
BEFORE OPERATING THIS UNIT**

TABLE OF CONTENTS

| | |
|---|-----|
| INTRODUCTION | 19 |
| NOTES ON SAFETY | 19 |
| BLOOD PRESSURE | 20 |
| PRECAUTIONS BEFORE USE | 20 |
| FEATURES OF THE TENSIOFLASH® KD-558 | 21 |
| MAIN UNIT | 22 |
| INSERT OR REPLACE BATTERIES | 22 |
| TIME AND DATE OF SYSTEM SETUP | 23 |
| CONNECTING THE CUFF TO THE MONITOR | 23 |
| A FEW SUGGESTIONS BEFORE MEASUREMENT | 23 |
| HOW TO USE THE UPPER ARM CUFF | 24 |
| BODY POSTURE DURING MEASUREMENT | 24 |
| HOW TO MEASURE BLOOD PRESSURE | 25 |
| RECALLING RECORD | 26 |
| HOW TO MAINTAIN AND STORE THE UNIT | 27 |
| HOW TO PREVENT A MALFUNCTION | 27 |
| SPECIFICATIONS | 28 |
| TROUBLESHOOTING | 29 |
| EXPLANATION OF SYMBOLS ON UNIT | 02 |
| ELECTROMAGNETIC COMPATIBILITY INFORMATION | 155 |
| WARRANTY | 159 |

THE MANUFACTURER KEEPS THE RIGHT TO MODIFY WITHOUT ANY PRELIMINARY OPINION TECHNICAL SPECIFICATIONS OF THE PRODUCT

INTRODUCTION

Thank you for purchasing this Visiomed upper arm Automatic Blood Pressure Monitor. This remarkable instrument is ideal for people who frequently monitor their own blood pressure because it's easy to use. A simple push of a button and the monitor measures your blood pressure and pulse and displays the reading on a clear digital panel. Perfect for quick, easy readings at home, at work, almost anywhere!

The Monitor uses the oscillometric method of blood pressure measurement. This means the monitor detects your blood's movement through the artery in your arm and converts the movements into a digital reading. An oscillometric monitor does not need a stethoscope so the monitor is simple to use and measurements are easy to obtain.

Please read this instruction manual thoroughly before using the upper arm Automatic Blood Pressure Monitor. For specific information on your own blood pressure, CONTACT YOUR PHYSICIAN. The product complies with the electromagnetic compatibility requirement of EN60601-1-2 and safety standards of 60601-1 as specified in EEC directive 93/42/EEC.

Material complies with the European norm concerning non-invasive blood pressure monitors, part 1 General Requirements In-1060-1 and part 3 Additional Requirements for the electromechanical measuring of blood pressure IN-1060-3

NOTES ON SAFETY

- Do not use the unit for purpose other than measuring blood pressure; it may cause accident or trouble.
- Do not inflate the upper arm cuff when it is not wrapped on the upper arm.
- Do not use force to bend or pull the cuff. Always use the specified accessories in the manual, the use of other parts not approved by the manufacturer may cause hazard.
- Do not knock or drop the main unit.
- Do not use cellular phone near the unit. The main unit may operate erroneously.
- Do not disassemble, repair or remodel the main unit or the cuff of the blood pressure monitor. It will cause the unit to function erroneously. Remove the batteries if you are not going to use the unit for a long period of time.
- Do not insert the batteries with their polarities in the wrong direction.
- It is inappropriate for people with serious arrhythmia to use this device.

BLOOD PRESSURE

CONSULT YOUR DOCTOR TO DETERMINE
YOUR NORMAL BLOOD PRESSURE

Categories for Blood Pressure Levels in Adults* (In mmHg, millimeters of mercury)

| Category | Systolic (Top number) | Diastolic (Bottom number) | Colour indication WHO |
|-------------------------|--------------------------|------------------------------|--------------------------|
| Optimal | Less than 120 | Less than 80 | Green |
| Normal | 120-129 | 80-84 | Green |
| Prehypertension | 130-139 | 85-89 | Green |
| Hypertension stage 1 | 140-159 | 90-99 | Yellow |
| Hypertension stage 2 | 160-179 | 100-109 | Orange |
| Hypertension stage 3 | Higher than 180 | Higher than 110 | Red |

* For adults 18 and older who:

Are not on medicine for high blood pressure

Are not having a short-term serious illness

Do not have other conditions such as diabetes and kidney disease

Note: There is not an universally accepted definition of hypotension (low blood pressure), but a systolic pressure below 99 mmHg is usually regarded as hypotension.

PRECAUTIONS BEFORE USE

- Read all of the information in the operation guide.
- Do not confuse self-monitoring with self-diagnoses. Blood pressure measurements should only be interpreted by a health professional who is familiar with your medical history.
- If you are taking medication, consult with your physician to determine the most appropriate time to measure your blood pressure. NEVER change a prescribed medication without first consulting with your physician.
- This product should not encourage self-medication or adaptation of your treatment.
- This device is designed for adults to use if this device is used on a child patient, old people or toddlers, consult with your physician.
- Measurements may be impaired if this device is used near televisions, microwave ovens, X-ray mobile phone equipment or other devices with strong electrical fields. To prevent such


interference, use the meter at a sufficient distance from such devices or turn them off.

- If there is overmuch pressure or you feel uncomfortable, please press *START* button immediately for quick deflation.
- Do not use this unit in a moving vehicle; this may result in erroneous measurement.
- Blood pressure measurements determined by this monitor are equivalent to those obtained by a trained observer using the cuff/stethoscope auscultation method, within the limits prescribed by the American National Standard Institute, Electronic or automated sphygmomanometers.

FEATURES OF THE TENSIOFLASH® KD-558

1. Oscillometric method
2. Silicon integrated pressure sensor
3. Automatic blood pressure and pulse rate measurements
4. Irregular heartbeat detection
5. The most recent 2 x 30 measurements can be stored in 2 memory banks
6. Easy to use buttons
7. Wide LCD screen for easy reading
8. Clear display of measurements: date/time, systolic pressure, diastolic pressure, pulse
9. Display the average of the last 3 measurements
10. Automatic inflation/ deflation by electric pump
11. The cuff is easy to wrap
12. Automatically power off

MAIN UNIT

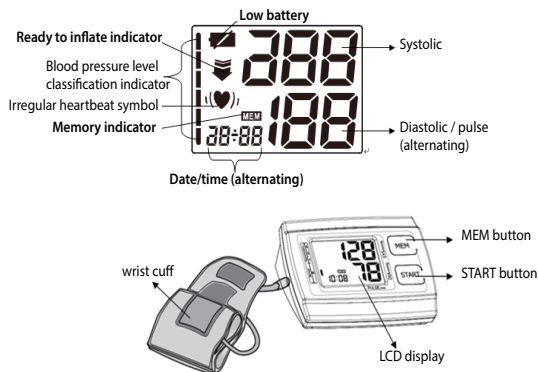INSERT OR REPLACE BATTERIES

When LCD shows battery symbol, replace all batteries with new ones.

1. Slide the battery cover in the direction of the arrow to open.
2. Insert four alkaline AA batteries in the correct polarity direction. Inserting them incorrectly may damage the device and void the warranty. Never use rechargeable batteries. Use single use batteries.
3. Replace the battery cover.

Disposal of empty battery to the authorized collecting party subject to the regulation of each individual territory.

With the use of four alkaline "AA" batteries, you can measure approximately 250 times at room temperature of 72°F (22°C) and inflating to around 170mmHg once a day. But the included batteries only for test purpose may not last as long as the specified period.

If the unit will not be used for a long period, please take out all batteries to avoid liquid leakage damaging the unit.


TIME AND DATE OF SYSTEM SETUP

1. Press the *START* and *MEM* button after a beep the month will blink.
2. Use the *MEM* button to adjust the month.
3. Press the *START* button to save your settings and display the day.
4. Use the *MEM* button to adjust the day.
5. Follow the steps to adjust the time then the minute.
6. The monitor will turn off automatically after 1 minute of inactivity.
7. Once you change the batteries, you should readjust the time and the date.

CONNECTING THE CUFF TO THE MONITOR

Insert the cuff tubing connector into the socket in the left side of the monitor. Make certain that the connector is completely inserted to avoid air leakage during blood pressure measurements.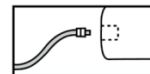



 Avoid compression or restriction of the connection tubing during measurement. This may cause inflation error, or harmful injury due to continuous cuff pressure.

A FEW SUGGESTIONS BEFORE MEASUREMENT

1. Avoid eating, smoking, and exercising for at least 30 minutes before taking a measurement. Also rest for at least 15 minutes before taking a reading.
2. Stress raises blood pressure. Avoid taking measurements during stressful times.
3. You would better apply the cuff on your left upper arm.
4. Measurement should be taken in a quiet place and you should be relaxed in a seated position. Rest your left arm on a table.
5. Remain still and do not talk during the measurement.
6. Wait 5-10 minutes between successive measurements. Waiting allows the engorged blood vessels to return to normal. You may require more rest time depending on your individual physical conditions.
7. Please refer to the cuff circumference range in "SPECIFICATIONS" to make sure that the appropriate cuff is used.

HOW TO USE THE UPPER ARM CUFF

1. Pass the end of the cuff (with sewn-in rubber stopper) through the metal stirrup so that a loop is formed. The Velcro closer must be facing outwards. (Ignore this step if the cuff has already been prepared.)
2. Push the cuff over the left upper arm so that the tube points in the direction of the lower arm and palm.
3. Lay the cuff on the arm as illustrated. Make certain that the lower edge of the cuff lies approximately 2 to 3 cm (3/4~1") above the elbow and that the rubber tube leaves the cuff on the inner side of the arm.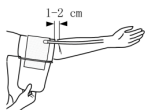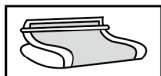

Important: The mark (about 3 cm – 1.2in long bar) must lie exactly over the artery which runs down the inner side of the arm.

4. Tighten the free end of the cuff and close the cuff with the closer.
5. There must be no free space between the arm and the cuff as this would influence the result. Clothing must not restrict the arm. Any piece of clothing which does (e.g. a pullover) must be taken off.
6. Secure the cuff with the Velcro closer in such a way that it lies comfortably and is not too tight. Lay the arm on the table (palm upwards) so that the cuff is at the same height as the heart. Make sure that the tube is not kinked.
7. Remain seated quietly for two minutes before you begin the measurement.

Note:

- If it is not possible to fit the cuff to the left arm, it can also be placed on the right arm. However all measurements should be made using the same arm for better cross reference.
- If the cuff is dirty, clean it with a damp cloth.

BODY POSTURE DURING MEASUREMENT


Sitting Comfortably Measurement:

- a) Be seated with your feet flat on the floor, and don't cross your legs
- b) Place palm upside in front of you on a flat surface such as a desk or table.
- c) The middle of the cuff should be at the level of the right atrium of the heart.

Lying Down Measurement:

- a) Lie on your back.
- b) Place your arm straight along your side with your palm upside.
- c) The cuff should be placed at the same level as your heart.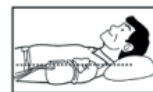


HOW TO MEASURE BLOOD PRESSURE

1. Apply the cuff as explained in the previous chapter
2. Press the **START** button. All the display characters are shown during 2 seconds. The current memory bank (U1 or U2) is blinking. Press **MEM** button to change over to other bank. Press **START** to confirm your selection.

3. The monitor starts to seek zero pressure. The monitor inflates the cuff until sufficient pressure has built up for a measurement. Then the monitor slowly releases air from the cuff and carries out the measurement. Finally the blood pressure and pulse rate will be calculated and displayed on the LCD screen separately. Irregular heartbeat symbol (if any) will blink.
4. After measurement, the result of systolic and diastolic pressure and pulse rate will all display on LCD.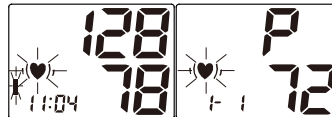

5. The result will be automatically stored in the current memory bank.
6. During measurement, you can press the **START** button to turn off the monitor manually.

Note: The value measured will be too low if you hold your cuff higher than heart level, or will be too high if you hold your cuff lower than heart level.

- During measurement do not speak or shake as it may affect the accuracy of the result.
- If you want to stop the measurement during measurement process, just press the **START** button to stop immediately.
- Please consult a health care professional for interpretation of pressure measurements.

RECALLING RECORD

This device contains 2 units of 30 memories each (U1 and U2). During the measurement, select the storage unit.


1. Press the **MEM** button to display the stored results. Now the LCD displays the amount of the results in the current bank. Press **START** button to change over to other bank. Confirm your selection by pressing **MEM** button. The current bank can also be confirmed automatically after 5 seconds with no operation.

2. The LCD will display the average value of the last three results in this bank. If no result stored, LCD will show dashes « - »

3. Press **MEM** button to review the result from the most recent to the oldest.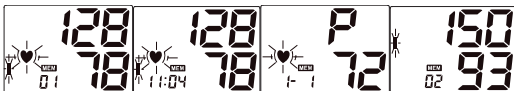

4. Press the button **START** to turn off the monitor manually, the monitor will turn off automatically after 1 minute of no operation.

Deleting measurements from the memory:
When the LCD displays the stored results in a bank, press the **MEM** button during three seconds. All results in the current memory bank will be deleted after three « beep ». Press the **MEM** or **START** button to turn of the monitor.

HOW TO MAINTAIN AND STORE THE UNIT

1. Use a dry soft cloth to clean the unit. If necessary, use a cloth lightly dampened with tap water.
2. Do not use alcohol, benzene, thinner or other harsh chemicals to clean the device or cuff.
3. Before using please wash your hand. Do not wash or wet the cuff.
4. Avoid placing the unit where direct sunlight, high humidity or dust.

Note: When the unit will not be used for a long period, be sure to remove the batteries. Otherwise, the batteries may leak and damage the unit

HOW TO PREVENT A MALFUNCTION

- Do not drop the device and avoid sudden jars or shocks.
- Do not insert other objects into any holes.
- Do not attempt to disassemble the unit.
- Do not crush the pressure cuff.
- The monitor might not meet its performance specifications or cause safety hazard if stored or used outside the specified temperature and humidity ranges in specifications.
- The monitor can maintain the safety and performance characteristics for a minimum of 10,000 measurements.

SPECIFICATIONS

| | |
|--------------------------|---|
| Product name | AUTOMATIC BLOOD PRESSURE MONITOR ARMBAND |
| Model | KD-558 |
| Measuring Method | Oscillometric System |
| Classification | Classe IIa |
| Indication | Digital LCD Display |
| Measuring range | Cuff pressure: 0~300 mmHg. Systolic : 60~260 mmHg Diastolic : 40~199 mmHg Pulse: 40~180 Beat/min. |
| Accuracy | Pressure +/- 3 mmHg Pulse +/- 5% |
| Memory | 2x30 measurements stored in 2 memory banks |
| Power source | 1.5V "AA" x 4 Alkaline Battery |
| Battery life | Approx 500 measurements |
| Working Temp. & humidity | 10~40°C (50~104°F). ≤90% RH |
| Storage Temp. & humidity | -20~55°F (-4~131°F). ≤90% RH Pression : 80KPa-105KPa |
| Outside dimensions | 138 mm (La)/ 98 mm (Lo)/ 48 mm (H) (5 7/16" X3 27/32" X1 7/8") |
| weight | Approx 211 g (Apparatus and Battery) (7 7/16 oz.) |
| Accessories | Cuff: fit arm 22~32 cm (8,7 - 12,7 in) 30~42cm (11 13/16" - 16 17/32") (optional) 42~48cm (16 17/32" - 18 29/32") (optional). User manual, carrying case, 4 batteries 1.5V AA |

Specifications may be changed without notice in the event of improvement being made

TROUBLESHOOTING

If you have one of the following problems, please refer to this breakdown service guide to help resolve the problem. If the problem persists, please contact our customer service at +33 892 350 334 (0,34€/mn).

| PROBLEM | POSSIBLE CAUSE | SOLUTION |
|-----------------------------------|--|---|
| LCD Display shows abnormal result | The cuff position was not correct or it was not properly tightened | Apply the cuff correctly and try again |
| | Body posture was not correct during testing | Review the "BODY POSTURE DURING MEASUREMENT" sections of the instructions and re-test. |
| | Speaking, arm or body movement, angry, excited or nervous during testing | Re-test when calm and without speaking or moving during the test |
| | Irregular heartbeat (arrhythmia) | It is inappropriate for people with serious arrhythmia to use this Electronic Sphygmomanometer. |
| LCD shows low battery symbol | Low Battery | Change the batteries |
| LCD shows "Er 0" | Pressure system is unstable before measurement | Don't move and try again. |
| LCD shows "Er 1" | Fail to detect systolic pressure | |
| LCD shows "Er 2" | Fail to detect diastolic pressure | |
| LCD shows "Er 3" | Pneumatic system blocked or cuff is too tight during inflation | Apply the cuff correctly and try again |
| LCD shows "Er 4" | Pneumatic system leakage or cuff is too loose during inflation | |

| | | |
|--|---|--|
| LCD shows "Er 5" | Cuff pressure above 300mmHg | Measure again after five minutes. If the monitor is still abnormal, please contact the local distributor or the factory. |
| LCD shows "Er 6" | More than 3 minutes with cuff pressure above 15 mmHg | |
| LCD shows "Er 7" | EEPROM accessing error | |
| LCD shows "Er 8" | Device parameter checking error | |
| LCD shows "Er A" | Pressure sensor parameter error | |
| No response when you press button or load battery. | Incorrect operation or strong electromagnetic interference. | Take out batteries for five minutes, and then reinstall all batteries. |

Note: Relax for at least 10 minutes and then take another measurement.

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EMC DATA

ELECTROMAGNETIC COMPATIBILITY INFORMATION

Table 1
For all ME EQUIPMENT and ME SYSTEMS

| Guidance and manufacturer's declaration - electromagnetic emissions | | |
|---|----------------|--|
| The [KD-558] is intended for use in the electromagnetic environment specified below. The customer or the user of the [KD-558] should assure that it is used in such an environment. | | |
| Emissions test | Compliance | Electromagnetic environment - guidance |
| RF emissions CISPR 11 | Group 1 | The [KD-558] uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment. |
| RF emissions CISPR 11 | Class B | The [KD-558] is suitable for use in all establishments other than domestic and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes. |
| Harmonic emissions IEC 61000-3-2 | Not applicable | |
| Voltage fluctuations/ flicker emissions IEC 61000-3-3 | Not applicable | |

Table 2
For all ME EQUIPMENT and ME SYSTEMS

| Guidance and manufacturer's declaration - electromagnetic immunity | | | |
|---|------------------------------|------------------------------|--|
| The [KD-558] is intended for use in the electromagnetic environment specified below. The customer or the user of the [KD-558] should assure that it is used in such an environment. | | | |
| IMMUNITY test | IEC 60601test level | Compliance level | Electromagnetic environment - guidance |
| Electrostatic discharge (ESD) IEC 61000-4-2 | ± 6 kV contact ± 8 kV air | ± 6 kV contact ± 8 kV air | Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30 %. |
| Power frequency (50/60 Hz) magnetic field IEC 61000-4-8 | 3 A/m | 3 A/m | Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment. |

Table 3
For all ME EQUIPMENT and ME SYSTEMS that are not LIFE-SUPPORTING

| Guidance and manufacturer's declaration - electromagnetic immunity | | | |
|---|------------------------|------------------|--|
| The [KD-558] is intended for use in the electromagnetic environment specified below. The customer or the user of the [KD-558] should assure that it is used in such an environment. | | | |
| IMMUNITY test | IEC 60601test level | Compliance level | Electromagnetic environment - guidance |
| RADIATED RF IEC 61000-4-3 | 3V/m 80 MHz to 2.5 GHz | 3 V/m | Portable and mobile RF communications equipment should be used no closer to any part of the [KD-558], including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter. Recommended separation distance: $d = 1.2 \sqrt{P}$ 80 MHz to 800 MHz $d = 2.3 \sqrt{P}$ 800 MHz to 2,5 GHz Where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in meters (m). Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey, a should be less than the compliance level in each frequency range. b Interference may occur in the vicinity of equipment marked with the following symbol: |
| <p>NOTE 1 At 80 MHz and 800 MHz, the higher frequency range applies.</p> <p>NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.</p> <p>a Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the [KD-558] is used exceeds the applicable RF compliance level above, the [KD-558] should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocating the [KD-558].</p> <p>b Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3V/m.</p> | | | |

Table 4
For ME EQUIPMENT and ME SYSTEMS that are not LIFE-SUPPORTING

| Recommended separation distances between Portable and mobile RF communications equipment and the [KD-558] | | | |
|--|---|---|--|
| The [KD-558] is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the [KD-558] can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the [KD-558] as recommended below, according to the maximum output power of the communications equipment | | | |
| Rated maximum output power of transmitter W | | Separation distance according to frequency of transmitter m | |
| | 150 kHz to 80 MHz $d = 1.2 \sqrt{P}$ | 80 MHz to 800 MHz $d = 1.2 \sqrt{P}$ | 800 MHz to 2,5 GHz $d = 2.3 \sqrt{P}$ |
| 0.01 | 0.12 | 0.12 | 0.23 |
| 0.1 | 0.38 | 0.38 | 0.73 |
| 1 | 1.2 | 1.2 | 2.3 |
| 10 | 3.8 | 3.8 | 7.3 |
| 100 | 12 | 12 | 23 |
| For transmitters rated at a maximum output power not listed above, the recommended separation distance d in meters (m) can be determined using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer. NOTE 1 At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies. NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people. | | | |

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EN: Visiomed* will repair or replace this product free of charge in the case of defective parts or manufacturing defects, in accordance with the conditions mentioned below as follows:

DURATION: 24 MONTHS RETURN TO WORKSHOP

LIMITS AND EXCLUSIONS: This guarantee concerns only the original final purchaser. A purchase invoice, or another proof of purchase, with this guarantee card will be required to obtain an after-sales service, in accordance with this guarantee. This guarantee card will not be extended to another person only the original final purchaser. This guarantee becomes void if the serial numbers on the product are modified, replaced, illegible, absent, or if repair has been carried out by a service not approved, including the user. This guarantee covers only the defects of the material or parts, occurring during normal use of the product. It does not cover the damage caused during the transport of the apparatus, causes due to repairs being carried

out by the distributor, by any modifications undertaken, any connection of equipment not approved by Visiomed*, or causes contrary to those written in the user manual or notice. Moreover, the present guarantee does not cover damage due to falls, bad handling, bad installations, damage by fire, floods, lightning, or any other natural disaster. This guarantee does not cover the packing of the material, the accessories, the defects caused by commercial exposure of the product, show room, sale space, demonstration etc... Normal maintenance, cleaning and the replacement of parts where wear is normal, are not covered by the terms of this guarantee. Visiomed* and its representatives and agents will not in any case be held responsible for any damage and consecutive damages due to the mishandling of this product. This guarantee is the only valid one at Visiomed*, any other guarantee (commercial guarantee) except this one will not be taken into account. IMPORTANT: During the guarantee period if you are dissatisfied with the repairs of this product, please contact the Visiomed* customer service.

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EN: Disposal of Waste Equipment by Users in Private Households in the European Union. The symbol on the product or on its packaging indicates that this product must not be disposed of with your other household waste. Instead, it is your responsibility to dispose of your waste equipment by handing it over to a designated collection point for the recycling of waste electrical and electronic equipment. The separate collection and recycling of your waste equipment at the time of disposal will help to conserve natural resources and ensure that it is recycled in a manner that protects human health and the environment. For more information about where you can drop off your waste equipment for recycling, please contact your local city office or your household waste disposal service.

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© 03/2016 KD-558
EN: TensioFlash® is a brand registred by Visiomed® Group SA. The constructor keeps the right to modify the photographs and characteristics without previous notice. Registered design. The logos and trademarks are the property of their respective owners.