FR

TensioFlash®

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Distribué par / Distributed by
www.visiomed-lab.com

Marché Européen / European Market
Visiomed Group SA
Service clients / Customers service
8, avenue Kléber
75116 Paris - France
Tel : +33 8 92 350 334
Fax : +33 1 44 17 93 10
contact@visiomed-lab.fr

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EN

USER MANUAL

**PLEASE CAREFULLY READ THIS MANUAL
BEFORE OPERATING THIS UNIT**

TABLES OF CONTENTS

INTRODUCTION.....	17
SAFETY INSTRUCTION	17
BLOOD PRESSURE.....	18
PRECAUTIONS BEFORE USE	20
FEATURES OF THE TENSIOLASH® KD-595	21
MAIN UNIT.....	21
INSERT OR REPLACE BATTERIES	22
TIME AND DATE OF SYSTEM SETUP	22
VOICE SETTING.....	23
A FEW SUGGESTIONS BEFORE MEASUREMENT	23
HOW TO USE THE UPPER ARM CUFF.....	24
HOW TO MEASURE BLOOD PRESSURE.....	25
RECALLING RECORD.....	25
HOW TO MAINTAIN AND STORE THE UNIT	26
HOW TO PREVENT A MALFUNCTION.....	27
SPECIFICATIONS.....	27
TROUBLESHOOTING.....	28
WARRANTY	165

The manufacturer saves himself the right to modify without any preliminary opinion technical specifications of the product

INTRODUCTION

Thank you for purchasing the upper arm Automatic Blood Pressure Monitor. This remarkable instrument is ideal for people who frequently monitor their own blood pressure because it's easy to use. Simply push of a button the monitor measures your blood pressure and pulse and displays the reading on a clear digital panel. Perfect for quick, easy readings at home, at work, almost anywhere!

The Monitor uses the oscillometric method of blood pressure measurement. This means the monitor detects your blood's movement through the artery in your arm and converts the movements into a digital reading. An oscillometric monitor does not need a stethoscope so the monitor is simple to use and measurements are easy to obtain.

Please read this instruction manual thoroughly before using the upper arm Automatic Blood Pressure Monitor. For specific information on your own blood pressure, CONTACT YOUR PHYSICIAN.

The product complies with the electromagnetic compatibility requirement of EN60601-1-2 and safety standards of 60601-1 as specified in EEC directive 93/42/EEC.

Material complies with the European norm concerning non invasive blood pressure monitors, part 1 General Requirements In-1060-1 and part 3 Additional Requirements for the electromechanical measuring of blood pressure IN-1060-3.

SAFETY INSTRUCTION

- Follow the care instructions included in these instructions.
- Only use this device for its intended purpose, as described in these instructions.
- This device can be used for personal use at home.
- Use this device within an ambient temperature range of between 10 and 40°C.
- This device must always be placed in a clean and dry place.
- Do not expose this device to extreme temperature conditions $T^{\circ} > 50^{\circ}\text{C}$ - $T^{\circ} < - 20^{\circ}\text{C}$.
- Do not use this device when relative humidity is greater than 85%.
- Do not expose this device to the sun, nor to water.
- Do not use this device outdoors.
- Do not expose this device to electric shocks.
- Never drop this device.
- Do not carry out maintenance on this device yourself. Do not try to open this device. In case of problems, contact your dealer.

- Do not leave this device within reach of children.
- Measurement results are only indicative. In case of doubt concerning your results, please contact your doctor.
- This device is not an encouragement to self-medication or to make adjustments to your treatment.
- Stop using this device in the event of a fault or if it malfunctions.
- This electrical medical equipment requires specific precautions concerning electromagnetic compatibility. It must be installed and operated in accordance with the electro-magnetic information supplied in the EMC Data table.

 **WARNING:** You cannot dispose of this product in the same way as your common waste. You are required to take it to a collection point approved for the recycling of used electrical and electronic equipment. For further information on collection facilities for used equipment, please contact your town hall or your household waste processing service.

BLOOD PRESSURE

Blood pressure is the force exerted on the walls of your blood vessels as blood flows through them.

Your heart is like a pump. When it contracts, or beats, it sends a surge of blood through the blood vessels and pressure increases. This is called your systolic pressure.


When your heart relaxes between beats, your blood pressure decreases. This is called your diastolic pressure.

When a doctor takes your blood pressure, he or she measures both your systolic and diastolic pressures and records them as numbers. For example, if your blood pressure reading is 126/76 (126 over 76), your systolic is 126 and your diastolic is 76. The numbers are calculated in millimetres of mercury and recorded as 126/76 mm Hg. These two numbers provide important information about your health. The more difficult it is for your blood to flow through your blood vessels, the higher both numbers will be. When blood pressure is consistently above normal it is called hypertension (High blood pressure).

The **TensioFlash®** automatically measures your systolic and dias-

Heart Contracts
(Pressure Increases)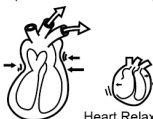

Heart Relaxes
(Pressure Decreases)

Systolic
(Upper Number)

126

76

Diastolic
(Lower Number)


tolic blood pressure for you, then displays the results on easy-to-read digital panel.

People with a condition that causes circulatory problems (diabetes, kidney disease, arteriosclerosis, or poor peripheral circulation) may get lower readings with upper arm type monitor than with a blood pressure monitor that is used on the upper arm.

Please consult your doctor to determine if your upper arm blood pressure accurately reflects your actual blood pressure.

Your blood pressure changes constantly. Blood pressure fluctuates from day to day and minute to minute according to your body's needs. For example, when you are exercising or angry your blood pressure increases, but when you are relaxing or sleeping your blood pressure decreases. These fluctuations are completely normal. This portable upper arm blood pressure monitor is especially helpful when tacking changes in your blood pressure because it travels anywhere and takes just seconds to use.

Fluctuation within a Day (Example: 35-year old male)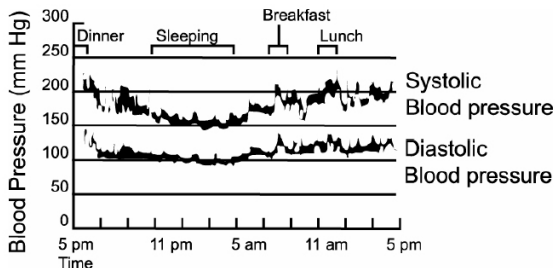

Factors that may cause the blood pressure to fluctuate: Breathing, Exercise, Mental stress, Worries, Environmental and/or temperature change, Eating, Talking, Taking a bath, Drinking alcohol, Smoking, etc...

Note: One or two readings will not provide a true indication of your normal blood pressure. It is very important to take regular, daily measurements and to keep accurate records. In partnership with your doctor, an accurate record of your blood pressure over a period of time can be a valuable aid in diagnosing and preventing potential health problems.

The World Health Organisation (WHO) developed the following Blood Pressure Classification. This classification, however, is only a general guideline because blood pressure varies from person to person according to age, weight, and health status.

CONSULT YOUR DOCTOR TO DETERMINE YOUR NORMAL BLOOD PRESSURE.

Categories for Blood Pressure Levels in Adults
(In mmHg, millimeters of mercury)

Category	Systolic (Top number)	Diastolic (Bottom number)	Color indication
Optimal	Less than 120	Less than 80	Green ■
Normal	120-129	80-84	Green ■
Prehypertension	130-139	85-89	Green ■
Hypertension stage 1	140-159	90-99	Yellow ■
Hypertension stage 2	160-179	100-109	orange ■
Hypertension stage 3	Higher than 180	Higher than 110	Red ■

*For adults 18 and older who:
Are not on medicine for high blood pressure
Are not having a short-term serious illness
Do not have other conditions such as diabetes and kidney disease*

Note: There is not an universally accepted definition of hypotension (low blood pressure), but a systolic pressure below 99 mmHg is usually regarded as hypotension.

PRECAUTIONS BEFORE USE


- Do not confuse self-monitoring with self-diagnoses. Blood pressure measurements should only be interpreted by a health professional who is familiar with your medical history.
- If you are taking medication, consult with your physician to determine the most appropriate time to measure your blood pressure. NEVER change a prescribed medication without first consulting with your physician.
- For persons with irregular or unstable peripheral circulation problems due to diabetes, liver disease, hardening of the arteries, etc., there may be fluctuation in blood pressure values measured at the upper arm versus at the upper arm .
- This device is designed for adults to use if use this device on a child patient, old people or toddlers, consult with your physician or measure with relative aid.
- Measurements may be impaired if this device is use near televisions, microwave ovens, X-ray mobile phone equipment

- or other devices with strong electrical fields. To prevent such interference, use the meter at a sufficient distance from such devices or turn them off.
- This device is not suitable for disease diagnoses, urgent treatment, continuous monitoring during medical emergencies or operations.
- Before using, should wash your hand.
- If there is overmuch pressure or you feel uncomfortable, please press «POWER» button immediately for quick deflation.

FEATURES OF THE TENSIOFLASH® KD-595

1. Using the fuzzy-logic measurement.
2. Detection irregular heartbeat IHB function
3. Memory can store 60 measurements.
4. Talking measurement in 6 languages: French, English, Spanish, German, Arabic, Dutch
5. Easy to use buttons.
6. Large and clear display.
7. Upper arm cuff is easy to wrap correctly.
8. Displays the last reading when the device is switched on.
9. Automatic Inflation / Deflation via an electric pump.
10. Desktop style for easy measurement.
11. Easy to use, Press a button to automatically measure, recording the measurement values and measurement time.
12. Only inflate the armband on the arm.
13. Compact size and easy to carry.
14. Automatically power off (within 2.5 minutes) to save power.

MAIN UNIT

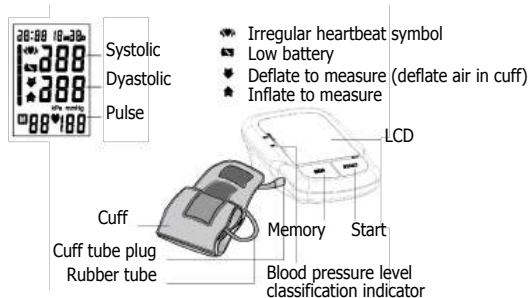
INSERT OR REPLACE BATTERIES

During the battery symbol appears on the display or nothing is display while inflation during measurement, you should replace with new batteries.

1. Slide the battery cover in the direction of the arrow to open.
2. Insert four alkaline AA batteries in the correct polarity direction.
3. Replace the battery cover.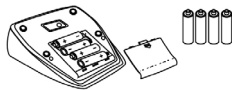


Disposal of empty battery to the authorized Collecting party subject to the regulation of Each individual territory.

With the use of four alkaline «AA» batteries, you can measure approximately 250 times at room temperature of 72°F (22?) and inflating to around 170mmHg once a day. But the included batteries only for test purpose may not last as specified long period. When the battery icon displays on the LCD, please replace four new alkaline «AA» batteries, please do not mix the new and old battery together.

If the unit will not be used for a long period, please take out all batteries to avoid liquid leakage damaging the unit.

TIME AND DATE OF SYSTEM SETUP

1. When the machine is turned off, press the button *START* and *MEMORY* at same time, the number of month will glitter on LCD. Press button *START* continuously, the numbers of month, date, hour, and minute will glitter. While the number is glittering, press button *MEMORY* the corresponding number will increase by 1. Hold on pressuring the button *MEMORY* the number will increase fast.
2. After pressuring *START* button the fifth time, numbers on LCD will not glitter, then you exit the time adjusting mode. See picture.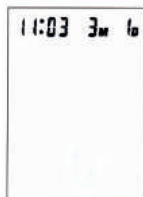

Noted: Please check and re-set Date/Time after replacing of new batteries.


VOICE SETTING

1. Voice language setting: In clock display mode, you can select the voice language or without talking function by keeping on press the button *M*. You can select the wanted language by releasing button *M* when display the corresponding language.

There are 6 languages: French, English, Spanish, German, Arabic, Dutch

2. Voice volume setting: You can select the voice volume by pressing the button *M* again and keeping after language setting. Now the bars on the LCD change and the number of the bars indicates the volume. You can select the wanted volume by releasing button *M* when display the corresponding bars.

3. Once you choose the language and the volume, the monitor will speak out measurement results and tips with that language during the measurement process. The voice setting is kept until you reset it.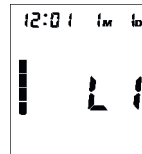

A FEW SUGGESTIONS BEFORE MEASUREMENT

1. Avoid eating, smoking, and exercising for at least 30 minutes before taking a measurement. Also rest for at least 15 minutes before taking a reading.
2. Stress raises blood pressure. Avoid taking measurements during stressful times.
3. You would better apply the cuff on your left upper arm.
4. Measurement should be taken in a quiet place and you should be relaxed, seated position. Rest your left arm on a table.
5. Remain still and do not talk during the measurement.
6. Keep a record of your blood pressure and pulse for your doctor. Remember, a single measurement does not provide an accurate indication of your true blood pressure. You need to take and record several measurements over a period of time. Try to measure your blood pressure at the same time each day for consistency. Blood pressure measurements fluctuate considerably.
7. Wait 5-10 minutes between successive measurements. Waiting allows the engorged blood vessels to return to normal. You may require more rest time depending on your individual physical conditions.

HOW TO USE THE UPPER ARM CUFF


1. Pass the end of the cuff (with sewn-in rubber stopper) through the metal stirrup so that a loop is formed. The Velcro closer must be facing outwards. (Ignore this step if the cuff has already been prepared.)
2. Push the cuff over the left upper arm so that the tube points in the direction of the lower arm and palm.
3. Lay the cuff on the arm as illustrated. Make certain that the lower edge of the cuff lies approximately 2 to 3 cm (3/4~1») above the elbow and that the rubber tube leaves the cuff on the inner side of the arm.

Important: The mark (about 3 cm – 1.2in long bar) must lie exactly over the artery which runs down the inner side of the arm.

4. Tighten the free end of the cuff and close the cuff with the closer.
5. There must be no free space between the arm and the cuff as this would influence the result. Clothing must not restrict the arm. Any piece of clothing which does (e.g. a pullover) must be taken off.
6. Secure the cuff with the Velcro closer in such a way that it lies comfortably and is not too tight. Lay the arm on the table (palm upwards) so that the cuff is at the same height as the heart. Make sure that the tube is not kinked.
7. Remain seated quietly for two minutes before you begin the measurement.

COMMENT: If it is not possible to fit the cuff to the left arm, it can also be placed on the right arm.

However all measurements should be made using the same arm for better cross reference.

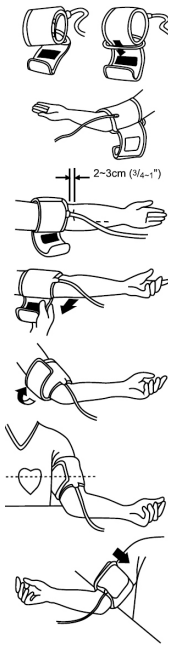HOW TO MEASURE BLOOD PRESSURE

1. After your body is in a comfortable position, press the **START** button. After a long “beep”, BPM will verify itself, and all LCD will glimmer automatically. See picture.
2. If it already has memory, then LCD will show the last result. If machine does not have memory before, the LCD will show of four “0”, while it verify itself.
3. If the voice function is on, the monitor will speak out the measurement tips with that language.

After two seconds (maybe longer because of verifying itself), LCD shows one “0”. If the voice function is on, the monitor will speak “Now if you are ready, Let’s begin” then the machine begin to pressurize automatically.

4. After measurement, the result of systolic, diastolic and pulse rate will all display on LCD. And irregular heartbeat symbol and blood pressure classification indicator will glimmer. The result will be automatically memorized by itself.

If the voice function is on, for example, the monitor will speak out “Your systolic pressure is 156 mmHg, your diastolic pressure is 93 mmHg, the heart rate is 78 beat/minute, according to WHO standard, your blood pressure level is Moderate Hypertension”.

5. Pressing **START** button after measuring, machine will be turned off. It will be automatically power off after 1 minute of none use.

Note:

- The value measured will be too low if you hold your cuff higher than heart level, or will be too high if you hold your cuff lower than heart level.
- The unit may store 60 measurement values, If more than 60 records, it will delete the earliest data.
- During measurement do not speak or shake as which may affect the accuracy of the result.
- If you want to stop the measurement during measurement process, just press the **POWER** button to stop immediately.

RECALLING RECORD

1. Under the clock-displaying mode, press button **M** and enter the mode of reading results. LCD will show the how many results it has, then the last result will be displayed, which is 01 for memory number, see picture 1. Continuously pressure the **M** button, the memory number will increase by 1, see picture 2.

- If the voice function is on, for example picture 8, the monitor will speak out "Memory 01, your systolic pressure is 156 mmHg, your diastolic pressure is 93 mmHg, the heart rate is 78 beat/minute, according to WHO standard, your blood pressure level is Moderate Hypertension."
- Keep on pressing *M* button after the last memory of result, the machine will power off. Under the memory-displaying mode, it will be automatically power off after ten seconds of none use. Or you can press *START* button to shut down the machine.
- If it has no memory, then LCD will show "0" for systolic, diastolic, and pulse rate, and show "00" for the number of memory. While press the *MEMORY* or *START*, machine will power off.
- Deletion of memories: Under the mode of reading memories, holding on pressing *MEMORY* button for three seconds. All memories will be deleted after three "beep", then come to the mode without memory.
- Shut down: After measurement, press button *START* to turn off the machine; the machine will be automatically power off after 1 minute of none use.

HOW TO MAINTAIN AND STORE THE UNIT

- Use a dry soft cloth to clean the unit. If necessary, use a cloth lightly dampened with tap water.
- Do not use alcohol, benzene, thinner or other harsh chemicals to clean the device or cuff.
- Before using please wash your hand. Do not wash or wet the cuff.

- Avoid placing the unit where direct sunlight, high humidity or dust.

Note: When the unit will not be used for a long period, be sure to remove the batteries. Otherwise, the batteries may leak and damage the unit.

HOW TO PREVENT A MALFUNCTION

- Do not drop the device and avoid sudden jars or shocks.
- Do not inset other objects into any holes.
- Do not attempt to disassemble the unit.
- Do not crush the pressure cuff.
- If the unit has been stored at temperatures below 32°F (0°C), leave it in a warm place for 30 minutes before using it, otherwise, may get wrong measurement result.

SPECIFICATIONS

Classification	Class IIa
Measuring Method:	Oscillometric System
Indication:	Digital LCD Display
Measuring Range:	Cuff pressure: 0~300 mmHg. Systolic: 60~260 mmHg Diastolic: 40~199 mmHg Pulse: 40~180 Beat/min.
Accuracy:	Pressure: +/- 3 mmHg; Pulse +/- 5 %
Inflation:	Electric Pump Inflation
Deflation:	Pressure Release Valve
Memory:	60 Memories
Power Source:	1.5V "AAA" x 4 Alkaline Battery
Battery Life:	Approx. 6 month - 180 times
Working Temp. & Humidity:	41~104°F (5~40°C), ≤90RH
Storage Temp. & Humidity:	-4~131°F (-20~55°C), ≤90RH
Outside Dimensions:	153mm x 108mm x 60mm - (6 1/32" x 4 1/4" x 2 3/8")

Weight:	Approx. 290 g (Apparatus)
Accessories:	Cuff (fit arm size 8.7~12.6in / 22~32cm), Instruction Manual, 4 AA batteries
Operation Voltage	6V DC

***Specifications may be changed without notice in the event of Improvement being made.**

TROUBLESHOOTING

If you have trouble while using the unit, please check the following points first.

ERROR DISPLAY	POSSIBLE CAUSE	HOW TO CORRECT
Nothing is displayed when you push the POWER button	No battery installation	Insert batteries
	Battery worn out	Replace new batteries
	The polarities of batteries placed wrongly	Insert battery in the correct polarities
Shows abnormal results - LCD shows "EE"	Check your cuff if any air leakage	Replace upper arm cuff with new one
	Upper arm cuff not properly adjust or placement, hand or body shaking.	Renew adjust upper arm cuff position and tightness, and keep hand still. Re-measurement or send back dealer for re-calibrate pressure
	Cuff too tight	Readjust cuff and measure again
	Cuff too loose	
Battery icon on	Battery low power	Replace battery, and measure again
The systolic pressure value or diastolic pressure value too high	The hand with the cuff was held lower than your heart	Keeping correct position and gesture to measure again
	The cuff was not attached properly	
	You moved your body or spoke during measurement	
The systolic pressure value or diastolic pressure value too low	The hand with the cuff was held higher than your heart	Keeping correct position and gesture to measure again
	You moved your body or spoke during measurement.	

Note: Relax for at least 10 minutes and then take another measurement.

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EMC DATA

ELECTROMAGNETIC COMPATIBILITY INFORMATION

Table 1
For all ME EQUIPMENT and ME SYSTEMS

Guidance and manufacturer's declaration - electromagnetic emissions		
The [KD-595] is intended for use in the electromagnetic environment specified below. The customer or the user of the [KD-595] should assure that it is used in such an environment.		
Emissions test	Compliance	Electromagnetic environment - guidance
RF emissions CISPR 11	Group 1	The [KD-595] uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	The [KD-595] is suitable for use in all establishments other than domestic and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	Not applicable	
Voltage fluctuations/ flicker emissions IEC 61000-3-3	Not applicable	

Table 2
For all ME EQUIPMENT and ME SYSTEMS

Guidance and manufacturer's declaration - electromagnetic immunity			
The [KD-595] is intended for use in the electromagnetic environment specified below. The customer or the user of the [KD-595] should assure that it is used in such an environment.			
IMMUNITY test	IEC 60601test level	Compliance level	Electromagnetic environment - guidance
Electrostatic discharge (ESD) IEC 61000-4-2	± 6 kV contact ± 8 kV air	± 6 kV contact ± 8 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30 %.
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	3 A/m	3 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.

Table 3
For all ME EQUIPMENT and ME SYSTEMS that are not LIFE-SUPPORTING

Guidance and manufacturer's declaration - electromagnetic immunity			
The [KD-595] is intended for use in the electromagnetic environment specified below. The customer or the user of the [KD-595] should assure that it is used in such an environment.			
IMMUNITY test	IEC 60601test level	Compliance level	Electromagnetic environment - guidance
RADIATED RF IEC 61000-4-3	3V/m 80 MHz to 2.5 GHz	3 V/m	Portable and mobile RF communications equipment should be used no closer to any part of the [KD-595], including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter. Recommended separation distance: $d = 1.2 \sqrt{P}$ 80 MHz to 800 MHz $d = 2.3 \sqrt{P}$ 800 MHz to 2,5 GHz Where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in meters (m). Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey, a should be less than the compliance level in each frequency range. b Interference may occur in the vicinity of equipment marked with the following symbol:
<p>NOTE 1 At 80 MHz and 800 MHz, the higher frequency range applies.</p> <p>NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.</p> <p>a Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the [KD-595] is used exceeds the applicable RF compliance level above, the [KD-595] should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocating the [KD-595].</p> <p>b Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3V/m.</p>			

Table 4
For ME EQUIPMENT and ME SYSTEMS that are not LIFE-SUPPORTING

Recommended separation distances between Portable and mobile RF communications equipment and the [KD-595]			
The [KD-595] is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the [KD-595] can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the [KD-595] as recommended below, according to the maximum output power of the communications equipment			
Rated maximum output power of transmitter W		Separation distance according to frequency of transmitter m	
	150 kHz to 80 MHz $d = 1.2 \sqrt{P}$	80 MHz to 800 MHz $d = 1.2 \sqrt{P}$	800 MHz to 2,5 GHz $d = 2.3 \sqrt{P}$
0.01	0.12	$d = 1.2 \sqrt{P}$	0.23
0.1	0.38	0.38	0.73
1	1.2	1.2	2.3
10	3.8	3.8	7.3
100	12	12	23
For transmitters rated at a maximum output power not listed above, the recommended separation distance d in meters (m) can be determined using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer. NOTE 1 At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies. NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.			

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EN: Visiomed* will repair or replace this product free of charge in the case of defective parts or manufacturing defects, in accordance with the conditions mentioned below as follows:

DURATION: 24 MONTHS RETURN TO WORKSHOP

LIMITS AND EXCLUSIONS: This guarantee concerns only the original final purchaser. A purchase invoice, or another proof of purchase, with this guarantee card will be required to obtain an after-sales service, in accordance with this guarantee. This guarantee card will not be extended to another person only the original final purchaser. This guarantee becomes void if the serial numbers on the product are modified, replaced, illegible, absent, or if repair has been carried out by a service not approved, including the user. This guarantee covers only the defects of the material or parts, occurring during normal use of the product. It does not cover the damage caused during the transport of the apparatus, causes due to repairs being carried

out by the distributor, by any modifications undertaken, any connection of equipment not approved by Visiomed*, or causes contrary to those written in the user manual or notice. Moreover, the present guarantee does not cover damage due to falls, bad handling, bad installations, damage by fire, floods, lightning, or any other natural disaster. This guarantee does not cover the packing of the material, the accessories, the defects caused by commercial exposure of the product, show room, sale space, demonstration etc... Normal maintenance, cleaning and the replacement of parts where wear is normal, are not covered by the terms of this guarantee. Visiomed* and its representatives and agents will not in any case be held responsible for any damage and consecutive damages due to the mishandling of this product. This guarantee is the only valid one at Visiomed*, any other guarantee (commercial guarantee) except this one will not be taken into account. IMPORTANT: During the guarantee period if you are dissatisfied with the repairs of this product, please contact the Visiomed* customer service.

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EN: Disposal of Waste Equipment by Users in Private Households in the European Union. The symbol on the product or on its packaging indicates that this product must not be disposed of with your other household waste. Instead, it is your responsibility to dispose of your waste equipment by handing it over to a designated collection point for the recycling of waste electrical and electronic equipment. The separate collection and recycling of your waste equipment at the time of disposal will help to conserve natural resources and ensure that it is recycled in a manner that protects human health and the environment. For more information about where you can drop off your waste equipment for recycling, please contact your local city office or your household waste disposal service.

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EN: TensioFlash® is a brand registered by Visiomed® Group SA. The constructor keeps the right to modify the photographs and characteristics without previous notice. Registered design. The logos and trademarks are the property of their respective owners.