Distribué par / Distributed by  
[www.visiomed-lab.com](http://www.visiomed-lab.com)

Marché Européen /  
European Market  
Visiomed Group SA  
Service clients /  
Customers service  
8, avenue Kléber  
75116 Paris - France  
Tel : +33 8 92 350 334  
Fax : +33 1 44 17 93 10  
[contact@visiomed-lab.fr](mailto:contact@visiomed-lab.fr)

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KD-795

By Visiomed®

Portable wrist blood pressure monitor model:  
TENSIOFLASH® KD-795

**PLEASE CAREFULLY READ THIS MANUAL BEFORE  
OPERATING THIS UNIT**

**TABLE OF CONTENTS**

<b>INTRODUCTION .....</b>	<b>27</b>
<b>NOTES ON SAFETY .....</b>	<b>28</b>
<b>BLOOD PRESSURE.....</b>	<b>30</b>
<b>PRECAUTIONS BEFORE USE .....</b>	<b>33</b>
<b>FEATURES OF THE TENSIOFLASH® KD-795 .....</b>	<b>34</b>
<b>MAIN UNIT .....</b>	<b>35</b>
<b>INSERT OR REPLACE BATTERIES .....</b>	<b>36</b>
<b>TIME AND DATE OF SYSTEM SETUP .....</b>	<b>37</b>
<b>VOICE SETTING .....</b>	<b>38</b>
<b>A FEW SUGGESTIONS BEFORE MEASUREMENT .....</b>	<b>39</b>
<b>HOW TO USE THE WRIST CUFF .....</b>	<b>40</b>
<b>HOW TO MEASURE BLOOD PRESSURE .....</b>	<b>41</b>
<b>RECALLING RECORDS .....</b>	<b>42</b>
<b>HOW TO MAINTAIN AND STORE THE UNIT .....</b>	<b>43</b>
<b>HOW TO PREVENT A MALFUNCTION.....</b>	<b>44</b>
<b>SPECIFICATIONS .....</b>	<b>44</b>
<b>TROUBLESHOOTING .....</b>	<b>46</b>
<b>EXPLANATION OF SYMBOLS ON UNIT .....</b>	<b>02</b>
<b>EMC DATA .....</b>	<b>225</b>
<b>WARRANTY .....</b>	<b>231</b>

**The manufacturer keeps the right to modify without  
any preliminary opinion technical specifications of the  
product.**

**INTRODUCTION**

Thank you for purchasing this **Visiomed** wrist Automatic Blood Pressure Monitor. This remarkable instrument is ideal for people who frequently monitor their own blood pressure because it is easy to use. Simple push of a button, and the monitor measures your blood pressure and pulse and displays the reading on a clear digital panel. Perfect for quick, easy readings at home, at work, almost anywhere! The Monitor uses the oscillometric method of blood pressure measurement. This means the monitor detects your blood's movement through the artery in your wrist and converts the movements into a digital reading. An oscillometric monitor does not need a stethoscope so the monitor is simple to use and measurements are easy to obtain. Please read this instruction manual thoroughly before using the wrist Automatic Blood Pressure Monitor. For specific information on your own blood pressure, **CONTACT YOUR PHYSICIAN.**

The product complies with the electromagnetic compatibility requirement of EN60601-1-2 and safety standards of 60601-1 as specified in EEC directive 93/42/EEC.

Material complies with the European norm concerning non invasive blood pressure monitors, part 1 General Requirements EN1060-1; part 3 Additional Requirements for the electromechanical measuring of blood pressure EN1060-3 and part 4 Test procedures to determine the overall system accuracy EN1060-4.

## NOTES ON SAFETY

- Use this appliance only for the use for which it is designed as described in this manual.
- This appliance may be used for personal use at home. Do not use the unit for other purpose.
- Use this appliance in an ambient temperature range of 10 to 40°C.
- Do not expose this appliance to extreme temperature conditions > 60°C ou < -20°C.
- Do not use this appliance at a relative humidity of more than 85%.
- This appliance must always be placed in a clean and dry place.
- Do not expose this appliance to sunlight or water.
- Do not expose this appliance to electric shocks.
- Do not use this appliance outdoors.
- Never drop the appliance.
- Follow the maintenance instructions specified in this manual.
- Do not attempt to open the device. In case of problems, contact your dealer.
- Do not leave this appliance within the reach of children.
- The measurement results are given for information. If in doubt about your results, please contact your doctor.
- This product should not encourage self-medication or adaptation of your treatment.
- Discontinue use of the device in case of anomalies or malfunction.
- This electrical medical equipment requires specific precautions regarding electromagnetic compatibility. It must be installed and used according to the electromagnetic information.

- Generally, portable and mobile RF communications equipment don't affect medical electrical equipment, but it can't be used in the strong electromagnetic interference environment: TV, microwave oven...
- This appliance is not designed for use by people (including children) with reduced physical, emotional or mental abilities or those lacking experience or knowledge unless used in the presence of someone responsible for their security, under surveillance or following instructions about how to use the appliance.

## BLOOD PRESSURE

Blood pressure is the force exerted on the walls of your blood vessels as blood flows through them.


Your heart is like a pump. When it contracts, or beats, it sends a surge of blood through the blood vessels and pressure increases. This is called your systolic pressure.

When your heart relaxes between beats, your blood pressure decreases. This is called your diastolic pressure.

When a doctor takes your blood pressure, he or she measures both your systolic and diastolic pressures and records them as numbers. For example, if your blood pressure reading is 126/76 (126 over 76), your systolic pressure is 126 and your diastolic pressure is 76. The numbers are calculated in millimetres of mercury and recorded as 126/76 mm Hg.

These two numbers provide important information about your health. The more difficult it is for your blood to flow through your blood vessels, the higher both numbers will be. When blood pressure is consistently above normal it is called hypertension (High blood pressure).

The **TensioFlash**<sup>®</sup> automatically measures your systolic and diastolic blood pressure for you, then displays the results on easy-to-read digital panel.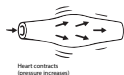


Systolic (upper number)

126 / 76

People with a condition that causes circulatory problems (diabetes, kidney disease, arteriosclerosis, or poor peripheral circulation) may get lower readings with upper arm type monitor than with a blood pressure monitor that is used on the upper arm.

Please consult your doctor to determine if your upper arm blood pressure accurately reflects your actual blood pressure. Your blood pressure changes constantly. Blood pressure fluctuates from day to day and minute to minute according to your body's needs. For example, when you are exercising or angry your blood pressure increases, but when you are relaxing or sleeping your blood pressure decreases. These fluctuations are completely normal. This portable upper arm blood pressure monitor is especially helpful when tracking changes in your blood pressure because it travels anywhere and takes just seconds to use.

### Fluctuation within a Day (Example: 35-year old male)

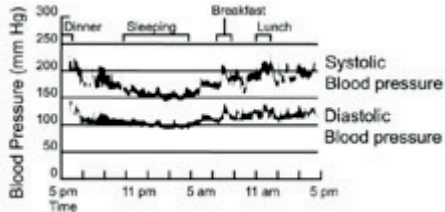Factors that may cause the blood pressure to fluctuate: Breathing, Exercise, Mental stress, Worries, Environmental and/or temperature change, Eating, Talking, Taking a bath, Drinking alcohol, Smoking, etc...

*Note: One or two readings will not provide a true indication of your normal blood pressure. It is very important to take regular, daily measurements and to keep accurate records. In partnership with your doctor, an accurate record of your blood pressure over a period of time can be a valuable aid in diagnosing and preventing potential health problems.*

The World Health Organisation (WHO) developed the following Blood Pressure Classification. This classification, however, is only a general guideline because blood pressure varies from person to person according to age, weight, and health status.

### **CONSULT YOUR DOCTOR TO DETERMINE YOUR NORMAL BLOOD PRESSURE**

Categories for Blood Pressure Levels in Adults  
(In mmHg, millimeters of mercury)

Category	Systolic (Top number)	Diastolic (Bottom number)	Color indication WHO
Optimal	Less than 120	Less than 80	Green
Normal	120-129	80-84	Green
Prehypertension	130-139	85-89	Green

Hypertension stage 1	140-159	90-99	Yellow
Hypertension stage 2	160-179	100-109	Orange
Hypertension stage 3	Higher than 180	Higher than 110	Red

*For adults 18 and older who: Are not on medicine for high blood pressure / Are not having short-term serious illnesses  
Do not have other conditions such as diabetes and kidney disease  
Note: There is not an universally accepted definition of hypotension (low blood pressure), but a systolic pressure below 99 mmHg is usually regarded as hypotension.*

### **PRECAUTIONS BEFORE USE**

- Do not confuse self-monitoring with self-diagnoses. Blood pressure measurements should only be interpreted by a health professional who is familiar with your medical history.
- If you are taking medication, consult with your physician to determine the most appropriate time to measure your blood pressure. NEVER change a prescribed medication without first consulting with your physician.
- This device is designed for adults to use if this device is used on a child patient, old people or toddlers, consult with your physician.
- Measurements may be impaired if this device is use near televisions, microwave ovens, X-ray mobile phone equipment or other devices with strong electrical fields. To prevent such interference, use the meter at a sufficient

distance from such devices or turn them off.

- If there is overmuch pressure or you feel uncomfortable, please press *START/STOP* button immediately for quick deflation.
- Do not use this unit in a moving vehicle, This may result in erroneous measurement;
- Blood pressure measurements determined by this monitor are equivalent to those obtained by a trained observer using the cuff/stethoscope auscultation method, within the limits prescribed by the American National Standard Institute, Electronic or automated sphygmomanometers.
- Please do not use the cuff other than supplied by the manufacturer, otherwise it may bring biocompatible hazard and might result in measurement error.
- Measurements may be impaired if this device is use near televisions, microwave ovens, X-ray mobile phone equipment or other devices with strong electrical fields. To prevent such interference, use the meter at a sufficient distance from such devices or turn them off.



### FEATURES OF THE TENSIOLASH® KD-795

1. Measurement indication of the SYS, DIA, Pulse/Min. and indication of the blood pressure level classification
2. Detection irregular heartbeat IHB function
3. High accuracy
4. Completely automatic inflation and deflation to provide home blood pressure monitoring convenience.
5. Saves up to 60 memories with date/time display
6. Easy to wrap wrist cuff suitable for wrist sizes 5 ¼" to 7 ¾"

(14 to 19.5 cm)

7. Talking measurement in 6 languages: French, English, Spanish, German, Arabic, Dutch.
8. Portable and travels almost anywhere operating on 2 AAA alkaline batteries.
9. Easy to use, press a button to automatically measure, recording the measurement values and measurement time.
10. Automatically power off (within 2 minutes) to save power.

### MAIN UNIT

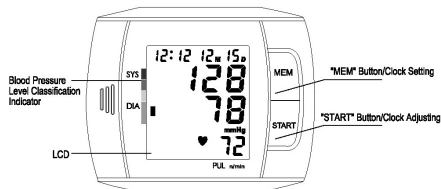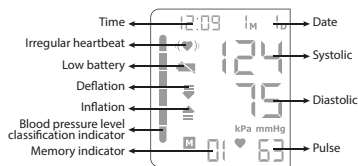## INSERT OR REPLACE BATTERIES

When LCD shows battery symbol, replace all batteries with new ones.

1. Slide the battery cover in the direction of the arrow to open.
2. Insert 2 alkaline AAA batteries in the correct polarity direction. Inserting them incorrectly may damage the device and the warranty. Never use rechargeable batteries. Use single use batteries.
3. Replace the battery cover.

Disposal of empty battery to the authorized collecting party subject to the regulation of each individual territory.


If the unit will not be used for a long period, please take out all batteries to avoid liquid leakage damaging the unit.

With the use of 2 alkaline "AAA" batteries, you can measure more than 50 times at room temperature of 72°F (22°C) and inflating to around 170mmHg once a day. But the included batteries only for test purpose may not last as specified long period.

## TIME AND DATE OF SYSTEM SETUP



1. When the machine is turned off, press the button "START" and "MEMORY" at same time, the number of month will glitter on LCD. Press button "START" continuously, the numbers of month, date, hour, and minute will glitter. While the number is glittering, press button "MEMORY" the corresponding number will increase by 1. Hold on pressuring the button "MEMORY" the number will increase fast.
2. After pressuring "START" button the fifth time, numbers on LCD will not glitter, then you exit the time adjusting mode. See picture.

*Note: Please check and re-set Date/Time after replacing of new batteries.*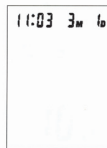

## VOICE SETTING

1. Voice language setting: In clock display mode, you can select the voice language or without talking function by keeping on press the button "M". You can select the wanted language by releasing button "M" when display the corresponding language. Here are 6 languages: French, English, Spanish, German, Arabic, Dutch
2. Voice volume setting: You can select the voice volume by pressing the button "M" again and keeping after language setting. Now the bars on the LCD change and the number of the bars indicates the volume. You can select the wanted volume by releasing button "M" when display the corresponding bars
2. Voice volume setting: You can select the voice volume by pressing the button "M" again and keeping after language setting. Now the bars on the LCD change and the number of the LCD change a

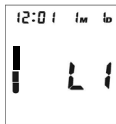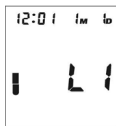## A FEW SUGGESTIONS BEFORE MEASUREMENT

1. Avoid eating, smoking, and exercising for at least 30 minutes before taking a measurement. Also rest for at least 15 minutes before taking a reading.
2. Stress raises blood pressure. Avoid taking measurements during stressful times.
3. You would better apply the cuff on your left upper arm.
4. Measurement should be taken in a quiet place and you should be relaxed, in a seated position. Rest your left arm on a table.
5. Remain still and do not talk during the measurement.
6. Wait 5-10 minutes between successive measurements. Waiting allows the engorged blood vessels to return to normal. You may require more rest time depending on your individual physical conditions.
7. Please refer to the cuff circumference range in "SPECIFICATIONS" to make sure that the appropriate cuff is used.

## HOW TO USE THE WRIST CUFF

1. Put off your watch and any jewel from your left hand.
2. Hold your left wrist with your palm facing up; apply the cuff so that the display of monitor is in easy to view position.
3. The cuff's edge should be approximately 1/4 to 1/2 inch (1~2 cm) below your palm.
4. Wrap the cuff snugly around your wrist, the top and bottom edges of the cuff should be tightened evenly around your wrist and lower arm.
5. When the cuff is positioned correctly, secure it in place by pressing the pile material firmly against the sewn hook material.
6. Sit in a chair with your feet flat on the floor and place your left arm on a table with your palm facing upward. The cuff should be at the same level as your heart.

*Note: Your heart is located slight below your armpit. If the cuff is not at the same level as your heart or if you cannot keep your arm completely still throughout the measuring, use the store box or other soft object to support your arm.*

7. Remain seated quietly for two minutes before you begin the measurement.

**Notes :**

- If it is not possible to fit the cuff to the left arm, it can also be placed on the right arm
- If the cuff is dirty, clean it with a damp cloth.

## HOW TO MEASURE BLOOD PRESSURE

1. After your body is in a comfortable position, press the "START" button. After a long "beep", BPM will verify itself, and all LCD will glimmer automatically.
2. If it already has memory, then LCD will show the last result. If machine does not have memory before, the LCD will show of four "0", while it verify itself **schema**.
3. If the voice function is on, the monitor will speak out the measurement tips with that language. After two seconds (maybe longer because of verifying itself), LCD shows one "0". Then the machine begins to pressurize automatically.
4. After measurement, the result of systolic, diastolic and pulse rate will all display on LCD. And irregular heartbeat symbol and blood pressure classification indicator will glimmer. The result will be automatically memorized by itself.
5. Pressing "START" button after measuring, machine will be turned off. It will be automatically power off after 1 minute of none use

**Notes:**

- The value measured will be too low if you hold your cuff higher than heart level, or will be too high if you hold your cuff lower than heart level.
- The unit may store 60 measurement values, If more than 60 records, it will delete the earliest data.
- During measurement do not speak or shake as it may affect the accuracy of the result.
- If you want to stop the measurement during measurement process, just press the START button to stop immediately.
- Please consult a health care professional for interpreting pressure measurements.

## RECALLING RECORDS

1. Press the **MEM** button to display the stored results.  
Now the LCD displays the amount of results. If no result stored, LCD will show dashes « - ».
2. The LCD will display the last measurement. Press the **MEM** button to review the data in ascending order. The time and date are stored for all results.
3. Deleting measurements from the memory: press the **MEM** key until 0 is displayed on the screen.
4. Press the **START** button to stop the machine.

## HOW TO MAINTAIN AND STORE THE UNIT

1. Use a dry soft cloth to clean the unit. If necessary, use a cloth lightly dampened with tap water.
2. Do not use alcohol, benzene, thinner or other harsh chemicals to clean the device or cuff.
3. Before using please wash your hand. Do not wash or wet the cuff.
4. Store the unit in places far from direct sunlight, high humidity or dust.
5. Fold the pressure cuff and put the unit into the store box.

*Note: When the unit is not used for a long period, be sure to remove the batteries. Otherwise, the batteries may leak and damage the unit.*

## HOW TO PREVENT A MALFUNCTION

- Do not drop the device and avoid sudden jars or shocks.
- Do not insert other objects into any holes.
- Do not attempt to disassemble the unit.
- Do not put the appliance in water
- Do not crush the pressure cuff.
- The monitor might not meet its performance specifications or cause safety hazard if stored or used outside the specified temperature and humidity ranges in specifications.
- If the unit has been stored at temperatures below 32°F (0°C), leave it in a warm place for 30 minutes before using it, otherwise, may get wrong measurement result.

## SPECIFICATIONS

Product name	Automatic talking wrist blood pressure monitor
Model	KD-795
Measuring Method	Oscillometric System
Classification	Class IIa
Indication	Digital LCD Display
Measuring range	Cuff pressure: 0~300 mmHg. Pulse: 40~180 Beat/min.
Accuracy	Pressure: +/- 3 mmHg Pulse +/- 5%
Inflation	Automatic Inflation (Air pump)

Deflation	Automatic air release control valve
Memory	60 Memories
Power source	1.5V AAA x 2 Alkaline Battery
Battery life	> 50 Times (Once a Day, Room Temperature 72°F (22°C))
Working Temp. & humidity	5~40°C (40~104°F). ≤85% RH
Storage Temp. & humidity	-20~55°F (-4~131°F). ≤95% RH
Outside dimensions	340mm(H) x 72mm(W) x 30mm(D) 1.33 in (H) x 0.281 in (W) x 0.12in (D)
Weight	Approx. 0.30lb - 140g (Apparatus and Battery)
Accessories	Wrist cuff (fit size 5.3~7.7in (13,5~19,5 cm)), Instruction Manual, Box, 2 AAA battery
Operation Voltage	3V DC

**The manufacturer keeps the right to modify without any preliminary opinion technical specifications of the product.**

## TROUBLESHOOTING

If you have one of the following problems, please refer to this breakdown service guide to help resolve the problem. If the problem persists, please contact our customer service at +33 892 350 334.

PROBLEM	POSSIBLE CAUSE	SOLUTION
Nothing is displayed when you push the START/STOP button	No battery installation	Insert batteries
	Battery worn out	Replace new batteries
	The polarities of batteries placed wrongly	Insert battery in the correct polarities
Battery icon on	Battery low power	Replace battery, and measure again
Measuring error (LCD shows EE)	Appears when the accurate blood pressure could not be obtained accurately	Press START/STOP button again and re-measure. Check cuff if wrapped at the wrist of according to instruction. Check palm, if exerting effort Check if talking or moving during measurement. Check if posture is incorrect.

The systolic pressure value or diastolic pressure value too high	The hand with the cuff was held lower than your heart	Keeping correct position and gesture to measure again.
	The cuff was not attached properly	
The systolic pressure value or diastolic pressure value too low	You moved your body or spoke during measurement	
	The hand with the cuff was held higher than your heart	
	You moved your body or spoke during measurement.	

*Note: Relax for at least 10 minutes and then take another measurement.*

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# EMC DATA

## ELECTROMAGNETIC COMPATIBILITY INFORMATION

**Table 1**  
**For all ME EQUIPMENT and ME SYSTEMS**

Guidance and manufacture's declaration - electromagnetic emissions		
The KD-795 is intended for use in the electromagnetic environment specified below. The customer or the user of the KD-795 should assure that it is used in such an environment.		
Emissions test	Compliance	Electromagnetic environment - guidance
RF emissions CISPR 11	Group 1	The KD-795 uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	The KD-795 is suitable for use in all establishments other than domestic and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	Not applicable	
Voltage fluctuations/ flicker emissions IEC 61000-3-3	Not applicable	

**Table 2**  
**For all ME EQUIPMENT and ME SYSTEMS**

Guidance and manufacturer's declaration - electromagnetic immunity			
The KD-795 is intended for use in the electromagnetic environment specified below. The customer or the user of the KD-795 should assure that it is used in such an environment.			
IMMUNITY test	IEC 60601 test level	Compliance level	Electromagnetic environment - guidance
Electrostatic discharge (ESD) IEC 61000-4-2	$\pm 6$ kV contact $\pm 8$ kV air	$\pm 6$ kV contact $\pm 8$ kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30 %.
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	3 A/m	3 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.

**Table 3**  
**For all ME EQUIPMENT and ME SYSTEMS that are not LIFE-SUPPORTING**

Guidance and manufacturer's declaration - electromagnetic immunity			
The KD-795 is intended for use in the electromagnetic environment specified below. The customer or the user of the KD-795 should assure that it is used in such an environment.			
IMMUNITY test	IEC 60601 test level	Compliance level	Electromagnetic environment - guidance
RADIATED RF IEC 61000-4-3	3V/m 80 MHz to 2.5 GHz	3 V/m	Portable and mobile RF communications equipment should be used no closer to any part of the KD-795, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.

			<p>Recommended separation distance:</p> $d = 1.2 \sqrt{P}$ <p>80 MHz to 800 MHz</p> $d = 2.3 \sqrt{P}$ <p>800 MHz to 2,5 GHz</p> <p>Where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in meters (m).            Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey,<sup>a</sup> should be less than the compliance level in each frequency range.<sup>b</sup>            Interference may occur in the vicinity of equipment marked with the following symbol:</p> 
<p>NOTE 1 At 80 MHz and 800 MHz, the higher frequency range applies.</p> <p>NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.</p> <p><sup>a</sup> Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the KD-795 is used exceeds the applicable RF compliance level above, the KD-795 should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocating the KD-795.</p> <p><sup>b</sup> Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3V/m.</p>			

**Table 4**  
**For ME EQUIPMENT and ME SYSTEMS that are not LIFE-SUPPORTING**

<b>Recommended separation distances between portable and mobile RF communications equipment and the KD-795</b>			
The KD-795 is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the KD-795 can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the KD-795 as recommended below, according to the maximum output power of the communications equipment.			
<b>Rated maximum output of transmitter W</b>	<b>Separation distance according to frequency of transmitter m</b>		
	150 kHz to 80 MHz $d = 1.2 \sqrt{P}$	80 MHz to 800 MHz $d = 1.2 \sqrt{P}$	800 MHz to 2,5 GHz $d = 2.3 \sqrt{P}$
0.01	0.12	0.12	0.23
0.1	0.38	0.38	0.73
1	1.2	1.2	2.3
10	3.8	3.8	7.3
100	12	12	23

For transmitters rated at a maximum output power not listed above, the recommended separation distance  $d$  in meters (m) can be determined using the equation applicable to the frequency of the transmitter, where  $P$  is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

NOTE 1 At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.

NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

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**EN:** Visiomed\* will repair or replace this product free of charge in the case of defective parts or manufacturing defects, in accordance with the conditions mentioned below as follows:

**DURATION:** 24 MONTHS RETURN TO WORKSHOP

**LIMITS AND EXCLUSIONS:** This guarantee concerns only the original final purchaser. A purchase invoice, or another proof of purchase, with this guarantee card will be required to obtain an after-sales service, in accordance with this guarantee. This guarantee card will not be extended to another person only the original final purchaser. This guarantee becomes void if the serial numbers on the product are modified, replaced, illegible, absent, or if repair has been carried out by a service not approved, including the user.

This guarantee covers only the defects of the material or parts, occurring during normal use of the pro-duct. It does not cover the damage caused during the transport of the apparatus, causes due to repairs being carried out by the distributor, by any modifications undertaken, any connection of equipment not approved by Visiomed\*, or causes contrary to those written in the user manual or notice. Moreover, the present guarantee does not cover damage due to falls, bad handling, bad installations, damage by fire, floods, lightning, or any other natural disaster. This guarantee does not cover the packing of the material, the accessories, the defects caused by commercial exposure of the product, show room, sale space, demonstration etc... Normal mainte-

nance, cleaning and the replacement of parts where wear is normal, are not covered by the terms of this guarantee. Visiomed\* and its representatives and agents will not in any case be held responsible for any damage and consecutive damages due to the mishandling of this product. This guarantee is the only valid one at Visiomed\*, any other guarantee (commercial guarantee) except this one will not be taken into account.

**IMPORTANT:** During the guarantee period if you are dissatisfied with the repairs of this product, please contact the Visiomed\* customer service.

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**EN:** *Disposal of Waste Equipment by Users in Private Households in the European Union. The symbol on the product or on its packaging indicates that this product must not be disposed of with your other household waste. Instead, it is your responsibility to dispose of your waste equipment by handing it over to a designated collection point for the recycling of waste electrical and electronic equipment. The separate collection and recycling of your waste equipment at the time of disposal will help to conserve natural resources and ensure that it is recycled in*

*a manner that protects human health and the environment. For more information about where you can drop off your waste equipment for recycling, please contact your local city office or your household waste disposal service.*

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**EN:** *TensioFlash is a brand registered by Visiomed Group SA. The constructor keeps the right to modify the photographs and characteristics without previous notice. Registered design. The logos and trademarks are the property of their respective owners.*

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© 03/2016 KD-795