**bewell**<sup>®</sup>  
connect

**VISIOMED GROUP SA**

112, Avenue Kléber  
75116 PARIS - France  
contact@visiomed-lab.fr  
Tel : +33 (8) 92 350 366  
(0,40€/mn)

[www.visiomed-lab.com](http://www.visiomed-lab.com)  
[www.bewell-connect.fr](http://www.bewell-connect.fr)

Made in China

**Visiomed Technology Co., LTD.**

2<sup>nd</sup> Floor of N°.1 building, Jia An  
Technological Industrial Park  
67 District, Bao An  
518101 SHENZHEN - China

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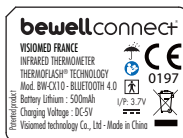
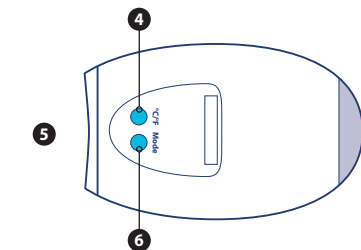
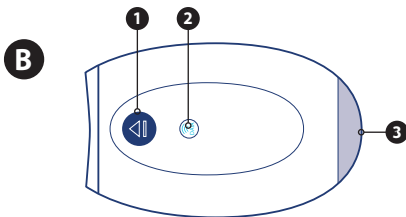
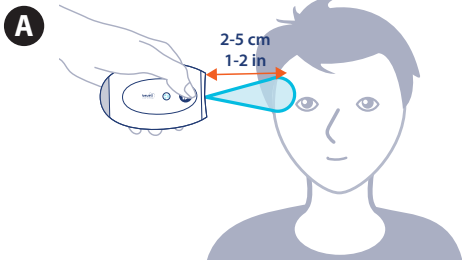
*The temperature of the human body varies throughout the day. It can also be influenced by numerous external factors: age, sex, type and thickness of skin...*

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The MyThermo BW-CX10 can take temperature readings below 30°C or above 43°C (90°F to 109.4°F) but precision is not guaranteed outside of this range.

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	< 35°C < 95°F	Hypothermie / Hypothermia / Hypothermie / Hipotermia / Unterkühlung / Ipotermia REÜLER, James B. Hypothermia: pathophysiology, clinical settings, and management. Annals of internal medicine, 1978, vol. 89, no 4, p. 519-527.
	35°C - 36,2°C 95°F - 97,2°F	Temp. à surveiller / Temp. to watch / Temp. moet in de gaten worden gehouden / Temp. en observación / Zu beobachten temp. / Temp. da tenere in osservazione
	36,3°C - 37,3°C 97,3°F - 99,1°F	Temp. normale / Normal temp. / Normale temp. / Temp. normal / Normalna temp Koushik Maharatna, Silvio Bonfiglio, Systems Design for Remote Healthcare, Springer Science & Business Media, 13 nov. 2013 - 332 pages BISSON, Jamie et YOUNKER, Jackie. Correcting arterial blood gases for temperature: (when) is it clinically significant?. Nursing in critical care, 2006, vol. 11, no 5, p. 232-238. Mackowiak PA, Wasserman SS, Levine MM (1992). A critical appraisal of 98.6°F, the upper limit of the normal body temperature, and other legacies of Carl Reinhold August Wunderlich. JAMA 268: 1578–1580.
	37,4°C - 37,9°C 99,3°F - 100,2°F	Temp. à surveiller / Temp. to watch / Temp. moet in de gaten worden gehouden / Temp. en observación / Zu beobachten temp. / Temp. da tenere in osservazione
	≥ 38°C ≥ 100,4°F	Fievre / Fever / Koorts / Fieber / Fieber / Febbre Wunderlich, C.A. and J.C. Reeve, The course of the temperature in diseases: a guide to clinical thermometry. American Journal of Medical Science, 1869, 57: p. 423-47 Various sources have slightly different temperatures for fever

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Serial number

SN:

Year                      Month                      Day                      Product serial number

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**bewell**  
connect

**EN**

**MicroSecond Flash™**  
TECHNOLOGY  
NEW GENERATION

**MyThermo**

**CE**  
0197

BW-CX10

By **Visiomed**

**COMPLIANT WITH EUROPEAN STANDARDS AND WITH PART 15 OF THE FCC RULES. BEWELLCONNECT BW-CX10 IS CLINICALLY TESTED AND APPROVED ISO 80601-2-56:2009.**

## TABLE OF CONTENTS

<b>1. SECURITY WARNINGS .....</b>	<b>21</b>
<b>2. PRECAUTIONS BEFORE USE.....</b>	<b>22</b>
<b>3. INTRODUCTION.....</b>	<b>22</b>
<b>4. OPERATING PRINCIPLES .....</b>	<b>22</b>
<b>5. FUNCTIONS .....</b>	<b>24</b>
<b>6. BATTERY CHARGING .....</b>	<b>24</b>
<b>7. DOWNLOAD THE APPLICATION.....</b>	<b>24</b>
<b>8. FIRST CONFIGURATION OF MYTHERMO BW-CX10.....</b>	<b>24</b>
<b>9. SETTINGS .....</b>	<b>25</b>
<b>9.1. CHOOSING THE TEMPERATURE MODE .....</b>	<b>25</b>
<b>9.2. CHOOSING THE TEMPERATURE UNIT .....</b>	<b>25</b>
<b>9.3. DATA MEMORIZATION.....</b>	<b>25</b>
<b>9.4 ADJUSTING THE ALARM - F1 .....</b>	<b>25</b>
<b>9.5. TOTAL DIFFERENCE - F2.....</b>	<b>26</b>
<b>9.6 ACTIVATING OR DEACTIVATING THE AUDIBLE ALARM F3 .....</b>	<b>26</b>
<b>9.7 EXIT THE SETUP MENU .....</b>	<b>26</b>
<b>10. TECHNICAL DETAILS AND SPECIFICATIONS .....</b>	<b>27</b>
<b>11. MYTHERMO BW-CX10 ACCURACY.....</b>	<b>28</b>
<b>12. ACCESSORIES INCLUDED.....</b>	<b>28</b>
<b>13. TROUBLESHOOTING.....</b>	<b>28</b>

Find out how to install and use the BewellConnect® app by visiting our website at [bewellconnect.fr/application-bewellconnect](http://bewellconnect.fr/application-bewellconnect)

*The manufacturer reserves the right to alter the specifications of the product without prior notification.*

Hereby, Visiomed declares that the radio equipment type MyThermo is in compliance with Directive 2014/53/EU.

The full text of the EU declaration of conformity is available at the following internet address:

<https://bewell-connect.com/en-gb/products-conformity/>

Dear Customer,

You have just purchased MyThermo by BewellConnect®. In order to use it optimally and efficiently, we recommend that you read these operating instructions carefully.

## 1. SECURITY WARNINGS

- Follow the maintenance instructions specified in this manual.
  - This device is suitable for home use.
  - Use this device only for its intended use as described in this notice.
  - Use this device in an ambient temperature range of between 15 and 40°C / 59°F / 104°F.
  - This appliance should always.
  - Do not expose this device to electric shock.
  - Do not expose this device to extreme temperatures T° > 50°C / 122°F - T° < - 20°C. /> 4°F.
  - Do not use this device at a relative humidity greater than 85%.
  - Do not expose this appliance to direct sunlight or water.
  - Do not use this device outside.
  - Never drop the device. A shock could damage it.
  - Do not attempt to service this device yourself. Do not attempt to repair or open this device. In the event of a problem, contact the after sales service department.
  - Keep out of the reach of children. This device is not intended for use by persons (including children) with impaired physical, sensory or mental capabilities, or persons with no experience or knowledge of the device, unless they have undergone supervision or prior instructions concerning the use of this device by a person responsible for their safety.
  - This electronic medical device requires specific precautions in relation to electromagnetic compatibility. It must be installed and commissioned in accordance with the electromagnetic information provided in the EMC Data table.
  - The measurements may be altered when used near a TV, microwave oven, mobile phone or other electric field device.
  - The measurement results are given for information only. If you are unsure about your results, please contact your doctor.
  - This device should not encourage self-medication or adjusting your treatment.
  - Stop using the device in the event of an abnormality or malfunction.
- In general, portable and mobile RF communication equipment do not affect the operation of this medical device. However, it cannot be used in an environment with high electromagnetic interference.

## 2. PRECAUTIONS BEFORE USE

THE MYTHERMO BW-CX10 IS PRE-CONFIGURED AT THE FACTORY. THE DEVICE DOES NOT HAVE TO BE CALIBRATED FOR ACTIVATION.

For reliable and stable results you are advised to leave the MyThermo BW-CX10 at ambient temperature for 15 to 20 minutes prior to use for each change in environment with a significant difference in ambient temperature. It is important to wait 1 minute between two readings.

- The protective glass of the lens is the most fragile part of the thermometer.
- Do not touch the glass of the infra-red lens with your fingers.
- Clean the glass with a cotton swab lightly dampened with 70° alcohol.
- Do not put the thermometer in contact with diseased, ulcerated or bruised skin.
- Do not drink hot or cold drinks or do rigorous exercise while your temperature is being taken.

## 3. INTRODUCTION

The MyThermo BW-CX10 is an infra-red thermometer specially designed to take the body temperature of children and adults without contact with the human body. The technology makes it possible to measure temperature on the temporal artery (TA) at a distance of approximately 2 to 5 cm / 1 to 2 in from the forehead/temple.

The MyThermo BW-CX10 thermometer is precise and instantaneous. It is perfectly adapted to taking temperature safely. It can be used at home or by healthcare professionals. Nevertheless, it is essential to use the MyThermo BW-CX10 appropriately in order to obtain a reliable and stable result. Therefore, you should read the directions and safety instructions carefully before use.

## 4. OPERATING PRINCIPLES

Each body, solid, liquid or gas, emits energy by radiation with an intensity that is related to its temperature. The MyThermo BW-CX10 infra-red thermometer measures a person's temperature by the energy they release. The temperature is taken with the external temperature sensor of the device which analyses and continuously records ambient temperature. In fact, as soon as the operator places the thermometer close to the body and turns on the radiation sensor, the temperature is immediately measured by detection of the infra-red heat emitted by arterial blood flow. The result of this body temperature measurement is then obtained without interference of the surrounding temperature.

### Benefits of temporal artery temperature (TA) measurement

The arterial infrared temperature can be measured by means of a device placed on the forehead in the region of the temporal artery.

The thermometer MyThermo BW-CX10 has been designed for the instantaneous measurement of the temporal/frontal temperature without physical contact with the temporal artery. This artery being relatively close to the surface of the skin and therefore accessible, the blood flow is continuous and regular and enables precise temperature measurement. This artery is connected to the heart by the carotid artery, itself connected directly to the aorta. It is part of

the central arterial system. The efficiency, speed and comfort of taking temperature in this area of the body make it ideal.

### Practical considerations in taking temperature

- For precision and measurement quality, it is essential for each user to be adequately informed and trained in the temperature-taking technique with a device such as this.
- It is essential to remember that temperature-taking procedures, although basic, should not be ignored.
- The patient should not carry out intense physical activity before taking their temperature.
- Temperature fluctuations due to physiological factors must be taken into account in the evaluation of the results:
  - The temperature increases by 0,5°C / 0.9°F between 6.00 and 15.00,
  - The average female temperature is higher by about 0,2°C / 0.36°F and varies, depending on their ovarian cycle,
  - It increases by 0,5°C / 0.9°F during the first half of the cycle and during the first months of pregnancy.
- The temperature of a seated person is about 0,3°C - 0,4°C / 0.54°F - 0.52°F lower than that of a standing person.

*"Haut Conseil de la santé publique" (Public Health Council) [online]. National heatwave plan 2014 health recommendations P123 [Accessed 25 July 2016] Available on [http://social-sante.gouv.fr/IMG/pdf/Recommandations\\_sanitaires\\_Plan\\_canicule\\_2014.Pdf](http://social-sante.gouv.fr/IMG/pdf/Recommandations_sanitaires_Plan_canicule_2014.Pdf). Temperature-taking modalities.*

### How to measure a temperature

See figure  page 2.

**Direct the device towards the FOREHEAD, towards the right temporal region, at a distance of approximately 2-5 cm / 1-2 in**, press the thermometer measurement button, and the temperature will be displayed instantly.

### Constraints

- Please observe the following procedure before any temperature measurement, to ensure a stable and reliable result:
- Brush hair away from the forehead.
  - If necessary, wipe sweat from the forehead.
  - Avoid draughts/any significant air flow (oxygen cannula, air conditioning...).
  - Respect a delay of 1 minute between two temperature-takes.

The reliability of the temperature measurement cannot be guaranteed on another anatomical region (e.g. arm, torso...).

## 5. FUNCTIONS

1. The result of the measurement is reliable and stable, thanks to the HEIMMANN infra-red sensor.
2. Audible alarm if the temperature is above the predefined limit (chapter 9-4).
3. Temperature diagnosis: colour indicator (green, yellow or red) to assist with interpreting the measurement obtained.
4. Backlit digital LCD screen.
6. Choice of data display in Celsius or Fahrenheit degrees.
5. Bluetooth connection to monitor the temperature curve via the BewellConnect application MyThermo.
7. Automatic Power-Off
8. Charge the device via USB

### Secondary applications:

The MyThermo **BW-CX10** can also be used to measure the temperature of a surface or the ambient temperature (via the TEMP or ROOM SURFACE mode).

## 6. BATTERY CHARGING

Before using MyThermo for the first time, please charge the battery via the USB cable. This function is in line with the European directive 89/336 / EEC - Electromagnetic Compatibility.

### CHARGE INDICATOR

When the MyThermo battery is low, the  symbol is displayed on the screen to indicate the low-battery status. You can continue to use the MyThermo which will make 5 continuous beeps and turn off automatically when the battery is completely flat. When the MyThermo battery is completely recharged, the power is automatically stopped. This prevents overcharging and extends the life of the battery. Recharge the batteries every 6 months if the device is not used for a long period.

## 7. DOWNLOAD THE APPLICATION

Download the BewellConnect® application to your smartphone or tablet:  
- on App Store or Google Play  
Then click on the icon MyThermo to activate the sub-application.

## 8. FIRST CONFIGURATION OF MYTHERMO BW-CX10

- Connect the supplied USB cable to your computer to charge the device. Once fully charged, you can unplug the cable.
- AT first use, wait about 10 minutes for the ambient temperature of the device.

- Direct the device toward the temporal lobe or one side / temple of the forehead, as shown in FIGURE **A** PAGE 2, at a distance of 2-5 cm / 1-2 in. Press the measurement button, the temperature is displayed instantly.
- You can then transfer your data to the BewellConnect application by pressing the transfer button, and monitor your temperature curve.

## 9. SETTINGS

### 9.1. CHOOSING THE TEMPERATURE MODE

Body mode : Body temperature measurement.

Measuring range : 30°C ~ 43°C (86.0°F ~109.4°F).

Surface Temp mode : Object, food's surface measurement.

Measuring range : 0°C to 93,3°C (32.0°F to 109.9°F).

ROOM mode : room temperature measurement.

Press the MODE button to select the temperature mode.

The screen displays : B for the Body mode – taking of the body temperature.

Press again the MODE button to go to the SURFACE TEMP mode. The screen displays S.

Press again the MODE button to go to the ROOM mode. The screen displays R.

NOTE : THE BW-CX10 MYTHERMO IS SET BY DEFAULT ON BODY.

Important : The surface temperature is different from the internal body temperature.

The body temperature can only be obtained by using the BODY mode.

Make sure to select the BODY mode for the internal temperature measurement and the SURFACE TEMP mode or the ROOM mode for the external temperature measurement.

### 9.2. CHOOSING THE TEMPERATURE UNIT

Press the °C/°F button to select the temperature unit. The screen displays the symbol of the unit chosen (°C for degrees Celsius and °F for degrees Fahrenheit).

### 9.3. DATA MEMORIZATION

Data transferred via Bluetooth are stored on the BewellConnect MyThermo application. You can monitor the history of your measurements on your smartphone or tablet

### 9.4 ADJUSTING THE ALARM - F1

Press the measurement button to turn on the device.

Press the °C/°F button until F1 is displayed on the screen.

The alarm limit is shown (38°C /100.4°F by default). The device will beep if the measurement is above this limit.

To change the limit, briefly press the MODE button to decrease it or the °C /° F button to increase it.

Then press the ° C /° F button a few seconds to display the next menu (F2).

 It is not advisable to change the alarm setting without the advice of your doctor.

### 9.5. TOTAL DIFFERENCE - F2

To adjust the temperature difference of **MyThermo** BW-CX10:

Press the °C / °F button for three seconds until F1 is displayed.

Press the button again for a few seconds to display the F2 menu.

Press the Mode / °C / °F button to change the total deviation from  $\pm 0,1^{\circ}\text{C} / 0,1^{\circ}\text{F}$ .

Confirm your new SETTING by pressing the °C / °F button for three seconds to display the F3 menu.

In case of doubt, it is advisable to keep the initial parameter at  $0,0^{\circ}\text{C} / 0,1^{\circ}\text{F}$ .

### 9.6 ACTIVATING OR DEACTIVATING THE AUDIBLE ALARM F3

Press the measurement button to turn on the device.

To display the F1 menu on the screen, press the °C/°F button for 3 seconds. Go to the F3 menu by pressing the °C / °F button twice for 3 seconds.

Press the °C/°F button to deactivate the audible alarm (0 is displayed on the screen), or on the MODE button to activate it (1 is displayed on the screen).

### 9.7 EXIT THE SETUP MENU

Press the °C / °F button for three seconds to confirm changes to the settings and exit the menu. The LCD screen displays **END** and powers off.

## 10. TECHNICAL DETAILS AND SPECIFICATIONS

Type of Product	Infrared Medical Thermometer
Model	MyThermo BW-CX10
Brand Name	BewellConnect
Classification	Class IIa
Temperature sensor	IR Sensor
Wavelength	5-14µm
Emissivity	0.95
Normal conditions of use	Ambient temperature Mode BODY/SURFACE TEMP: 15°C to 40°C / 59°F-104°F
Normal storage conditions	Temperature: -20°C to 50°C / -4°F to 122°F Relative humidity: 10% ~ 85%
Atmospheric pressure	80~105 kPa
Battery	DC3.7V (lithium battery) 
Dimensions	60,7 x 37,4 x 96,3 mm / 2.36 x 1.46 x 3.78 in
Weight	117 g / 0.26 lb
Measurement display	°C/°F
Measurement range:	BODY mode: 30° ~ 43°C / 86.0°F ~109.4°F SURFACE TEMP mode: 0° to 93,3°C / 32°F to 199.9°F ROOM Mode: 0° C to 50°C / 32° F to 122°F
Repeatability	$\leq \pm 0,3^{\circ}\text{C} / +/0,3^{\circ}\text{F}$
Measuring Distance	2 - 5 cm / 1-2 in
Display Resolution	0,1°C / 0.1°F
Accuracy	For Body mode : See table below In ROOM mode $\pm 1,0^{\circ}\text{C} / \pm 1,8^{\circ}\text{F}$ In SURFACE mode $\pm 0,4^{\circ}\text{C} / \pm 0,7^{\circ}\text{F}$
Bluetooth technology	4.0
Electricity consumption	< 50 mW

Voltage	3.6V ~ 4.2V
Power consumption in standby mode	≤15μA
Power consumption after power-up	≤150mA
Automatic shutdown	10 ~ 20 seconds

*DEVICE LONGEVITY: The MyThermo BW-CX10 is guaranteed for approximately 40,000 readings.*

## 11. MYTHERMO BW-CX10 ACCURACY

From 34°C to 35,9°C = / - 0,3°C From 93.2°F to 96.6°F = / - 0.5°F	ISO 80601-2-56: 2009
From 36°C to 39°C = / -0,2°C From 96.8° F to 102.2°F = / - 0.4°F	
From 39,1° C to 42,5° C = / -0,3°C From 102.4° F to 108.5° F = / - 0.5° F	

Accuracy outside the measuring range (0°C - 34°C / 32°F - 93.2°F) and ( 42,5°C - 93,3°C / 108.5°F - 199.9°F) cannot be guaranteed.

## 12. ACCESSORIES INCLUDED

- Instruction manual
- Charging cable
- Base

## 13. TROUBLESHOOTING

If you encounter the following problems in using your MyThermo BW-CX10, please refer to this troubleshooting guide. If the problem persists, contact customer service at 0892 350 334.

### The screen displays nothing:

Check that the battery is sufficiently charged.

### Temperatures over 95° are displayed:

The temperature is given in Fahrenheit. Change the unit of measurement to Celsius.

### The display shows a body temperature below 32°C / 89.6°F

To measure body temperature, the mode BODY must be selected. If the thermometer is in SURFACE TEMP or ROOM mode the displayed reading indicates the temperature displayed, <32°C / 89.6°F, corresponds to the external temperature of your body.

**The message HI is displayed:** The tested temperature is above the measurement range of the selected mode, which is above 43°C /109.4°F in BODY mode and above 93.3°C /199.9°F in SURFACE TEMP mode.

**The message LO is displayed:** The tested temperature is below the measurement range of the selected mode, which is below 32°C /89.6°F in BODY mode and below 0°C /32°F in SURFACE mode.

Causes of LO message display	Solutions
Measurement of temperature hampered by hair, sweat...	Ensure that the forehead is completely bare before taking any temperature.
Temperature measurement obstructed by draught /air flow.	Ensure that you are not in the axis of an air flow/draught as this could cause interferences with the infra-red system.
Temperature taken at intervals too close, the MyThermo did not have the time to reset.	Please respect the minimum 15 second delay between measurements - a 1 minute delay is recommended.
The measuring distance is too large.	Please observe the measuring distance. (Between 2-5 cm / 1-2 in).

## Bluetooth connection

If you have problems with the Bluetooth:

- Ensure that the MyThermo BW-CX10 is within reach of your smartphone/tablet with which you would like to synchronise it.
- Make sure that the Bluetooth is activated on your smartphone and that it is charged.
- Make sure you are in BODY mode on the MyThermo BW-CX10. No measurement taken in SURFACE or ROOM mode can be transferred to the application.

**bewell**  
connect

NL

**MicroSecond Flash™**  
TECHNOLOGY  
NEW GENERATION

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# EMC-DATA

## EMC data

**Guidance and manufacturer's declaration - electromagnetic immunity - for equipment and system that are not life-supporting.** The model BW-CX10 is intended for use in the electromagnetic environment specified below. The customer or the user of the model BW-CX10 should assure that it is used in such an environment.

Immunity test	EN 60601 test level	Compliance level	Electromagnetic environment - guidance
Conducted RF EN 61000-4-6	3 Vrms 150 kHz to 80 MHz	N/A	Portable and mobile RF communications equipment should be used no closer to any part of the Infrared Thermometer type BW-CX10, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter. Recommended separation distance
Radiated RF EN 61000-4-3	3 V/m 80 MHz to 2.5 GHz	3 V/m	$d = \left[ \frac{3.5}{V1} \right] \sqrt{P}$ 150 MHz to 80 MHz $d = \left[ \frac{3.5}{E1} \right] \sqrt{P}$ 80 MHz to 800 MHz $d = \left[ \frac{7}{E1} \right] \sqrt{P}$ 800 MHz to 2.5 GHz Where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in meters (m). Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey, should be less than the compliance level in each frequency range. b Interference may occur in the vicinity of equipment marked with the following symbol: 

NOTE 1 At 80 MHz and 800 MHz, the higher frequency range applies.

NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

a. Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the Infrared Thermometer type BW-CX10 is used exceeds the applicable RF compliance level above, the Infrared Thermometer type BW-CX10 should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as reorienting or relocating the Infrared Thermometer type BW-CX10.

b. Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3V/m.

**Recommended separation distances between portable and mobile RF communications equipment and the equipment or system - for equipment and systems that are not life-supporting.** The model BW-CX10 is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the model BW-CX10 can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the model BW-CX10 as recommended below, according to the maximum output power of the communications equipment.

Rated maximum output of transmitter w	Separation distance according to frequency of transmitter m		
	150 MHz to 80 MHz $d = \left[ \frac{3.5}{V1} \right] \sqrt{P}$	80 MHz to 800 MHz $d = \left[ \frac{3.5}{E1} \right] \sqrt{P}$	800 MHz to 2.5 GHz $d = \left[ \frac{7}{V1} \right] \sqrt{P}$
0.01	-	0.12	0.23
0.1	-	0.38	0.73
1	-	1.2	2.3
10	-	3.8	7.3
100	-	12	23

For transmitters rates at a maximum output power not listed above, the recommended separation distance d in meters (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

NOTE 1 At 80 MHz and 800MHz, the reparation distance for the higher frequency range applies.

NOTE 2 these guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

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**EN:** Visiomed\* will repair or replace this product free of charge in the case of defective parts or manufacturing defects, in accordance with the conditions mentioned below as follows:

**DURATION: 24 MONTHS RETURN TO WORKSHOP**

**LIMITS AND EXCLUSIONS:** This guarantee concerns only the original final purchaser. A purchase invoice, or another proof of purchase, with this guarantee card will be required to obtain an after-sales service, in accordance with this guarantee. This guarantee card will not be extended to another person only the original final purchaser. This guarantee becomes void if the serial numbers on the product are modified, replaced, illegible, absent, or if repair has been carried out by a service not approved, including the user.

This guarantee covers only the defects of the material or parts, occurring during normal use of the product. It does not cover the damage caused during the transport of the apparatus, caused due to repairs being carried out by the distributor, by any modifications undertaken, any connection of equipment not approved by Visiomed\*, or causes contrary to those written in the user manual or notice. Moreover, the present guarantee does not cover damage due to falls, bad handling, bad installations, damage by fire, floods, lightning, or any other natural disaster. This guarantee does not cover the packing of the material, the accessories, the defects caused by commercial exposure of the product, show room, sale space, demonstration etc... Normal maintenance, cleaning and the replacement of parts where wear is normal, are not covered by the terms of this guarantee. Visiomed\* and its representatives and agents will not in any case be held responsible for any damage and consecutive damages due to the mishandling of this product. This guarantee is the only valid one at Visiomed\*, any other guarantee (commercial guarantee) except this one will not be taken into account.

**IMPORTANT:** During the guarantee period if you are dissatisfied with the repairs of this product, please contact the Visiomed\* customer service.

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**EN:** Disposal of Waste Equipment by Users in Private Households in the European Union.

For more information about where you can drop off your waste equipment for recycling, please contact your local city office or your household waste disposal service.

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**EN:** BewellConnect® is a brand registered by Visiomed Group SA. The constructor keeps the right to modify the photographs and characteristics without previous notice. Registered design. The logos and trademarks are the property of their respective owners.

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© 08/2017 BW-CX10