bewell[®]
connect

MANUFACTURED FOR

European market /
Marché Européen

VISIOMED GROUP SA

112, Avenue Kléber
75116 PARIS - France
contact@visiomed-lab.fr
Tel : +33 (8) 92 350 366
(0,40€/mn)

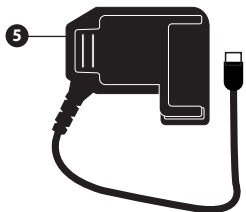
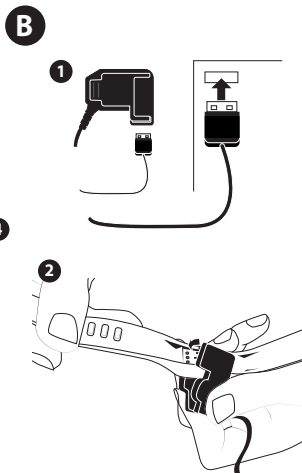
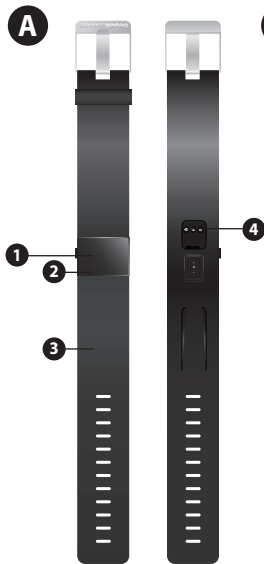
www.visiomed-lab.com
www.bewell-connect.fr

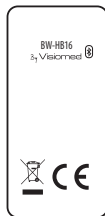
CE

BW-MyCoach[®] PULSE

BW-HB16 *By* Visiomed[®]

rating label

- 1 Bouton de navigation / Navigation button / Botón de navegación / Navigatieknop / Navigatieknop
- 2 Ecran LCD / LCD screen / Pantalla LCD / LCD-Bildschirm/ LCD-scherm
- 3 Bracelet silicone / Silicone wristband / Pulsera de silicona / Silikonarmband / Siliconen Armband
- 4 Point de contact (pince chargeur) / Contact point (Charging clip) / Punto de contacto (Pinza de carga) / Contact Punkt (Ladeklammer) / Contactpunt (Oplaadknijper)
- 5 Pince chargeur / Charging clip / Pinza de carga / Ladeklammer / Oplaadknijper

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CE

BW-HB16 By Visiomed[®]

COMPLIANT WITH EUROPEAN STANDARDS (ROHS).

CONTENTS

1. NOTICE	26
2. DESCRIPTION	27
3. INFORMATION.....	27
4. USE.....	28
4.1 PRECAUTIONS BEFORE USE	28
4.2. GETTING STARTED.....	28
4.3. STARTING UP	29
4.4. DATA MONITORING.....	29
4.4.1 DAILY DATA MONITORING.....	30
4.4.2. MONITORING THE HEART RATE.....	30
4.4.3. SLEEP MONITORING AND ANALYSIS	30
4.4.4. DAILY GOALS	31
4.4.5. MODE SPORT.....	31
4.4.6. FIND PHONE & ANTI-LOST FUNCTIONS	32
4.4.7. ALARM MODE.....	33
4.4.8. NOTIFICATIONS	33
4.4.9. INACTIVITY	34
4.4.10. WRIST SENSE.....	34
4.5. BATTERY LIFE - RECHARGING	34
4.6. SYNCHRONIZATION	35
5. CLEANING AND MAINTENANCE.....	35
6. TECHNICAL SPECIFICATIONS	35
7. TROUBLESHOOTING GUIDE	37

Learn how to install and use the BewellConnect® application by logging onto our website bewell-connect.fr/application-bewellconnect

The manufacturer reserves the right to modify the product specifications without prior notice.

Dear Customer,

You have just purchased a BW-MyCoach® Pulse, the connected activity, sleep and heart rate tracker by BewellConnect® and we thank you for your confidence. We hope you will enjoy using it and recommend that you carefully read this instruction leaflet.

1. NOTICE

Although your BW-MyCoach® Pulse activity tracker is designed to be worn at all times, improper use could damage its electronic components. Please read the safety instructions and warnings to ensure correct operation and maintenance.

- Use this device only for its intended purpose as described in this leaflet.
- This device can be used for personal use at home.
- Use this device in an ambient temperature range between -10°C and 45°C / 14°F and 113°F.
- Do not expose this device to conditions of extreme temperature > 60°C or < -20°C > 140°F or - 4°F.
- Do not use this device in relative humidity greater than 95%.
- This device must always be placed in a clean, dry place.
- Do not leave device in direct sunlight for extended periods of time or wear underwater.
- Do not expose this device to electric shock.
- Never drop the device.

- Follow the maintenance instructions described in this leaflet.
- Do not attempt to open the device. In the event of a problem, contact your dealer.
- Do not leave this device within reach of children.
- Stop using the device in the event of a fault or malfunction.
- This device is not intended to be used by persons (including children) with reduced physical, sensory or mental abilities, nor by persons lacking experience or knowledge, unless they have received supervision or prior instructions concerning the use of the device from a person responsible for their security.

2. DESCRIPTION

See A page 2

3. INFORMATION

Measurement of the heart rate

The heart rate is the number of heartbeats (contractions) per unit of time (usually per minute).

Patient	SLOW (BRADYCARDIA) IF BELOW	NORMAL (AT REST)	RAPID (TACHYCARDIA) IF ABOVE
Adult	60	60-80	100
Elderly		90	
Adolescent	50	60-105	105
Child (5-12 years)	60	60-120	120
Enfant (1-5 ans)	80	80-150	150

Infant	120	120-150	150
--------	-----	---------	-----

Reference: Mistovich, Joseph J., Brent Q. Hafen, and Keith J. Karren. *Prehospital emergency care*. Prentice-Hall, Inc., 2007

The pulse of a person in good health and at rest is between 50 (athlete with endurance training) and 80 heartbeats per minute. During an effort, the maximum heart rate is theoretically 220 minus the age of the person (e.g.: 190 at 30 years old)

Biological rhythms (such as digestion, waking/sleep, etc.) and physiological reactions (such as fear, stress, etc.) influence the heart rate, which is not constant.

4. USE

4.1 PRECAUTIONS BEFORE USE

BW-MyCoach® Pulse is resistant to rain, splashing and sweat. Nevertheless, our tracker is not designed to be immersed in water. Be sure to remove it before any water activity or before taking a shower. We also recommend you do not wear your activity tracker 24 hours a day, in order to let your skin breathe. Be sure to dry it when wet.

4.2. GETTING STARTED

How to wear the BW-MyCoach® Pulse ?

For optimal use wear your BW-MyCoach® Pulse on your wrist. You can also put it in your pocket or bag, but the indicators will be approximate.

The measurement of the heart rate is affected by various phenomena, such as the way the tracker is worn, body movements or the physiology of the wearer. This technology will not provide precise measurements for each activity, but we recommend you follow these instructions:

- Do not wear your activity tracker too tightly: this can decrease blood flow and alter the measurement. However, BW-MyCoach® Pulse must be adjusted to your wrist during physical activity.
- During physical activity requiring repeated movements of the wrists, we recommend you remain still for a few moments (approximately 10 seconds) in order to improve the measurement signal and to adjust your BW-MyCoach® Pulse on your wrist in relation to your movements to obtain a reliable measurement.

4.3. STARTING UP

Download the BewellConnect® application

Download the BewellConnect® application from the store of your smartphone or tablet (App Store for an iOS device, Play Store for an Android device).

First use – Programming personal data

Begin by creating your profile or that of the wearer.

Making the measurement

- Before the first use, charge your BW-MyCoach® Pulse activity tracker with the cable provided for approximately two hours.
- Make sure that the Bluetooth function (Bluetooth Low Energy 4.0 only) of your smartphone is turned on and that the version of the operating system of your smartphone is up to date (Android 4.3 or later Or iOS 7.1 or later)
- Then synchronize your activity tracker with the BewellConnect® application (more information in chapter 3.6. Synchronization).
- Once synchronized, your activity tracker is ready for use.

4.4. DATA MONITORING

You can follow your physical activity during the day and monitor

your sleep at night with your BW-MyCoach® Pulse.

4.4.1 DAILY DATA MONITORING

Your activity tracker allows you to automatically monitor your daily activity. Find the following information in the BewellConnect® application, as well as on the screen of your activity tracker:

	Your heart rate
	Calories burned
25.2 km	Distance traveled
	Number of steps taken

- To scroll through the different measurements on the screen, press the button located to the left of the screen. The time, date and battery level is displayed. Press the button again to change the menu (heart rate, number of steps taken, distance traveled, calories burned, time of the next alarm, "Find Phone" function).

These measurements are also displayed on the BewellConnect® application, as well as:

- The heart rate history
- The duration and quality of your sleep.

All your data are saved in the internal memory of your activity tracker for a period of 7 days.

4.4.2. MONITORING THE HEART RATE

4.4.3. SLEEP MONITORING AND ANALYSIS

BW-MyCoach® Pulse detects when you are asleep (drop in your heart rate, absence of movement) and automatically switches to

Night mode, in order to analyze the quality of your sleep and its evolution throughout the night (movements, awakenings and duration). These data are transferred to your smartphone via the BewellConnect® application to help you analyze the quality of your sleep.

4.4.4. DAILY GOALS

Set your daily goals and save them in the BewellConnect® application.

Set a target number of steps to perform daily in the application. You can monitor your goals at any time of the day on your BW-MyCoach® Pulse thanks to the progress bar located below the number of steps.

You can also set a goal for the duration of sleep per night, in order to improve your physical condition.

4.4.5. MODE SPORT

- You can monitor a specific sports activity thanks to the Sport mode. You can measure different metrics depending on the activity (a static activity such as fitness training will not measure the distance traveled).

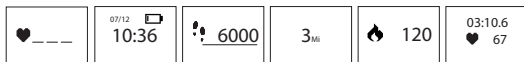
- To activate the Sport mode, press and hold the button of your BW-MyCoach® Pulse until it vibrates. A screen with the last reading and the last heart rate registered appears.

- To start the stopwatch, press the button of your tracker twice. Training begins when it vibrates.

You can navigate between screens at any time (as in normal mode) by pressing the button of your tracker. You can view:

- Elapsed time
- Heart rate and heart rate range
- Number of steps taken

- Distance traveled
- Calories burned
- Time

To stop the session, press and hold the button of your BW-MyCoach® Pulse until it vibrates. After synchronization, your data will appear in the BewellConnect® application.

4.4.6. FIND PHONE & ANTI-LOST FUNCTIONS

A. FIND PHONE

You can locate your smartphone thanks to your BW-MyCoach® Pulse. Press the button on the tracker until the “FIND PHONE” menu appears.

- Press and hold the button until the tracker starts to vibrate. Your smartphone will then ring or vibrate (depending on the mode activated).

Attention: This function will only work if your tracker is located within the area of Bluetooth coverage of your smartphone and if the BW-MyCoach® application is open on your smartphone (even in the background).

B. ANTI LOST

Your activity tracker is equipped with an “ANTI-LOST” function. As soon as your BW-MyCoach® Pulse tracker is separated from your smartphone by a distance of more than ten meters, it begins to vibrate to indicate to you that you are moving away from it.

4.4.7. ALARM MODE

- You can program up to 10 daily alarms to trigger reminders (medication, drink water, do sport, etc...).
- To add, modify, or delete an alarm, open the BewellConnect® application and then click on the BW-MyCoach® icon. In the drawer, click on "Alarms" and follow the instructions.

4.4.8. NOTIFICATIONS

- Over 120 compatible mobile devices allow you to receive notifications on your activity tracker.
- Go to <https://bewell-connect.com/compatible-devices> to see if your device is compatible.

Management of call notification

- To enable or disable notifications, go to the BewellConnect® application on your smartphone and then click on the BW-MyCoach® Pulse icon.
- In the Settings, click on "Notifications" and follow the indications.

Notifications

- When you receive a call, text or a message (social networks), it is displayed on the screen of your BW-MyCoach® Pulse.

4.4.9. INACTIVITY

- BW-MyCoach® Pulse helps you remain active throughout the day!

Schedule an inactivity alert on your BewellConnect® application. BW-MyCoach® Pulse starts to vibrate and an icon appears on the screen when it detects prolonged inactivity.

In the drawer, click on "Alarms" and follow the instructions.

4.4.10. WRIST SENSE

The screen of your BW-MyCoach® Pulse automatically turns on when it detects a movement of the wrist similar to the one that you make to look at a watch dial.

4.5. BATTERY LIFE - RECHARGING

- Your BW-MyCoach® Pulse activity tracker is equipped with a lithium-polymer rechargeable battery, which in normal conditions of use allows you to monitor your activity for approximately 5 - 10 days.
- Press the button to the left of the screen to display the battery level.
- The screen displays  in the event of a low battery.
- To recharge your BW-MyCoach® Pulse, connect the USB cable to your computer or to a power adapter and then connect the other end to the place provided at the rear of the activity tracker (See B page 2).
- Allow one to two hours to fully recharge your BW-MyCoach® Pulse.

4.6. SYNCHRONIZATION

- Your activity tracker automatically syncs when you open the BewellConnect® application and press the BW-MyCoach® icon.
- To sync manually, go to the menu of the application by clicking on the icon  and then press Synchronize.
- Your activity tracker is configured directly via the application. Remember to sync your tracker in the application for new settings to be taken into account.

5. CLEANING AND MAINTENANCE

Your BW-MyCoach® Pulse is designed for daily use. It is important to clean it regularly, including the inner part of the wristband that comes in contact with your skin.

- Use a soft, dry cloth to clean the device. If necessary, use a cloth slightly moistened with tap water.
- Do not use alcohol, benzene, thinner or other chemical products to clean the device.
- This product is not waterproof - avoid any contact with liquids. Be sure to dry the wristband before wearing it.

It is recommended to regularly clean the activity tracker using a soft, dry cloth to avoid the accumulation of microscopic dust and dirt that could affect the reliability of the measurements.

6. TECHNICAL SPECIFICATIONS

PRODUCT NAME	BewellConnect® BW-MyCoach® Pulse	
REFERENCE	BW-HB16	
SETTINGS	MIN.	MAX.

Steps	0	99999
Distance (km)	0	9999
Calories (kcal)	0	9999
TIME FORMAT	12/24 hours	
NIGHT MODE	Automatic	
ALARM	Up to 10 configurable alarms on the BewellConnect® application	
NOTIFICATIONS	Text messages Calls Facebook Facebook Messenger WhatsApp WeChat Twitter LinkedIn Instagram	
ALARMS	Inactivity Anti-Lost Telephone localization	
OTHER FEATURES	Wrist Sense	
SCREEN	OLED 0.49 in	
MEMORY	16kb RAM + 256kb ROM	

WRISTBAND	silicone / stainless steel
MEASURING RANGE	10 / 20 M
WRISTBAND DIMENSIONS	240 x 20 x 12 mm / 9,44 x 0,78 x 0,47 in
WEIGHT	26,7 g / 0,05 lb
BATTERY	55mAh Lithium Polymer
OPERATING VOLTAGE	3.7 V
BLUETOOTH	BLE 4.0
AUTONOMY	5 - 10 days depending on use
CHARGING TIME	Approximately 2 hours depending on the charging mode (mains or computer)
DATA STORAGE	7 days
OPERATING TEMPERATURE	From -10°C to 45°C (from 14°F to 113°F)
CHARGING CABLE	Charging clip

7. TROUBLESHOOTING GUIDE

Problem	SOLUTION
Nothing is displayed on the screen of the BW-MyCoach	Recharge the device with the USB cable provided.

No transfer of data to your phone / tablet	Bluetooth connection disabled: Enable Bluetooth on your phone / tablet. Phone or tablet incompatible: Check that your phone or tablet is equipped with Bluetooth 4.0.
The BewellConnect® application does not detect the BW-MyCoach® Pulse activity tracker when pairing.	Check that the Bluetooth function on your smartphone is enabled and that the version of the operating system of your smartphone is updated (Android 4.3 or later Or iOS 7.1 or later). Check that your activity tracker is located within range of the smartphone you want to synchronize with (50 cm maximum). If the problem persists, please contact customer service or your dealer.
You are unable to connect your BW-MyCoach® Pulse activity tracker to the BewellConnect® application.	Activate Bluetooth. Restart your smartphone. If the problem persists, please contact customer service or your dealer.

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EN: Visiomed® will repair or replace this product free of charge in the case of defective parts or manufacturing defects, in accordance with the conditions mentioned below as follows:

DURATION: 24 MONTHS RETURN TO WORKSHOP

LIMITS AND EXCLUSIONS: This guarantee concerns only the original final purchaser. A purchase invoice, or another proof of purchase, with this guarantee card will be required to obtain an after-sales service, in accordance with this guarantee. This guarantee card will not be extended to another person only the original final purchaser. This guarantee becomes void if the serial numbers on the product are modified, replaced, illegible, absent, or if repair has been carried out by a service not approved, including the user.

This guarantee covers only the defects of the material or parts, occurring during normal use of the product. It does not cover the damage caused during the transport of the apparatus, causes due to repairs being carried out by the distributor, by any modifications undertaken, any connection of equipment not approved by Visiomed®, or causes contrary to those written in the user manual or notice. Moreover, the present guarantee does not cover damage due to falls, bad handling, bad installations, damage by fire, floods, lightning, or any other natural disaster. This guarantee does not cover the packing of the material, the accessories, the defects caused by commercial exposure of the product, show room, sale space, demonstration etc... Normal maintenance, cleaning and the replacement of parts where wear is normal, are not covered by the terms of this guarantee. Visiomed® and its representatives and agents will not in any case be held responsible for any damage and consecutive damages due to the mishandling of this product. This guarantee is the only valid one at Visiomed®, any other guarantee (commercial guarantee) except this one will not be taken into account. **IMPORTANT:** During the guarantee period if you are dissatisfied with the repairs of this product, please contact the Visiomed® customer service.

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EN: *Disposal of Waste Equipment by Users in Private Households in the European Union.*

For more information about where you can drop off your waste equipment for recycling, please contact your local city office or your household waste disposal service.

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EN: *BewellConnect® is a brand registered by Visiomed Group SA. The constructor keeps the right to modify the photographs and characteristics without previous notice. Registered design. The logos and trademarks are the property of their respective owners.*

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