# bewell<sup>®</sup> connect

Distribué par / Distributed by  
[www.visiomed-lab.com](http://www.visiomed-lab.com)

Visiomed SAS France  
Service clients / Customers service  
8, avenue Kléber  
75116 Paris - France  
Tel : +33 8 92 350 366 (0,34€/mn)  
Fax : +33 1 44 17 93 10  
[contact@visiomed-lab.fr](mailto:contact@visiomed-lab.fr)

Made in P. R. C.

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BW-SC2

By Visiomed<sup>®</sup>

This product complies with the Reach (RoHS) standard.

## TABLE OF CONTENTS

<b>1. WARNINGS</b> .....	<b>19</b>
<b>2. INFORMATION</b> .....	<b>20</b>
<b>2-1. PRODUCT DESCRIPTION</b> .....	<b>20</b>
<b>2-2. ABOUT BODY FAT MASS</b> .....	<b>21</b>
<b>2-3. ABOUT BODY WATER WEIGHT</b> .....	<b>21</b>
<b>2-4. ABOUT MUSCLE MASS</b> .....	<b>22</b>
<b>2-5. ABOUT BONE MASS</b> .....	<b>22</b>
<b>2-6. ABOUT THE BMI - BODY MASS INDEX</b> .....	<b>22</b>
<b>2-7. ABOUT DAILY CALORIE NEEDS</b> .....	<b>23</b>
<b>3. USE</b> .....	<b>23</b>
<b>3-1. PRECAUTIONS BEFORE USE</b> .....	<b>23</b>
<b>3-2. USING THE WEIGHING SCALES</b> .....	<b>24</b>
<b>3-3. CHANGING THE BATTERIES</b> .....	<b>25</b>
<b>4. CLEANING AND MAINTENANCE</b> .....	<b>25</b>
<b>5. AVOIDING MALFUNCTIONS</b> .....	<b>26</b>
<b>6. TECHNICAL DETAILS</b> .....	<b>26</b>
<b>7. TROUBLESHOOTING GUIDE</b> .....	<b>28</b>

Learn how to install and use the BewellConnect® application by connecting to our site

[www.bewell-connect.com/bewellconnect-application](http://www.bewell-connect.com/bewellconnect-application)

*The manufacturer reserves the right to modify the technical characteristics of the product without prior notice.*

Dear customer,

You have just acquired this set of BewellConnect connected and smart impedance meter weighing scales and we thank you. We hope you get the best out of it and advise you to read this instruction manual carefully.

## 1. WARNINGS

Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation.

If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.
- No modification of this equipment is allowed.
- Information regarding potential electromagnetic or other interference between the MyScale Analyzer and other devices together with advice on ways to avoid or minimize such interference.
- This device requires no calibration.
- The user must check that the equipment functions safely and see that it is in proper working condition before being used.
- Conditions that can unacceptably affect the ME equipment :
  - Effects of lint, dust, light
  - List of known devices or other sources that can potentially cause interference problems
  - Effects caused by pets, pests or children

- Use this appliance only for the use for which it is designed as described in this manual.
- This appliance may be used for personal use at home. Any other use could void the warranty and cause injury.
- Use this appliance in an ambient temperature range of 0 to 40°C.
- Do not expose this appliance to extreme temperature conditions > 60°C or < 20°C.
- Do not use this appliance at a relative humidity of more than 85%.
- This appliance must always be placed in a clean and dry place.
- Do not expose this appliance to sunlight or water.
- Do not expose this appliance to electric shocks.
- Do not use this appliance outdoors.
- Never drop the appliance.
- Follow the maintenance instructions specified in this manual.
- Do not attempt to open the appliance. In case of problems, contact your dealer.
- Do not leave this appliance within the reach of children.
- The measurement results are given for information. If in doubt about your results, please contact your doctor.
- Discontinue use of the device in case of anomalies or malfunction.
- Do not use any battery other than the mentioned batteries. Do not attempt to recharge non-rechargeable batteries. Do not throw them into a fire.
- Remove the batteries if the appliance is not used for a long period.
- This appliance is not designed to be used by persons (including children) whose physical, sensory or mental capabilities are reduced, or persons without experience or knowledge, unless they have been able to benefit, by the intermediary of a person responsible for their safety, from surveillance or prior instructions concerning use of the appliance.

**Important**

*Observe the polarity of the batteries. Incorrect polarity can cause damage and compromise the warranty of your appliance.*

**2. INFORMATION**

The human body is composed almost exclusively of the following elements: bone, muscle, organs, fat and water.

**2-1. PRODUCT DESCRIPTION**

MyScale Analyzer BW-SC2 is indicated to measure body weight, estimate body fat, body water percentage, body muscle mass, bones mass and daily calorie intake (DCI) using BIA (bioelectrical impedance analysis). This product is for use by generally healthy adults who are not ill, feverish, have a chronic or acute disease, or a condition that affects the level of hydration such as pregnancy.

**2-2. ABOUT BODY FAT MASS**

Fat protects the organs, regulates the body temperature and contains vitamins and energy. It is not unhealthy, but excess body fat can cause health problems. The percentage of fat gives an indication on the physical condition of a person.

The ideal percentage of body fat - also called adiposity - of an individual varies according to the age and sex.

STANDARD PERCENTAGE OF BODY FAT (GIVEN AS A GUIDELINE ONLY)								
Age	Female				Male			
	Slim	Normal	High	Excessive	Slim	Normal	High	Excessive
15 – 39	14 – 21	22 – 27	28 – 32	33 – 40	10 – 18	19 – 24	25 – 30	31 – 40
40 – 55	15 – 22	23 – 28	29 – 33	34 – 40	11 – 19	20 – 25	26 – 31	32 – 40
56 – 70	16 – 23	24 – 29	30 – 34	35 – 40	12 – 20	21 – 26	27 – 32	33 – 40

*We advise you to consult a doctor if in doubt.*

**2-3. ABOUT BODY WATER WEIGHT**

Water is an essential component for the body. Its rate is one of the indicators of the health of an individual. Water plays a vital role for almost all the organs of the human body. It gives shape to the cells, helps to maintain the body temperature, transports nutrients to the whole body, hydrates the skin and improves the digestive process. It allows the elimination of waste from the body, protects the vital organs and lubricates the joints, etc.

The ideal percentage of body water mass of an individual varies according to the age and sex.

STANDARD PERCENTAGE OF BODY WATER (GIVEN AS A GUIDELINE ONLY)		
Age	Female	Male
10-15	57-67%	58-72%
16-30	47-57%	53-67%
31-60	42-52%	47-61%
61-80	37-47%	42-56%

*We advise you to consult a doctor if in doubt. These results may be altered if the person is dehydrated.*

## 2-4. ABOUT MUSCLE MASS

The muscles play an important role because they act like the engine of your body by consuming energy (calories). The muscle mass is calculated from the weight of the muscles of your body (skeletal muscle, smooth muscle and water contained in the muscles).

### STANDARD PERCENTAGE OF MUSCULAR MASS (GIVEN AS A GUIDELINE ONLY)

Female	Male
34%	40%

## 2-5. ABOUT BONE MASS

The bone mass represents the part of the body weight made up by the bones of the skeleton. It represents approximately 15% of the total weight. This percentage decreases progressively from 50 years of age. In case of accelerated bone loss, we advise you to consult your doctor.

### STANDARD BONE MASS IN KG (GIVEN AS A GUIDELINE ONLY)

Weight	Female
< 50	1,95
50 - 75	2,40
> 75	2,95
Weight	Male
< 65	2,66
65 - 95	3,29
> 95	3,69

## 2-6. ABOUT THE BMI - BODY MASS INDEX

THE BMI (or BMI in English) is an indicator of your corpulence and potential medical risks, based on the ratio of weight to height.

### BMI DIAGNOSTIC (GIVEN AS A GUIDELINE ONLY)

BMI	Diagnostic
< 16.5	Malnutrition
16.5 - 18.5	Slim
18.5 - 25	Normal
25 - 30	Overweight
30 - 35	Moderate obesity

35 - 40	Severe obesity
> 40	Morbid obesity

## 2-7. ABOUT DAILY CALORIE NEEDS

TAll food provides calories and all calories provide energy. The human body uses this energy to live, breathe and perform all daily activities. A man consumes more energy than a woman. Usually, his energy needs are from 5 to 10% higher than those of a woman - with the exception of a pregnant or lactating woman, whose daily calorie needs ensure the good health of the mother and the baby. Depending on your age, size and type, the BewellConnect® scales tell you the number of calories to absorb each day to reach a standard weight.

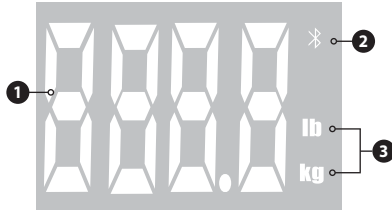
## 3. USE

### 3-1. PRECAUTIONS BEFORE USE

- This device is not suitable for children under 10 years for the calculation of body fat, water mass and bone mass. Do not leave children unsupervised when using the appliance.
- This device is not suitable for persons over 100 years for the calculation of body fat, water mass and bone mass.
- This device is not suitable for professional athletes or bodybuilders. The displayed results might be inaccurate, due to variations of physiological muscle density and level of hydration of athletes. If in doubt, consult your doctor.
- Place the scales on a sturdy, level and non-slip surface (avoid installing them on a rug, for example).
- Be barefooted (without shoes or socks), otherwise the result will not be displayed or will be false.
- The weight and fat mass vary during the day. We advise you to weigh yourself before meals to obtain comparable results. It is recommended that you should measure your weight and body fat mass at the same time each day, preferably on an empty stomach.
- The rates assessed by this device are only a reference and are not designed for a medical purpose. If your rates are higher or lower than normal, please contact your doctor for advice.

### 3-2. USING THE WEIGHING SCALES

- Weighing mode

- 1 Display                      2 Bluetooth                      3 Weight unit

**Overload warning :** Immediately remove anything on the scales if the screen displays Err. Irreversible damage could be caused to the appliance.

- Impedance meter mode (weighing and calculations: fat, muscle, water, bone and daily calorie needs) : available on the App.

#### Choice of the weighing unit

Go to the App MyScale to select Kilos or Pounds.

#### Weighing mode

Step onto the scales. It lights up. The screen displays 0:0, and then your weight. The digits flash, and then stabilize and remain on continuously.

#### Programming of personal data

Schedule your personal data, such as age, size or sex directly in the MyScale application.

#### Impedance meter mode (weighing and calculations)

##### DOWNLOADING THE APPLICATION

-  Download the BewellConnect® application to your phone or tablet:
  - by the App store or Google Play
  - or by flashing the QR Code on the side of the box.
-  Then click the MyScale icon.

##### DATA TRANSFER

To follow the evolution of your weight and your proportions, connect your device (phone, tablet) by Bluetooth (connectivity Bluetooth 4.0 smart Low Energy, compatible from iPhone 4S).

Open the app and select your profile. Open MyScale and go to the measure screen before weighing yourself, in order to save your datas.

Switch on the scales by stepping on them.

To obtain accurate measurements, it is important to position your feet on the plate sensors correctly and stay still during the weighing.

The data transfer is automatic with the BewellConnect® application. Simply assign the measurement made to a profile. You can consult your history on the application.

### 3-3. CHANGING THE BATTERIES

The battery compartment is located on the back of the appliance.

- When using for the first time, insert 4 new AAA batteries in the scales.

- When the LCD screen displays the icon , the batteries are low or flat. Replace the flat batteries, taking care to observe the correct polarity. Incorrect polarity can cause damage and compromise the warranty of your appliance.

Never insert rechargeable batteries. Use single-use batteries. Do not mix old and new batteries.

 Do not dispose of your old batteries with your household waste.

 Take your flat batteries to a collecting agency.

If the appliance is not to be used for a long time, be sure to remove the batteries to avoid any leakage of liquid that may damage the unit.

### 4. CLEANING AND MAINTENANCE

- Do not wet the weighing plate. This could make it slippery and cause an accident.

- Store your scales in the horizontal position.

- Put your scales away in dry place away from humidity.

- Use a soft and dry cloth to clean the unit. If necessary, use a soft lightly moistened (tap water) cloth.
- Do not use alcohol, benzene, thinner or other chemicals to clean the appliance.
- This product is not waterproof. Avoid any contact with liquid.
- If the unit has been exposed to a temperature below 0°C, be sure to leave it for 30 minutes at room temperature before use. Without this waiting, the results could be distorted.
- If the unit is not used for a long period of time, be sure to remove the batteries (they could damage the unit). Observe the polarity of the batteries. Do not mix old and new batteries. Do not use rechargeable batteries. Do not dispose of your old batteries with your household waste.

## 5. AVOIDING MALFUNCTIONS

- Do not drop the appliance. Avoid shaking and shocks.
- Do not insert anything into the appliance, except batteries.
- Do not dismantle the unit.
- Do not put the appliance in water.
- Observe the temperature and humidity ranges stipulated in the specifications.

## 6. TECHNICAL DETAILS

Product name	BewellConnect® MyScale Analyzer		
Reference	BW-SC2		
Identified Software version	V1		
Item	Unités	Minimum	Maximum
Weight measurement range	kg	5	180
BMI range	-	2.5	90
Body fat range	%	5	75
Water mass range	%	10	80
Muscle mass range	%	25	75
Bone mass range	kg	0.1	10
Calorie range	Kcal	385	3999
Accuracy of graduations			
Accuracy up to 50 kg	kg	0.5	
Accuracy up to 100 kg	kg	1	
Accuracy of 150 kg	kg	1.5	
Precision percentage			

Body fat mass	%	5
Water mass	%	5
Muscle mass	%	5
Bone mass accuracy	kg	0.2
Calories accuracy	Kcal	115
Weight units	kg / lb	
Profiles	8 user profiles Age: 10-80 years Size: 100-200 cm	
Dimensions	320 x 320 x 26mm	
Weight	2kg	
Normal conditions of use	Temperature : 5 - 40°C Humidity : 15 to 93% RH Pressure altitude : 700hPa to 1060hPa	
Storage conditions	Temperature : -25 - 70°C Humidity : 15 to 93% RH Pressure altitude : 700hPa to 1060hPa	
Bluetooth technology	4.0 smart Low Energy	
Power supply	4 AAA batteries (supplied)	
Battery life	3-4 months (for 4 daily measurements) Low battery indicator	
Automatic shut-off	±10 seconds	
Mode of operation	Continuous	
Internally powered equipment		
Supplied accessories	A connected and smart impedance meter set of scales, instructions, a quick installation guide, 4 AAA batteries.	

## 7. TROUBLESHOOTING GUIDE

If one of the following problems occurs when using your MyScale, refer to this troubleshooting guide. If the problem persists, please contact our customer support.

SCREEN ERROR	POSSIBLE CAUSES	SOLUTION
The icon  is displayed	Flat batteries	Insert new batteries
"O-LD" is displayed	Overload warning	<i>The maximum weight supported by the scales is 180 kg. Immediately remove the weight from the scales. Irreversible damage could be caused to the appliance</i>

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EMC DATA

## EMC DECLARATION

GUIDANCE AND MANUFACTURER'S DECLARATION – ELECTROMAGNETIC IMMUNITY			
THE BW-SC2 IS INTENDED FOR USE IN THE ELECTROMAGNETIC ENVIRONMENT SPECIFIED BELOW. THE CUSTOMER OR THE USER OF THE BW-SC2 SHOULD ENSURE THAT IT IS USED IN SUCH AN ENVIRONMENT.			
Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment – guidance
Electrostatic discharge (ESD) IEC 61000-4-2	±6 kV contact ±8 kV air	±6 kV contact ±8 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30 %.
Electrical fast transient/burst IEC 61000-4-4	±2 kV for power supply lines ±1 kV for input/output lines	±2 kV for power supply lines ±1 kV for input/output lines	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	±1 kV differential mode ±2 kV common mode	±1 kV differential mode ±2 kV common mode	Mains power quality should be that of a typical commercial or hospital environment.
Voltage dips, short interruptions and voltage variations on power supply input lines  IEC 61000-4-11	<5 % Ur (>95 % dip in Ur) for 0,5 cycles  40 % Ur (60 % dip in Ur) for 5 cycles  70 % Ur (30 % dip in Ur) for 25 cycles  <5 % Ur (>95 % dip in Ur) for 5 sec	<5 % Ur (>95 % dip in Ur) for 0,5 cycles  40 % Ur (60 % dip in Ur) for 5 cycles  70 % Ur (30 % dip in Ur) for 25 cycles  <5 % Ur (>95 % dip in Ur) for 5 sec	Mains power quality should be that of a typical commercial or hospital environment. If the user of the BW-SC2 requires continued operation during power mains interruptions, it is recommended that the BW-SC2 be powered from an uninterruptible power supply or a battery.
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	3 A/m	3 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.
NOTE UT is the a.c. mains voltage prior to application of the test level.			

## EMC data

**Guidance and manufacturer's declaration - electromagnetic immunity - for equipment and system that are not life-supporting.** The model BW-SC2 is intended for use in the electromagnetic environment specified below. The customer or the user of the model BW-SC2 should assure that it is used in such an environment.

Immunity test	EN 60601 test level	Compliance level	Electromagnetic environment - guidance
Conducted RF EN 61000-4-6	3 Vrms 150 kHz to 80 MHz	N/A	Portable and mobile RF communications equipment should be used no closer to any part of the Infrared Thermometer type BW-SC2, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter. Recommended separation distance  $d = 1.2 \sqrt{P}$ $d = 1.2 \sqrt{P}$ 80MHz to 800MHz $d = 2.3 \sqrt{P}$ 800MHz to 2.5GHz
Radiated RF EN 61000-4-3	3 V/m 80 MHz to 2.5 GHz	3 V/m	Where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in meters (m). Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey, should be less than the compliance level in each frequency range. b Interference may occur in the vicinity of equipment marked with the following symbol: 

NOTE 1 At 80 MHz and 800 MHz, the higher frequency range applies.

NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

a. Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the BW-SC2 is used exceeds the applicable RF compliance level above, the Medical BW-SC2 should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as reorienting or relocating the BW-SC2.

b. Over the frequency range 150 kHz to 80 MHz, field strengths should be less than [V1] V/m.

**Guidance and manufacturer's declaration - electromagnetic emissions.** The BW-SC2 is intended for use in an electromagnetic environment specified below. The customer or the user of the BW-SC2 should ensure that it is used such an environment.

Emissions test	Compliance	Electromagnetic environment - guidance
RF emissions CISPR 11	Group 1	The BW-SC2 uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	The BW-SC2 is suitable for use in all establishments, including domestics and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	Class A	
Voltage fluctuations/ flicker emissions IEC 61000-3-3	Complies	

**Recommended separation distances between portable and mobile RF communications equipment and the Medical BW-SC2.** The BW-SC2 is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the Medical BW-SC2 can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the BW-SC2 as recommended below, according to the maximum output power of the communications equipment.

Rated maximum output of transmitter w	Separation distance according to frequency of transmitter m		
	150 MHz to 80 MHz $d = \frac{[3.5]}{\sqrt{1}}\sqrt{P}$	80 MHz to 800 MHz $d = \frac{[3.5]}{\sqrt{E1}}\sqrt{P}$	800 MHz to 2.5 GHz $d = \frac{[7]}{\sqrt{E1}}\sqrt{P}$
0.01	0.12	0.12	0.23
0.1	0.38	0.38	0.73
1	1.2	1.2	2.3
10	3.8	3.8	7.3
100	12	12	23

For transmitters rates at a maximum output power not listed above, the recommended separation distance  $d$  in meters (m) can be estimated using the equation applicable to the frequency of the transmitter, where  $P$  is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

**NOTE 1** At 80 MHz and 800MHz, the reparation distance for the higher frequency range applies.

**NOTE 2** these guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

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**EN:** Visiomed\* will repair or replace this product free of charge in the case of defective parts or manufacturing defects, in accordance with the conditions mentioned below as follows:

**DURATION:** 24 MONTHS RETURN TO WORKSHOP

**LIMITS AND EXCLUSIONS:** This guarantee concerns only the original final purchaser. A purchase invoice, or another proof of purchase, with this guarantee card will be required to obtain an after-sales service, in accordance with this guarantee. This guarantee card will not be extended to another person only the original final purchaser. This guarantee becomes void if the serial numbers on the product are modified, replaced, illegible, absent, or if repair has been carried out by a service not approved, including the user.

This guarantee covers only the defects of the material or parts, occurring during normal use of the product. It does not cover the damage caused during the transport of the apparatus, causes due to repairs being carried out by the distributor, by any modifications undertaken, any connection of equipment not approved by Visiomed\*, or causes contrary to those written in the user manual or notice. Moreover, the present guarantee does not cover damage due to falls, bad handling, bad installations, damage by fire, floods, lightning, or any other natural disaster. This guarantee does not cover the packing of the material, the accessories, the defects caused by commercial exposure of the product, show room, sale space, demonstration etc... Normal maintenance, cleaning and the replacement of parts where wear is normal, are not covered by the terms of this guarantee. Visiomed\* and its representatives and agents will not in any case be held responsible for any damage and consecutive damages due to the mishandling of this product. This guarantee is the only valid one at Visiomed\*, any other guarantee (commercial guarantee) except this one will not be taken into account.

**IMPORTANT:** During the guarantee period if you are dissatisfied with the repairs of this product, please contact the Visiomed\* customer service.

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**EN:** *Disposal of Waste Equipment by Users in Private Households in the European Union.*

*The symbol on the product or on its packaging indicates that this product must not be disposed of with your other household waste. Instead, it is your responsibility to dispose of your waste equipment by handing it over to a designated collection point for the recycling of waste electrical and electronic equipment. The separate collection and recycling of your waste equipment at the time of disposal will help to conserve natural resources and ensure that it is recycled in a manner that protects human health and the environment. For more information about where you can drop off your waste equipment for recycling, please contact your local city office or your household waste disposal service.*

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© 11/2015 BW-SC2
**EN:** *BewellConnect® is a brand registred by Visiomed Group SA. The constructor keeps the right to modify the photographs and characteristics without previous notice. Registred design. The logos and trademarks are the property of their respective owners.*