Distribué par / Distributed by
www.visiomed-lab.com

Marché Européen / European Market
Visiomed SAS France
Service clients / Customers service
8, avenue Kléber
75116 Paris - France
Tel : +33 8 92 350 334
Fax : +33 1 44 17 93 10
contact@visiomed-lab.fr

Made in P. R. C.
Nexmed Technology Co., LTD.
2nd Floor of N°.1 building, Jia An
Technological Industrial Park
67 District, Bao An
518101 Shenzhen - China

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PRO LX-261E

By Visiomed®

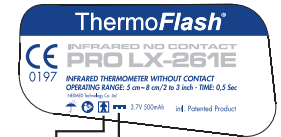
Infrared Thermometer is intended for intermittent measurement of human body temperature in people of all ages without contact.

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
A

B

IEC60417-5031
Direct current
IEC60417-5840
Type BF applied part

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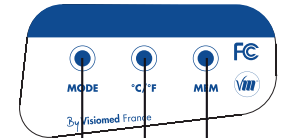The temperature of the human body varies throughout the day. It can also be influenced by numerous external factors: age, sex, type and thickness of skin...

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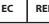
The THERMOFLASH® PRO LX-261E can take temperature readings below 32°C or above 42.9°C (90°F to 109°F) but precision is not guaranteed outside of this range.

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SN:

Year Month Day Product serial number

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USER MANUAL

CE
0197

MicroSecond Flash™
TECHNOLOGY
NEW GENERATION

THERMOFLASH® PRO LX-261 EVOLUTION is clinically tested and approved ASTM 1965-1998 (2009).

CONTENTS

I. SAFETY PRECAUTIONS	20
II. INTRODUCTION	22
III. PRECAUTIONS BEFORE USE	22
IV. OPERATING PRINCIPLES	23
The different methods of temperature measurement	23
Advantages of temporal artery (ta) temperature	24
Practical considerations when taking a temperature	25
V. DESCRIPTION OF THE THERMOFLASH® PRO LX-261 EVOLUTION	25
VI. FUNCTIONS	25
VII. CHARGING THE BATTERY	26
VIII. STARTING UP YOUR THERMOFLASH® PRO LX-261 EVOLUTION	26
IX. SETTINGS	27
X. TECHNICAL SPECIFICATIONS AND PRECISION	28
XI. CLEANING INSTRUCTIONS	29
XII. ACCESSORIES SUPPLIED	29
XIII. TROUBLESHOOTING	29

I. SAFETY PRECAUTIONS

Please respect the general safety precautions when using the **ThermoFlash®**. This device must only be used for the purpose described in this instruction manual.

- Follow the maintenance advice stipulated in this instruction manual.
- This device may be used for professional purposes or for personal home use.
- This device must only be used for the purposes described in this instruction manual.
- This device must only be used in an ambient temperature range of between 10 and 40°C.
- This device must always be kept in a clean, dry area.
- Do not expose this thermometer to electric shocks.
- Do not expose this thermometer to extreme temperature conditions of > 50°C or < - 20°C.
- Do not use this device in relative humidity higher than 85%.
- Do not expose the thermometer to sunlight or to water.
- Do not use this device outside.
- Never drop the device, Shock may damage this device.

- Should a problem occur with your device, please contact your retailer. Do not attempt to repair this device yourself.
- Please keep the thermometer off the children's touch. Prevent the product from children playing, swallowing and eating.
- The MEDICAL ELECTRICAL EQUIPMENT needs special precautions regarding EMC and needs to be installed and put into service according to the EMC information provided in the ACCOMPANYING DOCUMENTS.
- Generally, Portable and mobile RF communications equipment don't affect MEDICAL ELECTRICAL EQUIPMENT, but it can't be used in the strong electromagnetic interference environment.
- The measuring result of this product is only for your reference. If you have any doubt, please contact your doctor.
- This appliance must not encourage self-medication or changing your treatment.
- Discontinue use of the device in case of anomalies or malfunction.
- Measures may be altered when used near a TV, a microwave oven, a mobile phone or any other device with an electric field.

Important safeguards

For your safety, ONLY use the charger supplied.
Do not dismantle the device or the rechargeable base station.

The manufacturer reserves the right to alter the specifications of the product without prior notification.

Warning

- To avoid electrical discharge: do not use around water, for example, in a bathroom.
- Make sure your hands are dry when handling the device or the electrical supply cord.
- Do not submerge the AC adaptor or the device in a liquid.
- Unplug the device immediately if it has fallen into water before attempting to retrieve it.
- Do not use the device if any of its components are damaged (in particular the electrical supply lead or the plug), if it is submerged in water or if it has fallen.
- Return your device immediately to the after-sales service for checking and repair.
- The device must not be used where flammable gas, oxygen or aerosol products are stored or used.
- Always ensure that air can pass through the air vents. Do not place the device on a soft surface that may obstruct the apertures.
- In the case of malfunction, stop using the product and return it to the after-sales service for checking and repair.
- The device must not be left unsupervised while plugged in.

- Unplug the device from the mains before cleaning it.
- Only use the accessories recommended by the manufacturer.
- Do not use a sharp tool when handling the device.
- Do not change the AC adaptor, the battery and the charger. Use the adaptor, battery and charger supplied in order not to compromise the minimum safety requirements.
- Do not use the battery with any device other than the **THERMOFLASH® PRO LX-261 EVOLUTION**, or attempt to modify it for any other use.
- Avoid placing the battery on metal or wet surfaces, or inside a metallic container. This could short-circuit the contacts (positive and negative) of the battery and cause it to overheat or even to explode.
- Always remove the plugs from the device, the battery and the charger when the device is put away and not used for a long period of time.

II. INTRODUCTION

The **THERMOFLASH® PRO LX-261 EVOLUTION**, developed by **Visiomed®**, is a revolutionary infra-red medical thermometer without contact, fitted with cutting edge MicroSecondFlash™ automatic calibration technology using latest infrared technology.

This latter allows temporal artery (TA) temperature to be taken at a distance of about 5cm away from the right forehead (temple).

Precise, instantaneous and without contact, the **THERMOFLASH® PRO LX-261 EVOLUTION** is totally suitable for a precise temperature measurement without risk. It has been shown that this TA temperature measurement, for newborn, is more precise than tympanic thermometry and better tolerated than rectal thermometry (1). However, as with other types of thermometer, it is essential to use the **THERMOFLASH® PRO LX-261 EVOLUTION** properly in order to obtain reliable and stable results. You are therefore advised to read this instruction manual and the safety precautions carefully before use.

(1) Greenes D, Fleisher G. Accuracy of a Non-invasive Temporal Artery Thermometer for Use in Infants. Arch Pediatr Adolesc Med 2001; 155:376.

III. PRECAUTIONS BEFORE USE

The **THERMOFLASH® PRO LX-261 EVOLUTION** is pre-set at the factory. It is not necessary to calibrate the device when starting it up.

In order to obtain reliable and stable results, you are advised each time there is a significant change in the ambient temperature due to a change in environment, to allow the **THERMOFLASH® PRO LX-261 EVOLUTION** to acclimatize to this ambient temperature for 15 to 20 minutes before using it.

Please observe the following before any temperature measurement to ensure a

stable and reliable result :

- Push back hair from the right forehead (temple).
- Wipe away any perspiration from the right forehead (temple).
- Allow a 1 minute interval between two measurements.
- Avoid any drafts (e.g. from nasal specs, air conditioning...).

IV. OPERATING PRINCIPLES

All objects, solid, liquid or gas, emit energy by radiation. The intensity of this energy depends on the temperature of the object. The **THERMOFLASH® PRO LX-261 EVOLUTION** infrared thermometer is therefore able to measure the temperature of a person by the energy the person emits. This measurement can be taken thanks to an external temperature probe on the device which permanently analyses and registers the ambient temperature. Therefore, as soon as the operator holds the thermometer near the right forehead (temple) and activates the radiation sensor, the measurement is taken instantly by detection of the infrared heat generated by the arterial blood flow. Body heat can therefore be measured without any interference from the heat of the surrounding environment.

BODY TEMPERATURE

A important concept: the estimation of body temperature

The normal value of the body temperature is located on an interval, and not fixed on an invariable value. Each person has his own temperature scale. The temperature can vary depending on age, sex, the time of day, activity and emotions, depending on the temperature-taking method (rectal, oral, axillary, ear, forehead). Temporal temperature-taking, located the nearest possible to the brain and the heart, is considered as a reliable measurement because it is close to the core temperature.

When the body temperature drops or increases, the body first regulates its temperature from the brain. As the forehead and ear are the nearest body parts to the brain, they are the first to be informed of a temperature rise.

Your physiological temperature when you are in good health **Visiomed®**, temperature-taking specialist, advises taking your forehead temperature when you are in good health. You can compare these results with when you have a fever.

The different methods of temperature measurement

- Core temperature

Core temperature is the most precise measurement and involves measuring the temperature in the pulmonary artery by means of a catheter equipped with a thermal probe which can read the temperature in situ. The same method is employed for probes measuring the esophageal temperature. However, such invasive temperature measurement methods require specific equipment and expertise.

- Rectal temperature

Rectal temperature adjusts slowly in comparison to the evolution of the body's internal temperature. It has been demonstrated that rectal temperature remains raised long after the internal temperature of the patient has started to drop and vice versa. Furthermore, rectal perforations have been known to occur as a result of this method and without appropriate sterilization techniques, rectal thermometry can spread germs often found in faeces.

- Oral temperature

Oral temperature is easily influenced by recent ingestion of food or drinks and by breathing through the mouth. To measure oral temperature, the mouth must remain closed and the tongue lowered for three to four minutes which is a difficult task for young children to accomplish.

- Axillary (armpit) temperature

Although it may be easy to measure axillary temperature, it has been proven that it does not provide an accurate measurement of the child's internal temperature. To take this type of temperature, the thermometer must be wedged tightly over the axillary artery. Despite the low sensitivity and relative inaccuracy of axillary temperature in detecting fever, this method is recommended by *The American Academy of Pediatrics* as a screening test for fever in newborns.

- Tympanic thermometry

In order to obtain a precise temperature reading, good command of the measurement technique is required. The thermometer probe must be placed as close as possible to the warmest part of the external ear canal. An incorrectly placed probe could lead to a false temperature reading.

Advantages of temporal artery (ta) temperature

Infrared arterial temperature can be measured using a device placed on the right forehead (temple), in the temporal artery region. It has been demonstrated that this relatively new method of measuring temperature is more precise than tympanic thermo-metry and better tolerated than rectal thermometry.

The **THERMOFLASH® PRO LX-261 EVOLUTION** thermometer has been designed to produce an instant forehead temperature reading without any contact with the temporal artery. As this artery is quite close to the surface of this skin and therefore accessible and given the blood flow is permanent and regular, it allows precise measurement of the temperature. This artery is linked to the heart by the carotid artery which is directly linked to the aorta. It forms part of the main trunk of the arterial system. The efficiency, speed and comfort of taking a temperature from this area make it ideal compared with other temperature measurements methods.

Practical considerations when taking a temperature

- It is essential to remember that although procedures such as taking a temperature may be simple they must not be trivialized.
- In order to ensure that precise and accurate temperature measurements are obtained, it is essential that each user has received adequate information on and training in the temperature measurement technique when using such a device.
- Temperature should be taken in a neutral context. The patient must not have undertaken vigorous physical activity prior to taking his/her temperature and the room temperature must be moderate.
- Be aware of physiological variations in temperature which must be taken into consideration when evaluating the results : temperature increases by 0.5C° between 6am and 3pm. Women have a temperature that is higher, on average, by around 0.2C°. Their temperature also varies in accordance with their ovarian cycle. It rises by 0.5C° in the second half of the cycle and at the early stages of pregnancy.
- When sitting, temperature is lower by about 0.3° to 0.4°C than when standing.

V. DESCRIPTION OF THE THERMOFLASH® PRO LX-261 EVOLUTION

SEE SCHEMA **B** PAGE 2.

- 1** LCD screen
- 2** Setting keys
- 3** Measurement button
- 4** Infrared probe
- 5** Recharging base station
- 6** Mains adaptor
- 7** Wall mounting holes
- 8** Infrared lens
- 9** LED indicator : low battery, charging

VI. FUNCTIONS

1. Specially designed to take the body temperature of a person without interference from the heat of the surrounding environment.
2. Quick and reliable results as it use the HEIMMANN infrared detection system.
3. Sound alarm if temperature is exceeded.
4. Memorization of the 32 last measures.
5. LCD back-lighted digital screen.
6. Data displayed in Celsius or Fahrenheit.
7. Automatic switch-off (energy saving device).
8. Rechargeable base station.
9. Small, convenient, easy to use.

Additional functions:

In *SURFACE TEMP* mode, the **THERMOFLASH® PRO LX-261 EVOLUTION** can also take the temperature of the bath water, baby's bottle...

In *ROOM* mode, the **THERMOFLASH® PRO LX-261 EVOLUTION** can also take the room temperature.

WARNING, when the device is placed on his recharging base, it set itself in pause and charging mode. To take a temperature (*BODY*, *ROOM* or *SURFACE TEMP*) remove the **ThermoFlash®** from it base.

VII. CHARGING THE BATTERY

Before using the **ThermoFlash® PRO LX-261 EVOLUTION** for the first time, charge the battery for at least 3 hours.

Internal battery

3.7V Li-ion battery - Rechargeable battery that can be used regularly.

Recharging base station

Input : Voltage AC 100~240V - 50 ~ 60Hz - Output : DC 6.0V 400mA - Power cable VDE 2 plug.

In order to charge your **ThermoFlash®** and for reasons of safety, place the device on the recharging base station.

Charge indicator light

When the **ThermoFlash®** battery needs charging, the  symbol will be displayed on the screen.

You may continue to use the **ThermoFlash®** which will emit 5 continuous beeps and will switch off automatically once the battery is completely flat.

When the **ThermoFlash®** is placed onto the recharging base station, the LED indicator will be blue during charging. Once the **ThermoFlash®** battery has been completely recharged, the LED will be green and power to the recharging base station will be automatically switched off to avoid overload and prolong the life of the battery.

Recharge battery every half year when the device is not used for a long period of time.

VIII. STARTING UP YOUR THERMOFLASH® PRO LX-261 EVOLUTION

1. Connect the mains adaptor to the recharging base station.
2. Before using the **THERMOFLASH® PRO LX-261 EVOLUTION** for the first time, charge the battery for at least 3 hours. During charging the device will display the room temperature.
3. Press the button *MODE* to select your temperature measurement mode.
4. Aim at the right forehead (temple) (SEE SCHEMA  PAGE 2) from distance of 2-5 cm (2 in), press the measurement button. The temperature is instantly displayed. The temperature can also be taken behind the ear lobe.
5. Before taking any temperature, make sure to remove hair and perspiration from the right forehead (temple).

The reliability of the measurement cannot be guaranteed if the temperature is measured over another part of the body (e.g. arm, torso...).

IX. SETTINGS

1. Choosing the temperature mode

The **THERMOFLASH® PRO LX-261 EVOLUTION** is specially designed to measure body temperature without contact, of children or adults. For this, use the *BODY* mode. Measurement range for *BODY* mode : 32°C – 42.9°C (90°F – 109°F).

Note : You can also use the THERMOFLASH® PRO LX-261 EVOLUTION to measure the temperature of an area or an object, food, or a liquid. For this, use the SURFACE TEMP mode.

Measurement range for *SURFACE TEMP* mode : 0°C – 60°C (32°F - 140°F).

By selecting the *ROOM* Mode, it will display the ambient temperature of the room you are in.

Press the *MODE* button for one second, the screen will display : *BODY* for the body measurement temperature. Press the *MODE* button again to switch to *SURFACE TEMP* or *ROOM* mode.

Note : The THERMOFLASH® PRO LX-261 EVOLUTION is automatically set to BODY.

Important : The surface temperature differs from the internal body temperature. To obtain the internal temperature, always use the *BODY* mode. Please make sure to select the *BODY* mode for an internal temperature reading and the *SURFACE TEMP* mode or the *ROOM* mode for an external area reading (bottle, bath, room...).

2. Choosing the temperature unit

Press the °C/°F button to select the temperature unit. The symbol of the chosen temperature unit is displayed (°C for Celsius degrees and °F for Fahrenheit degrees).

3. Data memory

To display the last temperature measurement, press the *MEM* button.

You will obtain the last temperature measurement. To switch on the last measurement, press the °C/°F button.

The number indicated in interval of two measures corresponds to the number of measurements.

Press on the button *MODE* for review of the previous measurements.

To exit data memory, press the measure key. The **THERMOFLASH® PRO LX-261 EVOLUTION** switches off after 5 seconds.

4. Alarm setup

The alarm threshold default value is 38°C (100.4°F). It cannot be changed. In case of higher temperatures than 38°C, a signal beeps while measuring the temperature.

5. Default settings

To reset the default settings, press the *RESET* button using a pointed object.

X. TECHNICAL SPECIFICATIONS AND PRECISION

1. Normal use conditions : Operating temperature: 10°C~40°C (50~104°F). Normal conditions of storage and transportation temperature : -20°C~50°C (-4°F~122°F). Humidity level: 10 - 85%
2. Measurement display unit: °C or °F.
3. Display resolution: 0.1°C (0.1°F).
4. Measurement range:
In *BODY* mode: 32.9°C - 42.9°C (90°F – 109°F).
In *SURFACE TEMP* mode: 0°C - 60°C (32°F - 140°F).
5. Consumption: ≤ 50mW.
6. Measurement distance: 5 cm (2 in).
7. Automatic switch-off: 5 seconds.
8. Accuracy: 36°C - 39°C (96.8°F - 102.2°F) = ± 0.2°C.
9. Battery voltage: DC 3.7V.
10. Charging time: 3 hours
11. Use of battery at full charge: > 10,000 cycles.
12. Battery life: > 3,600,000 cycles.
13. Calibration check: 100,000 cycles.
14. Net weight: 180g.
15. Size: 155 x 134 x 66 mm.

Recharging base station

1. Input: Voltage AC 100~240V – 50~60Hz.
2. Output: DC 6.0V 400mA - Power cable VDE 2 plug.

The **THERMOFLASH® PRO LX-261 EVOLUTION** can take temperature measurements below 32.9°C or above 42.9°C. Outside of these temperature ranges, precision cannot be guaranteed.

Compliance

This device meets the requirements of EMC Standard CEI 60601-1-2 Edition 2 concerning medical electrical devices. The manufacturer shall not be held liable for any interference caused by accessories other than those supplied or by unauthorized changes or modifications made to the device. Any unauthorized changes or modifications made by the user may result in the user forgoing the right to use the device.

XI. CLEANING INSTRUCTIONS

- The protective glass over the lens is the most important and fragile part of the thermometer, please take great care of it.
- Clean the glass with cotton fabric, wet with water or 70° alcohol.
- Do not immerse the device in water to avoid damage.
- Disconnect the power source from the device and remove the battery before cleaning.
- Clean all parts after use as indicated in the manual.
- Ensure that the electrical connector socket does not come into contact with the water solution or disinfectant (socket at the base of the device).

XII. ACCESSORIES SUPPLIED

Charger and recharging base station, Wrist strap, Double-sided adhesive patch, Wall-fixing screws, User manual, Guarantee card.

XIII. TROUBLESHOOTING

If you have one of the following problems while using your **THERMOFLASH® PRO LX-261 EVOLUTION**, please refer to this breakdown service guide to help resolve the problem. If the problem persists, please contact our customer service at +33 892 350 334.

- **The screen displays temperature superior to 95°**: the temperature is in Fahrenheit. Change the measurement unit to Celsius.
- **The screen displays the body temperature inferior to 32°C (89.6°F)** : to measure a body temperature, the *BODY* mode must be selected. If you are on *SURFACE TEMP* mode, the 32°C (89.6°F) temperature displayed is showing the external temperature that your body releases.
- **The screen displays the message HI** : when using the **THERMOFLASH® PRO LX-261 EVOLUTION**, the message *HI* can show on the screen. The analysis is above the measurement range selected, either superior to 42.9°C (109°F) in *BODY* Mode or superior to 60°C (140°F) in *SURFACE TEMP* Mode.
- **The screen displays the message LO** : when using the **THERMOFLASH® PRO LX-261 EVOLUTION**, the message *Lo* can show on the screen. The temperature analysed is under the measuring range selected, either less than 32°C (90°F) in *BODY* Mode or less than 0°C (32°F) in *SURFACE TEMP* Mode.

This message displays in various cases – please find below a list of the main cases :

Reasons for LO message display	Advice
Temperature reading hampered by hair, perspiration...	Make sure that there is no obstruction prior to taking a temperature.
Temperature hampered by an air flux.	Make sure there is no air flux as this could interfere with the infrared system.
Temperature readings too close together, the ThermoFlash® did not have the chance to boot itself.	Make sure there is no air flux as this could interfere with the infrared system.
The measuring distance is too far.	Please respect the measuring distance (between 2 and 5 cm).

ES

ThermoFlash®

MANUAL DE UTILIZACIÓN

CE
0197

MicroSecond Flash™
TECHNOLOGY
NEW GENERATION

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EMC DATA

EMC data

Guidance and manufacturer's declaration - electromagnetic immunity – for ME equipment and ME systems that are not life-supporting. The PRO LX-261 EVOLUTION is intended for use in the electromagnetic environment specified below. The customer or the user of the PRO LX-261 EVOLUTION should assure that it is used in such an environment.

The customer or the user of the PRO LX-261 EVOLUTION should assure that it is used in such an environment.

Immunity test	EN 60601 test level	Compliance level	Electromagnetic environment - guidance
Conducted RF	3 Vrms	3V	Portable and mobile RF communications equipment should be used no closer to any part of the model PRO LX-26E, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.
IEC 61000-4-6	150 kHz to 80 MHz		Recommended separation distance $d = \left[\frac{3.5}{V1} \right] \sqrt{P}$
Radiated RF	3 V/m	3V/m	$d = \left[\frac{3.5}{E1} \right] \sqrt{P}$ 80 MHz to 800 MHz
IEC 61000-4-3	80 MHz to 2.5 GHz		$d = \left[\frac{7}{E1} \right] \sqrt{P}$ 800 MHz to 2.5 GHz where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in meters (m). Fields strengths from fixed RF transmitters, as determined by an electromagnetic site survey, should be less than the compliance level in each frequency range. Interference may occur in the vicinity of equipment marked with the following symbol: 

Note 1: At 80 MHz and 800 MHz, the higher frequency range applies.

Note 2: These guidelines may not apply in all situations. Electromagnetic is affected by absorption and reflection from structures, objects and people.

a. Fields strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the model PRO LX-261E is used exceeds the applicable RF compliance level above. The

model PRO LX-261E should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary such as reorienting or relocating the model PRO LX-261E.

b. Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3V/m.

Recommended separation distances between portable and mobile RF communications equipment and the equipment or system - for equipment and systems that are not life-supporting. The model PRO LX- 261E is intended for use in an electro-

magnetic environment in which radiated RF disturbances are controlled. The customer or the user of the model PRO LX-261E can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the model PRO LX-261E as recommended below, according to the maximum output power of the communications equipment.

Rated maximum output of transmitter	Separation distance according to frequency of transmitter m		
	80 MHz to 800 MHz $d = \left[\frac{3.5}{V1} \right] \sqrt{P}$	800 MHz to 2.5 GHz $d = \left[\frac{3.5}{E1} \right] \sqrt{P}$	800 MHz to 2.5 GHz $d = \left[\frac{7}{E1} \right] \sqrt{P}$
0.01	0.12	0.12	0.23
0.1	0.38	0.38	0.73
1	1.2	1.2	2.3
10	3.8	3.8	7.3
100	12	12	23

For transmitters rated at a maximum output power not listed above the recommended separation distance d in meters (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

Note 1: At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.

Note 2: These guidelines may not apply in all situations. Electromagnetic propagations is affected by absorption and reflection from structures, objects and people.

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EN: Visiomed* will repair or replace this product free of charge in the case of defective parts or manufacturing defects, in accordance with the conditions mentioned below as follows:

DURATION: 24 MONTHS RETURN TO WORKSHOP

LIMITS AND EXCLUSIONS: This guarantee concerns only the original final purchaser. A purchase invoice, or another proof of purchase, with this guarantee card will be required to obtain an after-sales service, in accordance with this guarantee. This guarantee card will not be extended to another person only the original final purchaser. This guarantee becomes void if the serial numbers on the product are modified, replaced, illegible, absent, or if repair has been carried out by a service not approved, including the user.

This guarantee covers only the defects of the material or parts, occurring during normal use of the product. It does not cover the damage caused during the transport of the apparatus, causes due to repairs being carried out by the distributor, by any modifications undertaken, any connection of equipment not approved by Visiomed*, or causes contrary to those written in the user manual or notice. Moreover, the present guarantee does not cover damage due to falls, bad handling, bad installations, damage by fire, flooding, lightning, or any other natural disaster. This guarantee does not cover the packing of the material, the accessories, the defects caused by commercial exposure of the product, show room, sale space, demonstration etc... Normal maintenance, cleaning and the replacement of parts where wear is normal, are not covered by the terms of this guarantee. Visiomed* and its representatives and agents will not in any case be held responsible for any damage and consecutive damages due to the mishandling of this product. This guarantee is the only valid one at Visiomed*, any other guarantee (commercial guarantee) except this one will not be taken into account.

IMPORTANT: During the guarantee period if you are dissatisfied with the repairs of this product, please contact the Visiomed* customer service.

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EN: Disposal of Waste Equipment by Users in Private Households in the European Union.

The symbol on the product or on its packaging indicates that this product must not be disposed of with your other household waste. Instead, it is your responsibility to dispose of your waste equipment by handing it over to a designated collection point for the recycling of waste electrical and electronic equipment. The separate collection and recycling of your waste equipment at the time of disposal will help to conserve natural resources and ensure that it is recycled in a manner that protects human health and the environment. For more information about where you can drop off your waste equipment for recycling, please contact your local city office or your household waste disposal service.

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©12/2015 PRO LX-261E
EN: ThermoFlash is a brand registered by Visiomed Group SA. The constructor keeps the right to modify the photographs and characteristics without previous notice. Registered design. The logos and trademarks are the property of their respective owners.