MANUFACTURED FOR

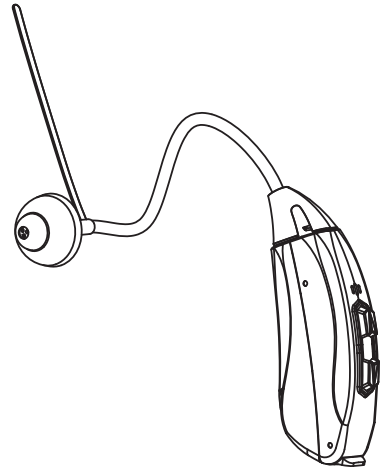
EC REP

VISIONED GROUP SA
112, Avenue Kléber
75116 PARIS - France

Made in China

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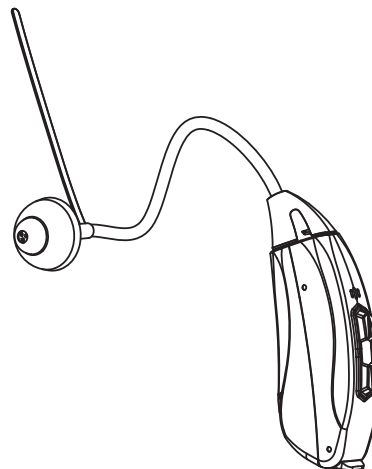
VM-AS10

TABLE OF CONTENTS

1. CONTENTS OF YOUR ASTON PACK	21
2. WARNINGS AND SAFETY PRECAUTIONS	22
A. SAFETY INSTRUCTIONS	22
B. PRECAUTIONS FOR USE	23
3. INDICATION	24
4. INSTRUCTIONS FOR USE	24
A. ASSEMBLY OF THE ASTON HEARING AMPLIFIER	24
B. STARTING THE ASTON ASSISTIVE HEARING AMPLIFIER	25
C. STARTING THE ASTON HEARING AMPLIFIER	26
D. ADJUSTING THE VOLUME	27
E. MEMORY BUTTON	27
5. MAINTENANCE AND SERVICE	27
A. CHANGING THE BATTERIES	27
B. CHANGING THE EAR TIP	28
C. CHANGING THE TUBE	28
D. CLEANING	29
6. TROUBLESHOOTING GUIDE	30
7. DISPOSAL AND RECYCLING	31
8. TECHNICAL CHARACTERISTICS	31

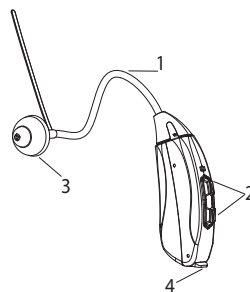
The manufacturer reserves the right to modify the product technical specifications without prior notice.

Dear client,


Thank you for purchasing the VISIOMED Aston VM-AS10 hearing amplifier. In order to use it optimally and efficiently, we recommend that you read these instructions carefully

1. CONTENTS OF YOUR ASTON PACK

- 1 Aston hearing amplifier
- 2 tubes (1 right, 1 left)
- 3 ear pieces (small, medium, large)
- 1 cleaning brush and cleaning wire
- 1 pack of 6 batteries

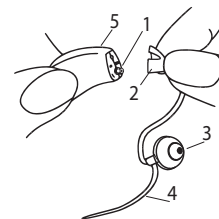1. Tube
2. Adjustment wheel
3. Ear tip dome
4. Battery compartment

1. Microphone
2. Tube triangular base
3. Ear tip dome
4. Stabilizer bar
5. Main device

2. WARNINGS AND SAFETY PRECAUTIONS

Aston is a pre-set hearing amplifier. This medical device is a regulated health product, which, under these regulations, bears CE marking. Its sale is authorised in accordance with the ministerial decree of August

13, 2014.

WARNING :

Before use, an adjustment period is necessary in order to fully appreciate the qualities of your Aston product. Certainly, by amplifying the surrounding sounds, your Aston hearing amplifier will open up a new world to you. At first, these sounds may seem disruptive, but as you progress, you will gradually become familiar with the new sound in your environment.

A. SAFETY INSTRUCTIONS

- Follow the maintenance instructions in this manual.
- This device is suitable for home use.
- Use this device only for its intended use as described in this notice.
- Use this device in an ambient temperature range between 10°C - 38°C / 14 - 100.4°F.
- Do not use this device in relative humidity greater than 85%.
- Storage and transport between -20 and 60°C / -4 - 140°F.
- This device should always be stored in a clean and dry area.
- If the device is stored or used in temperature or humidity conditions exceeding the limits specified in this manual, its performance and safety may be corrupted.
- Do not drop the device, avoid jolts and shocks. A shock could damage it.
- Do not insert anything into the device except the batteries.
- Do not attempt to open the device. In case of problems, contact the after sales service department.
- Do not disassemble the device.
- Do not put it in your mouth. If swallowed, contact a doctor.
- Never drop the device. A shock may damage it.
- Do not expose this device to electric shock.
- Do not expose this device to extreme temperature conditions.
- Do not expose this device to direct sunlight or water.

- Do not put the device in water.
- Do not bathe with the device. In general, protect the device from moisture.
- Only use accessories and spare parts specified by the manufacturer or sold with the device.
- It is essential to remove your Aston hearing amplifier before taking a shower or bath. In general, protect your device from moisture.
- Stop using the device in the event of an abnormality or malfunction.
- Keep away from electronic devices such as TV or audio equipment.
- Never use the device for any purpose other than that for which it was designed. Any other use should be considered dangerous and inappropriate.
- Keep out of the reach of children and pets.
- This device is not intended for use by persons (including children) with impaired physical, sensory or mental capabilities, or persons with no experience or knowledge of the device, unless they have undergone supervision or prior instructions concerning the use of this device by a person responsible for their safety.
- This device should not encourage self-medication or adjusting your treatment.

B. PRECAUTIONS FOR USE

- Do not use this device in case of ear infection. Ask your doctor for advice.
- For any questions about your hearing loss, please consult your doctor.
- This device is not a hearing aid.
- Remove the batteries when not using the device for a long period.
- Do not use batteries other than the batteries mentioned, do not recharge non-rechargeable batteries, or throw them into a fire.
- Take care to observe correct battery polarity. A polarity error may cause damage and void the guarantee.
- Do not handle the tube with your fingernails to avoid malfunction.
- In case of persistent discomfort, ear canal obstruction or allergic reaction, please consult an ENT specialist.
- Some irritation is possible if you are not used to wearing this type of device. In case of swelling or redness, remove the device and consult your doctor.

3. INDICATION

Correction of mild to moderate age-related hearing loss (presbycusis). It is not a hearing aid. This hearing amplifier amplifies surrounding sounds and removes some of the background noise. It allows hearing-impaired people to compensate for their hearing loss.

4. INSTRUCTIONS FOR USE

A. ASSEMBLY OF THE ASTON HEARING AMPLIFIER

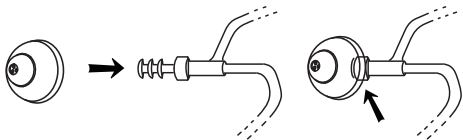
1. Select the tube corresponding to the ear (right or left) on which the ASTON Hearing amplifier must be worn.

The tube with the red mark 2B corresponds to the right ear. The tube with the blue mark 2B corresponds to the left ear.


2. Select the ear tip best suited to your ear.

Note: It is advisable to start with the larger ear tips. If it does not fit properly into your ear canal, replace it with a smaller size.


3. Insert the end of the tube into the hole in the centre of the ear tip and exert a slight pressure.

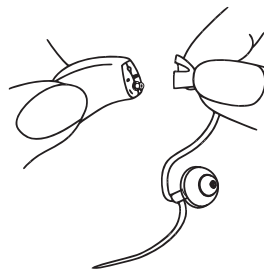4. Connect the hearing amplifier and the triangular base of the tube by holding them at each end. Do not twist the tube during this operation.

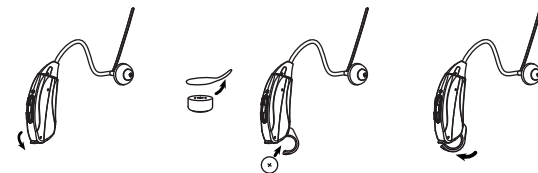B. STARTING THE ASTON ASSISTIVE HEARING AMPLIFIER




1. Peel off the tab on the flat side of the battery.
2. Carefully open the battery compartment and insert the battery.
Caution: Handle the battery compartment carefully. A polarity error prevents closing of the device. Do not force closure.
3. The hearing amplifier is on and ready for use.

Note: For optimal use, we recommend putting the hearing amplifier in its case with the battery compartment open when not in use.


C. STARTING THE ASTON HEARING AMPLIFIER

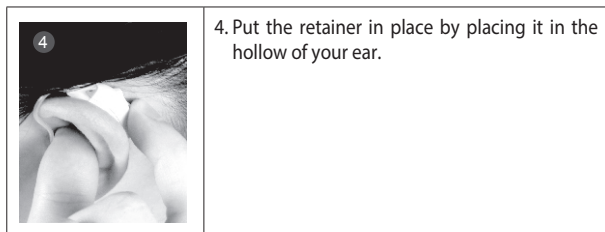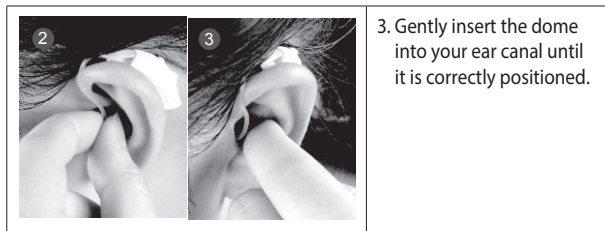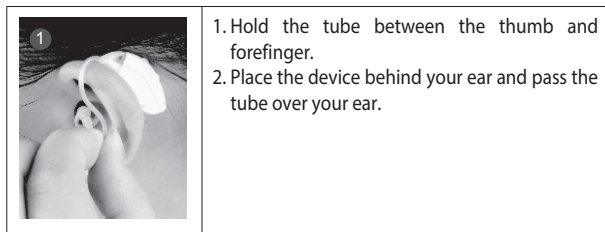Clean the ear in which the device is to be inserted. Make sure the battery compartment is securely closed.

Note: If too long or uncomfortable the stabilizer can be adjusted (cut by scissor).

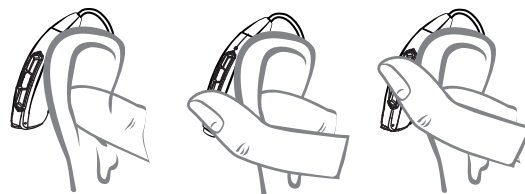D. ADJUSTING THE VOLUME

The switch on the unit is used to control the volume. Briefly press the top button to increase the volume or the lower button to decrease the volume. Each short press adjusts the volume.

E. MEMORY BUTTON

The switch on the unit is also used to choose the programmes. By default, the hearing amplifier is set to programme 1.

To switch to programme 2, press the switch for 3 seconds to change the programme. The device beeps once for programme 1 and twice for programme 2.

Select the one that best suits your hearing.

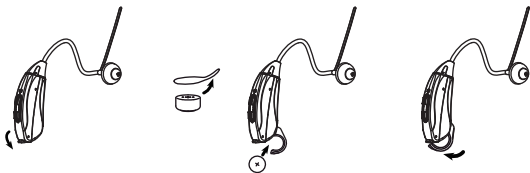
5. MAINTENANCE AND SERVICE

A. CHANGING THE BATTERIES

When the device emits a short intermittent beep, the batteries are low or empty. The sound quality may be affected until the battery is replaced.

The batteries (supplied in the box) allow a continuous supply of about 7 days.

1. Open the battery compartment.
2. Replace the used battery, taking care to observe polarity. Never insert rechargeable batteries. Use disposable batteries.
3. Handle the battery compartment carefully. A polarity error prevents closing of the device. Do not force shut.

Do not dispose of old batteries with household waste. Take your used batteries to a collection point.

If not used for a long period of time, be sure to remove the battery from the device to prevent leakage of liquid that may damage the device. Use high-quality batteries to prevent malfunctions.

Keep batteries out of the reach of children. Do not put the device in your mouth. If swallowed, contact the emergency services.

B. CHANGING THE EAR TIP

1. Hold the end of the tube and gently pull on the ear tip. Do not handle the tube with your fingernails.
 2. Replace the ear tip as described in the section Instructions for use.
- In order to preserve the quality of your Aston hearing amplifier, it is advisable to change the ear tips every 2 weeks.

C. CHANGING THE TUBE

1. Disconnect the tube from the main unit by rotating the triangular base of the tube by 90 degrees. Pull the triangular base firmly to release it from the device.
2. Replace the tube as indicated in the instructions for use section > Assemble your hearing amplifier.
3. In order to preserve the quality of your Aston hearing amplifier, it is advisable to change the tube every 3 months.

D. CLEANING

Cleaning the main device:

Use a soft, clean cloth to wipe the main device daily before and after use.

Do not use water or other liquids.

Cleaning the tube and the ear tip:

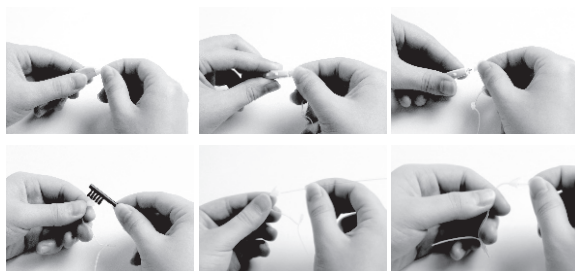
Ear wax is one of the main causes of malfunction of the hearing amplifier.

Filters and other devices are present inside the device to prevent wax deposit. It is easier to clean the device before using it, when the earwax is dry.

Using the brush, remove the impurities located on the ear tip and inside it.

Twist the brush to remove impurities and cerumen particles inside the ear tip.

To clean the inside of the tube, insert the cleaning wire inside the tube, and push it until it comes out through the other end of the tube.

Remove the impurities and particles of cerumen from the tube and ear tip with the brush.

Clean the inside of the tube with the cleaning wire (green).

6. TROUBLESHOOTING GUIDE

SYMPTOM	POSSIBLE CAUSES	SOLUTIONS
No sound	No battery	Insert the battery.
	Battery low or empty	Replace the battery.
	Battery incorrectly inserted	Insert the battery correctly.
	Earwax or debris in the microphone or receiver	Clean the microphone and receiver with the brush.
The battery compartment cannot be closed	The battery is inserted in the wrong direction (polarity)	Insert the battery with the correct polarity.
Sound not powerful enough	Battery empty	Replace the used battery.
	Damaged ear tip	Replace the ear tip.
	Volume too low	Turn the condenser clockwise.
	Wax or debris in the microphone or receiver	Clean the microphone and receiver with the brush.
	Change in hearing loss	Contact your ENT specialist.
	Too much earwax	Contact your ENT specialist.

Whistling, Larsen	Ear tip not adapted	Try a larger ear tip.
	Battery low	Replace the battery.
	Hearing amplifier incorrectly inserted	Remove the device and replace it.
	Too much earwax	Contact your ENT specialist.

7. DISPOSAL AND RECYCLING

Dispose of your Aston hearing amplifier in accordance with local laws and regulations. Aston contains electronics and a battery; it cannot be disposed of with household waste.

8. TECHNICAL CHARACTERISTICS

Product name		Aston hearing amplifier
Model		VM-AS10 (tube)
Classification		Class IIa
Overall sound pressure level (dB)		96±3
Overall sound pressure level HF-Medium (dB)		86±4
Amplification (dB)		23±5
Average HF amplification (dB)		16±5
Equivalent Input Noise (dB)		≤32
THD Distortion (%)	500	≤1
	800	
	1600	
Current consumption		≤1.5


Bandwidth (Hz)	F1	≤ 500
	F2	≥ 3500

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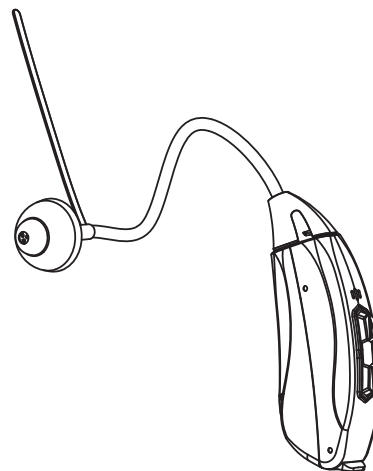EN: Visiomed® will repair or replace this product free of charge in the case of defective parts or manufacturing defects, in accordance with the conditions mentioned below as follows:

DURATION: 24 MONTHS RETURN TO WORKSHOP

LIMITS AND EXCLUSIONS: This guarantee concerns only the original final purchaser. A purchase invoice, or another proof of purchase, with this guarantee card will be required to obtain an after-sales service, in accordance with this guarantee. This guarantee card will not be extended to another person only the original final purchaser. This guarantee becomes void if the serial numbers on the product are modified, replaced, illegible, absent, or if repair has been carried out by a service not approved, including the user.

This guarantee covers only the defects of the material or parts, occurring during normal use of the product. It does not cover the damage caused during the transport of the apparatus, causes due to repairs being carried out by the distributor, by any modifications undertaken, any connection of equipment not approved by Visiomed®, or causes contrary to those written in the user manual or notice. Moreover, the present guarantee does not cover damage due to falls, bad handling, bad installations, damage by fire, floods, lightning, or any other natural disaster. This guarantee does not cover the packing of the material, the accessories, the defects caused by commercial exposure of the product, show room, sale space, demonstration etc... Normal maintenance, cleaning and the replacement

of parts where wear is normal, are not covered by the terms of this guarantee. Visiomed® and its representatives and agents will not in any case be held responsible for any damage and consecutive damages due to the mishandling of this product. This guarantee is the only valid one at Visiomed®, any other guarantee (commercial guarantee) except this one will not be taken into account. **IMPORTANT:** During the guarantee period if you are dissatisfied with the repairs of this product, please contact the Visiomed® customer service.

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EN: Disposal of Waste Equipment by Users in Private Households in the European Union.

For more information about where you can drop off your waste equipment for recycling, please contact your local city office or your household waste disposal service.

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EN: Aston is a brand registered by Visiomed Group SA. The constructor keeps the right to modify the photographs and characteristics without previous notice. Registered design. The logos and trademarks are the property of their respective owners.

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© 08/2017 Aston