

THERMOISOFT VM-DS400 - VM-DS200 - DIGITAL THERMOMETER USER MANUAL

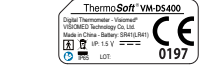1 2 3 4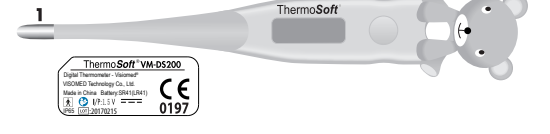

1 Sonde flexible de mesure / Flexible measurement probe / Flexibler Messfühler / Flexibele meetsonde / Sondadi misur a flessibile / Punta flexible de medición / Sonda de medição flexível / Elastyczna sonda pomiarowa / إسبار قابل من / Ölçüm sondası

2 Ecran / Screen / Anzeige / Scherm / Display / Pantalla / Ecr  / Ekran / شاشة / Otkry

3 Touch / On/Off / On/Off Switch / Ein/Aus-Taste/Aan/Uit-schakelaar / Tasto On/Off / Interruptor On/Off / Tecla On/Off / Prizykcion On/Off / الازفاتير التاشيفي / Açma / Kapama du mesi

4 Compartiment a pile / Battery compartment / Batteriefach / Batterivak / Scompartito pile / Compartimento bacteria / Compartimento de pilhas / Zámknki ce baterii / موقع الفلزات / Pili yuvası

Températures normales relevées sur différentes parties du corps / Normal temperatures measured on different parts of the body / Normaltemperaturen an verschiedenen Körperparten gemessen / Normalte temperaturen gemeten op diverse lichaamsdelen / Temperature normali rilevate su diverse parti del corpo / Temperaturas normales medidas en diferentes partes del cuerpo / Normalne temperature mierzona na r nych cz ciaa / قياس وحدات الحرارة العادية من مختلف اجزاء الجسم / Vudum farklı bolgelerinde kaydedilen Normal sıcaklıklar

Auriculaire / Auricular / Aurikular / Oor / Auricolare / Oldo / Ucho / Aurik�ler	35.8 -37.6°C 95 -98.6°F
Orale / Oral / Oraal / Usta	35.5 -37.8°C 95 -98.6°F
Axillaire / Axillary / Axillar / Oksel / Ascellare / Axilla / Axillar / Pacha / Aksiller	34.8 -37.4°C 93.2 -98.6°F
Rectale / Rectal / Rektal / Rectaal / Rettale / Retal / Odbyt / Rektal	36.4 -38.2°C 96.8 -100.4°F

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UK INTENDED USE: The product is exclusively meant to be used for human body temperature measuring, suitable for all age infants, children and adults. It can be used for households, and for medical purposes, providing the reference for clinical diagnosis. Measuring areas of body can be axillary, oral or rectum.
Advice : A temperature taken in the rectum is the closest way to finding the body's true temperature.
Axillary and oral temperature measurements need over long time periods and have a disadvantage to frequently underestimate measurement.
SAFETY & WARNING PRECAUTION

- Please keep the thermometer off the children touch. Prevent the product from children playing, swallowing and eating.
- Do not shock the thermometer or let it fall down.
- The product is 100% waterproof, it is not recommended to immerse it in water or other liquids.
- Do not expose the thermometer to direct sunlight, high temperature/humidity or any extreme environment, or the function will be reduced.
- Stop using the thermometer when it's abnormal or malfunctions.
- Do not open the thermometer case unless replacing the batteries.
- Do not put the thermometer in contact with diseased, ulcer or bruise skin.
- Taking rectal temperatures can cause internal injury. In general, bleeding from rectum, hemorrhoids, and recent lower intestinal surgery would place the person at high risk for further injury.
- Be sure that the baby or patient has not had a bath or been swaddled (when babies are wrapped tightly for warmth) in the past 20 minutes. This can cause an inaccurate reading.
- Do not drink cold/hot beverages or exercise vigorously when measuring the temperature.

- THIS MEDICAL ELECTRICAL EQUIPMENT needs special precautions regarding EMC and needs to be installed and put into service according to the EMC DATA table.
- Portable and mobile RF communications equipment can affect MEDICAL ELECTRICAL EQUIPMENT.
- Only use the recommended batteries, do not burn them.
- The measuring result of this product is only for your reference. If you have any doubt, please contact your doctor.
- Do not modify this equipment without authorization of the manufacturer.
- Do not store or operate the device outside prescribed environmental conditions, as running performance of the device might be affected or the device might not work properly.
- Stop using the thermometer if the LCD displays the "E", it indicates the sensor circuit detecting an error (such as open or short circuit).
- The default alarm temperature is 37.5°C (99.5°F), the alarm sound is "Di Di Di...". If the test temperature is $\geq 32^{\circ}\text{C}$ (89.6°F), LCD display "L"; if the temperature $\geq 44^{\circ}\text{C}$ (111.2°F), LCD display "H"; the buzzer will alarm.

INSTRUCTIONS BEFORE USE

Clean the tip of the rectal thermometer with soapy water or rubbing alcohol. For hygiene reasons, if the thermometer is ever used in rectal mode it should never again be used for measurement of another part of the body.

OPERATING INSTRUCTIONS

- Switch on the thermometer, press the On/Off button. The appliance emits a beep. The function symbols are displayed on the screen for a few seconds. The result of the last temperature measurement is displayed for two seconds, then "36.5 C/97.7 F" and "C" or "F" icons appear in the top right hand corner of the screen. The thermometer is then ready to use.
- How do I take a temperature?
- To take a rectal temperature
- This is the most reliable way to ensure a fever is not missed.
- If necessary lubricate the tip of the thermometer with a small amount of petroleum jelly. Place your child face down across the top of the thermometer. Place your child up with legs bent toward his or her chest. Gently insert the tip of the thermometer about ½ inch (1.25 cm). Never force the thermometer into the rectum. Stop if you feel any resistance. Hold it there until it beeps.

- Remove the thermometer carefully. Read the temperature and record it. Temperatures taken rectally usually read ¼ degree F (0.5 degree C) higher than temperatures taken orally. As you remove the thermometer, be sure the disposable sleeve is removed from the patient's rectum, if you use one on the thermometer.
- To take an oral temperature
- The oral method is not recommended for children younger than 5 years old because it is hard for them to hold the thermometer under their tongue for long enough.
- Put the tip under your tongue as far as it can go. Close your lips gently around the thermometer. Do not bite the thermometer. Relax and breathe after your nose. Keep the thermometer under your tongue until it beeps.

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