**VISIONED GROUP SA**  
 112, Avenue Kléber  
 75116 PARIS - France

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## USER MANUAL

# EN

KINECARE<sup>®</sup> products are suitable for Heat therapy or Cold therapy to relieve your daily aches and pains.

### Multizone (GB1)

M - 11x27 cm

**Cold usage:** Sprains, bruising, inflammation, muscle strain and post-workout pain, dental pain.

**Heat usage:** Muscle pain, joint stiffness, pre-workout pain.

### Multizone (GB2)

M - 10x15 cm

**Cold usage:** Sprains, bruising, inflammation, muscle strain and post-workout pain, dental pain.

**Heat usage:** Muscle pain, joint stiffness, pre-workout pain.

### Multizone (GB3)

XL - 20x30 cm

**Cold usage:** Sprains, bruising, inflammation, muscle strain and post-workout pain, dental pain.

**Heat usage:** Muscle pain, joint stiffness, pre-workout pain.

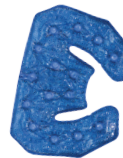
### Eye mask (GB4)

M - 10x20 cm

**Cold usage:** Sinusitis, headache, palpebral edema (swollen eyes), contact eczema, post-eyelid surgery.

**Heat usage:** Eye dryness, eye rosacea, nasal congestion.

*Available in 4 colours*

### Neck and shoulders (GB5)

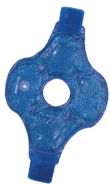
XXL - 45x32 cm

**Cold usage:** Shoulder pain.

**Heat usage:** Neck muscular pain, stiffness, stiff neck.

*\* To be used according to your surgeon's recommendations.*

By **Visiomed**<sup>®</sup>

#### **Knee (GB6)**

L - 30x22 cm

**Cold usage:** Arthritis, joint pain, pre and post workout pain, inflammation after surgery.

**Heat usage:** Osteoarthritis.

#### **Lumbar (GB7)**

XL - 43x17 cm

**Cold usage:** Contracture, blow, hematoma.

**Heat usage:** Back-spinal pain, sciatica, menstrual pain.

#### **Ankle, wrist, elbow (GB8)**

M - 35x11 cm

**Cold usage:** Sprain, oedema, carpal tunnel syndrome, contractures, blow, hematoma, bruising, inflammation, muscle strain and post-workout pain.

**Heat usage:** Muscle pain, contractures, osteoarthritis, pre-workout pain.

#### **Kids (GB9)**

S - 12,5x8 cm

**Cold usage:** Little pains, blow/bumps, hematomas, little burns, insect stings, fever/headaches.

**Heat usage:** Belly stomach, infant colic.

*Available in 6 colours*

#### **Forehead, Temple, Sinus (GB10)**

M - 35x10 cm

**Cold usage:** Sinus pain, headaches.

**Heat usage:** Sinus pain, tensions of the frontal/temporal muscles.

## Medical device class I, compliant with European Standards.

### TABLE OF CONTENTS

<b>1. COMPOSITION</b> .....	<b>16</b>
<b>2. SAFETY NOTES</b> .....	<b>16</b>
<b>2.1. WARNINGS</b> .....	<b>16</b>
<b>2.2. PRODUCT PRECAUTIONS (CAUTION)</b> .....	<b>17</b>
<b>3. TERMS AND CONDITIONS</b> .....	<b>18</b>
<b>4. CLEANING AND MAINTENANCE</b> .....	<b>19</b>
<b>4.1. CLEANING</b> .....	<b>19</b>
<b>4.2. MAINTENANCE</b> .....	<b>19</b>

Dear customer,

You have just purchased the KINECARE® Micro-Ball Gel Thermal Pad and we thank you.

We strongly recommend that you read these instructions carefully.

#### 1. COMPOSITION

Outer material: PVC, 100% Polyester;

Fill: Glycerin, Water, Sodium Polyacrylate.

#### 2. SAFETY NOTES

Only use this device for its intended purpose as described in this notice.

##### 2.1. WARNINGS

- This product is intended for external use only.
- This device is suitable for home use.
- Store this product in its original packaging in a clean, dry place to prolong its life.
- Do not leave continually this product in the freezer.

- This product can be stored in the refrigerator so it is always fresh and ready for use.
- Do not swallow.
- Do not get in eyes.
- In case of contact with eyes, immediately rinse with clean water and contact your doctor.
- A doctor should be consulted before using the thermal pack on the elderly or in case of diabetes or cardiovascular diseases and/or reduced skin sensitivity (Raynaud's Syndrome)
- Keep this product out of the reach of children.
- The use of this device is not intended for people (including children) with limited physical, sensory or mental abilities or lack of experience and/or lack of knowledge, unless they are supervised by a person responsible for their safety or have been instructed on the use of the device.
- This device should not encourage self-medication or modification of your treatment.
- Follow the maintenance instructions specified in this manual.
- Keep this product in its original packaging in a clean, dry place. Store the product at -20°C to 30°C.

##### 2.2. PRODUCT PRECAUTIONS (CAUTION)

- Never use the compress if it is damaged.
- Do not attempt to perforate the product or gel beads. In the event of a problem, please contact your local dealer.
- Do not use with local analgesic products (such as creams, balms, lotions, liniments...).
- When using, do not press the compress firmly.
- Do not use on damaged skin.
- If the use of heat or cold becomes uncomfortable, remove the compress for a few minutes and then re-apply.
- Do not reheat in a traditional oven or grill. Never reheat immediately after freezing.

- Do not overheat the compress. Overheating can damage the product and cause skin burns.
- If you wish to reheat the compress, check the heating times and temperatures given in the table below.
- Stop heating if the compress begins to inflate like a balloon: this indicates that it is overheated and is likely to burst. Allow to cool before removing.
- If the compress bursts in the microwave oven, clean the oven thoroughly (with water) in order to eliminate any traces of gel.
- If the microwave power is undetermined, use the lowest setting (defrost mode).
- Always check the heat before placing the compress on a baby or a child.
- Stop use and ask a doctor if pain worsens or persists for more than 7 days, or if you have any discomfort, burning, swelling, rash or other changes in your skin.
- Immediately stop application of the product in the event of an uncomfortable or unpleasant sensation.

### 3. TERMS AND CONDITIONS

#### Cold usage

- Place in the freezer for at least 2 hours.
- Apply the compress to the painful area for at least 20 minutes with 20 minutes breaks between applications.
- Check to make sure the temperature of the pack is suitable before application.

#### Heat usage

Heating the bag in the microwave:

- Place the thermal compress flat at the centre of the microwave oven.
- Make sure that the edges do not touch the walls of the microwave oven
- Heat the thermal compress according to the temperature and times indicated.
- The temperature of the product can be higher on its edges. It is advisable to mix it in order to distribute heat on the whole product.

- Always lightly touch test the pack to determine uniform and safe heat levels prior to use to avoid burn.
- If the thermal pack is too hot, allow it to cool down.
- If more heat required, reheat in additional time indicated in the table below, by turning it over on the opposite face, until safe desired temperature is reached for comfortable application.
- Apply the compress to the painful area for at least 20 minutes with 20 minutes breaks between applications.

PRODUCT SIZE CODE	RATING	RECOMMENDED HEATING TIME	REHEATING TIME IF MORE HEAT REQUIRED
<b>S</b>	700 Watts	10 seconds	5 seconds
	1000 Watts	5 seconds	5 seconds
<b>M</b>	700 Watts	15 seconds	5 seconds
	1000 Watts	10 seconds	5 seconds
<b>L</b>	700 Watts	30 seconds	10 seconds
	1000 Watts	20 seconds	5 seconds
<b>XL</b>	700 Watts	40 seconds	10 seconds
	1000 Watts	30 seconds	5 seconds
<b>XXL</b>	700 Watts	50 seconds	15 seconds
	1000 Watts	40 seconds	10 seconds

*Do not reheat in a traditional oven or grill.*

## **4. CLEANING AND MAINTENANCE**

### **4.1. CLEANING**

Wash the gel pack with soap and water, as long as it is not punctured.

The product is not washable.

### **4.2. MAINTENANCE**

Store the thermal compress flat with no pressure on top.

You can store the pack in the refrigerator.

DISPOSAL METHOD: This material is not a hazardous waste.

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© 06/2017 KINECARE
**EN:** Kinecare® is a brand registered by Visiomed Group SA. The constructor keeps the right to modify the photographs and characteristics without previous notice. Registered design. The logos and trademarks are the property of their respective owners.