# inhaler ▶

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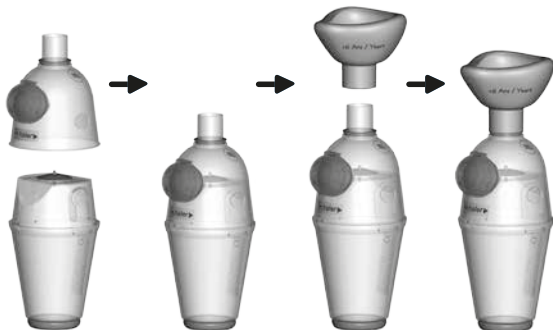
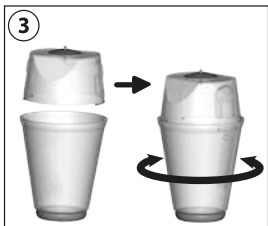
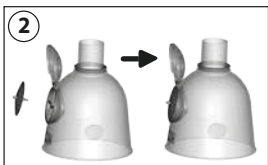
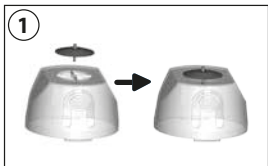
**Fabricado por:**  
**Visiomed Technology Co., LTD.**  
 2<sup>nd</sup> Floor of N°.1 building, Jia An  
 Technological Industrial Park  
 67 District, Bao An  
 518101 SHENZHEN - China

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USER MANUAL

CE

**CLASS I MEDICAL DEVICE CONFORM TO CURRENT EUROPEAN DIRECTIVES.**

**TABLE OF CONTENTS**

<b>1. NOTES ON SAFETY .....</b>	<b>7</b>
<b>2. CHARACTERISTICS .....</b>	<b>7</b>
<b>3. USE.....</b>	<b>8</b>
<b>3.1. DESCRIPTION .....</b>	<b>8</b>
<b>3.2. PRECAUTIONS BEFORE USE .....</b>	<b>8</b>
<b>3.3. USE.....</b>	<b>8</b>
<b>3.4. EXPLANATION OF THE INHALATION PROCESS .....</b>	<b>9</b>
<b>4. CLEANING AND MAINTENANCE.....</b>	<b>9</b>

*The manufacturer reserves the right to modify the technical characteristics of the product without prior notice.*

Dear Client,

We would like to thank you for purchasing this Inhaler® valved-holding chamber. We recommend that you read this instruction manual for the safe and efficient use of your chamber.

The valved-holding chamber facilitates the delivery of aerosolized medication. Inhaler® helps reduce the particle speed and their size. It improves their deposit in the lungs while reducing oropharyngeal deposit.

*For specific information (pulmonary or other), CONTACT YOUR DOCTOR.*

**1. NOTES ON SAFETY**

- Only use this device for its intended purpose as described in this notice.
- This device is suitable for home use.
- This device should always be stored in a clean, dry and dust-free location.
- Clean the device in accordance with cleaning and maintenance instructions.
- Follow the maintenance instructions specified in this manual.
- In the event of a problem, please contact your local retailer.
- Keep this device out of the reach of children.
- This product should not encourage self-medication or modification of your treatment.
- Stop using the device in case of anomaly or malfunction.
- The use of this device is not intended for people (including children) with limited physical, sensory or mental abilities or lack of experience and/or lack of knowledge, unless they are supervised by a person responsible for their safety or have been instructed on the use of the device.

**2. CHARACTERISTICS**

- Double valve system
- Universal adapter compatible with most metered dose inhalers
- Volume 320 ml
- Bisphenol A-free and latex-free
- Replacement valve set
- Three models according to age: 0 to 9 months / 9 months to 6 years / 6 years and over

### 3. USE

#### 3.1. DESCRIPTION

See pictures page 2

1. Position of the central valve
2. Position of the air intake valve
3. Position the top cover on the base and turn a quarter circle to lock.

#### 3.2. PRECAUTIONS BEFORE USE

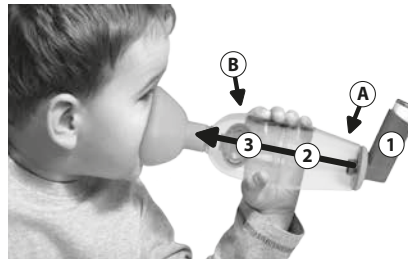
- This product should not encourage self-medication or adjustment of your treatment.
  - Check the assembly of the chamber (see pictures on page 2) and the correct operation of the valves.
- Note: The air intake valve is already positioned when purchased.*
- Check that the chamber is not broken or damaged.
  - It is important to install the chamber calmly, crying (in the case of toddlers in particular) decreases lung deposit and poor sealing of the mask on the face significantly reduces the amount of medication delivered.

#### 3.3. USE

- Install properly or correctly install your child in a sitting position. The child must not have a soother in their mouth.
- Shake the inhalation spray, remove protective cap (refer to instructions of the medication to be administered) and insert the inhalation spray in the lower opening of the inhalation chamber.
- For 0-9 months and 9 months to 6 years, place the facemask over the nose and mouth and ensure an effective seal.
- For 6 years and over, put the mouthpiece into mouth.
- The chamber must be maintained in a horizontal position.
- Press the Inhaler® to release a dose into the inhalation chamber and hold this position while breathing in for about 15 seconds.
- Wait at least 30 seconds before administering a second puff, if necessary (Be sure to respect your doctors prescription).
- Avoid breathing quickly or talking while using the inhalation chamber.

To avoid the appearance of redness and irritation on the skin, it is recommended to rinse the face after the administration of treatments such as corticosteroids.

### 3.4. EXPLANATION OF THE INHALATION PROCESS

1. A dose of medicine is released into the first compartment of the chamber.
2. INHALATION: A puff of product passes into the second compartment during inhalation by the child and is absorbed into the lungs.
3. EXHALATION: When the child exhales, the air exits through the fresh air intake valve without damaging the rest of the dose of suspended medication.

CAUTION: The central valve separates the two compartments and enables airflow from compartment A to B and not the reverse. Check the correct positioning of the latter.

#### 4. CLEANING AND MAINTENANCE

- It is important to clean the various parts of the chamber at least once a week.
- Dismantle the various parts of the Inhaler® chamber (when handling valves, hold by the centre rather than the edges). Hand wash the various parts in lukewarm water with a little mild detergent. Rinse thoroughly and air dry at room temperature, without wiping.
- Once the chamber is dry, store in a dust-free location.
- If the chamber is not in use for an extended period, we recommend you clean it before the next use to remove any dust.
- Change the chamber if you notice that it is damaged or broken.

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©09/2017 VM-IN
**EN** : *Inhaler® is a brand registred by Visiomed Group SA. The constructor keeps the right to modify the photographs and characteristics without previous notice. Registred design. The logos and trademarks are the property of their respective owners.*