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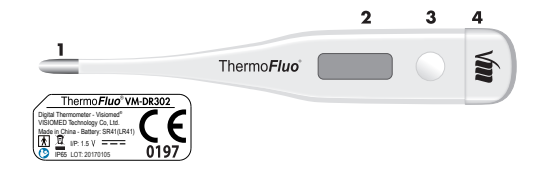
UK INTENDED USE: The product is exclusively meant to be used for human body temperature measuring, suitable for all age infants, children and adults. It can be used for households, and for medical purposes, providing the reference for clinical diagnosis. Measuring areas of body can be axillary, oral or rectum.

Advice : A temperature taken in the rectum is the closest way to finding the body's true temperature. Axillary and oral temperature measurements need over long time periods and have a disadvantage to frequently underestimate measures.

SAFETY & WARNING PRECAUTION

- Please keep the thermomter off the children touch. Prevent the product from children playing, swallowing and eating.
- Do not shock the thermomter or let it fall down.
- The product is 100% waterproof, it is not recommended to immerse it in water or other liquids.

- Do not expose the thermomter to direct sunlight, high temperature/humidity or any extreme environment, or the function will be reduced.
- Stop using the thermomter when it's abnormal or malfunctions.

- Do not use the thermomter for anything but the intended use.
- Do not put the thermomter in contact with diseased, ulcer or bruise skin.
- Taking rectal temperatures can cause internal injury. In general, bleeding from rectum, hemorrhoids, and recent lower intestinal surgery would place the person at high risk for further injury.

- Be sure that the baby or patient has not had a bath or been swaddled (when babies are wrapped tightly for warmth) in the past 20 minutes. This can cause an inaccurate reading.

- Do not drink cold hot beverages or exercise while measuring the temperature.

- THIS MEDICAL ELECTRICAL EQUIPMENT needs special precautions regarding EMC and needs to be installed and put into service according to the EMC DATA table.

- Portable and mobile RF communications equipment can affect MEDICAL ELECTRICAL EQUIPMENT.
- Only use the recommended batteries, do not burn them.
- The measuring result of this product is only for your reference. If you have any doubt, please contact your doctor.

- Do not modify this equipment without authorization of the manufacturer.
- Do not store or operate this equipment under prescribed environmental conditions, as running performance of the device might be affected or the device might not work properly.

- Stop using the thermomter if the LCD displays the "E"; it indicates the sensor circuit detecting an error (such as open or short circuit).
- The default alarm temperature is 37.5°C (99.5°F), the alarm sound is "Di Di Di Di...".
- The measuring range of this thermomter is 32 - 44°C (89.6 - 111.2°F). If the test temperature is < 32°C (89.6°F), LCD display "L"; if the test temperature > 44°C (111.2°F), LCD display "H", the buzzer will alarm.

INSTRUCTIONS BEFORE USE
Clean the tip of the rectal thermometer with soapy water or rubbing alcohol. For hygiene reasons, if the thermometer is ever used in rectal mode it should never again be used for measurement of another part of the body.

OPERATING INSTRUCTIONS
-To switch on the thermometer, press the On/Off button. The appliance emits a beep. The function symbols are displayed on the screen for a few seconds. The result of the last temperature measurement is displayed on the screen for two seconds, then "36.5/97.7°F" and "C" or "F" icons appear in the top right hand corner of the screen. The thermometer is then ready to use.
- To take a rectal temperature?

This is the most reliable way to ensure a fever is not missed.
If necessarily lubricate the tip of the thermometer with a small amount of petroleum jelly. Place your child face down across your lap or on a firm surface. Do place your child face up with their back toward his or her chest. Gently insert the tip of the thermometer about ½ inch (1.25 cm). Never force the thermometer into the rectum. Stop if you feel any resistance. Hold it there until it beeps.

Remove the thermometer carefully. Read the temperature and record it. Temperatures taken rectally usually read a 1 degree F (0.5 degree C) higher than temperatures taken orally. As you remove the thermometer, be sure the disposable sleeve is removed from the patient's rectum, if you used one on the thermometer.

The oral method is not recommended for children younger than 5 years old because it is hard for them to hold the thermometer under their tongue for long enough.

Put the tip under your tongue as far as it can. Close your lips gently around the thermometer. Do not bite the thermometer. Relax and breathe through your nose. Keep the thermometer under your tongue until it beeps.

Wait 20 to 30 minutes after you or your child finish eating or drinking to take an oral temperature.

To take an axillary temperature?
Important: The axillary method is usually used to check for fever in newborns and young children, but it is not as accurate as a rectal temperature. If an axillary temperature does not show a fever but your child feels warm and seems unwell, take a rectal measurement.

Wipe armpit with a dry towel.
Put the tip in you or your child's armpit. Make sure the thermometer is touching skin and not clothes. Squeeze your arm against your body to hold the thermometer in place. Keep the thermometer in your armpit until it beeps.

If you compare temperature between two thermometers, it is possible to have two different readings, even if both are thermometers are within their accuracy tolerances.

Place the thermometer thoroughly before storing it. Dry it and store it in its packaging so it is ready to use next time.

- When taking a measurement, the temperature displayed on the screen changes. The symbol "°C" or "°F" symbol flashes. When the beep sounds 4 times, this means that the temperature has reached a stable level and that the temperature measurement is complete. The measurement is then displayed on the screen for 10 minutes until the appliance is switched off automatically. Press the On button to stop the appliance manually.

- When the battery voltage is 1.25V ± 0.06V, and the LCD display the symbol of ▀; it means you have to replace the battery.
- Remove the battery cover first, pull the inner part 3-5 mm (no more than 10mm), put into the new LR41 battery, keep positive above and make sure the battery well snap into the buckle, then close the battery cover, re-boat, and test.

- "C/F" : Under the shutdown status, press the On/Off button 3s, you can switch between "C" and "F".
- MEMORISING THE DATA: The appliance automatically stores the last temperature in memory so that any variations may be evaluated. When the appliance is switched on, the last measurement is displayed after the initialization test.

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